

Model 8000JFW

Adult FlexiWrap®


Single Patient Disposable Adhesive Sensor Wrap



1. Grasp the blue tab on the FlexiWrap and peel away paper backing just far enough to apply Flex Sensor. Place wrap printed side down onto table.




2. Position Flex Sensor into cutouts on adhesive side of wrap as shown. Press sensor firmly onto adhesive and fully remove paper backing.

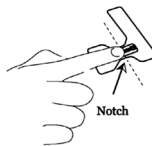


3. Position tip of finger between notches on FlexiWrap.



4. Wrap side flaps of FlexiWrap around sides of the finger. Pull FlexiWrap with sensor over the tip of the finger and adhere wrap to top and side of finger.

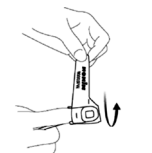

5. Wrap the long flap of the FlexiWrap around the entire finger. Do not wrap too tightly, so as to restrict blood flow.

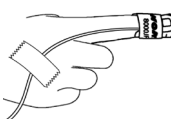

6. For best results, secure sensor cable independently from sensor with medical grade tape on the back of the hand. Be sure the tape does not restrict blood flow.


Indicated Use

Use the FlexWrap Model 8000JFW to secure the NONIN Adult Flex Sensor Model 8000J. The reusable 8000J sensor is for use on adults weighing more than 20 kg (44 lb).


Model 8008JFW

Infant FlexiWrap®


Single Patient Disposable Adhesive Sensor Wrap

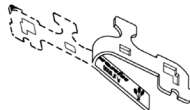


1. Grasp the blue tab and peel paper backing away from adhesive side.

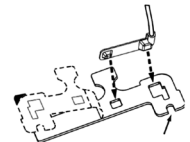

2. Locate cutouts and place sensor on exposed adhesive side.

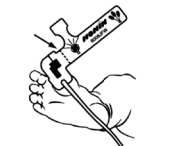

3. Place on large toe (adhesive toward toe) so that dashed line is positioned on the side of the toe.

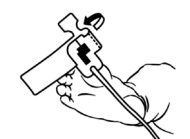

4. Wrap the tape around the toe, and ensure the cutouts are opposed.



5. Wrap the remaining tape around the toe.



6. Secure the sensor cable independently from the sensor along the bottom of the foot toward the heel.


Indicated Use

Use the FlexWrap Model 8008JFW to secure the NONIN Infant Flex Sensor Model 8008J. The reusable 8008J sensor is for use on infants weighing from 2 to 22 kg (4 to 44 lb).


Model 8001JFW

Neonatal FlexiWrap®


Single Patient Disposable Adhesive Sensor Wrap

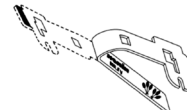


1. Grasp the blue tab and peel paper backing away from adhesive side.

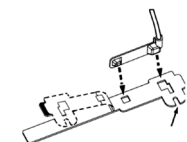

2. Locate cutouts and place sensor on exposed adhesive side.

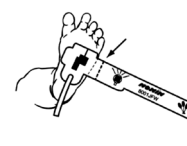

3. Place on foot at base of toes (adhesive toward foot) so that dashed line is positioned on the side of the foot.

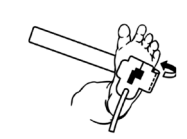

4. Wrap the tape around the foot, and ensure the cutouts are opposed.



5. Wrap the remaining tape around the foot.



6. Secure the sensor cable independently from the sensor along the bottom of the foot toward the heel.

Indicated Use

Use the FlexWrap Model 8001JFW to secure the NONIN Neonatal Flex Sensor Model 8001J. The reusable 8001J sensor is for use on neonates weighing less than 2 kg (4 lb).

NOTE: Do NOT throw away the reusable sensor. Discard the adhesive wrap only.

Nonin Medical, Inc.

13700 1st Avenue North
 Plymouth, MN ■ 55441-5443 ■ USA
 P +1.763.553.9968 TF 800.356.8874
 F +1.763.553.7807 E info@nonin.com

Nonin Medical AB

Fibervägen 2, 82450 Hudiksvall, Sweden
 P +46-(0)650 401500
 F +46-(0)650 401514
 E infointl@nonin.se



QUESTIONS? Contact NONIN Customer Support at
customer.support@nonin.com

1.800.356.8874 or +1.763.553.9968