

When performing a fit test with the Quantifit, the subject inhales through the adapter, pulling in fresh air. The exhalation goes through the exhalation valve of the respirator and not through the fit test adapters. However, you may wish to take every precaution when cross contamination is possible.

When disinfecting fit test adapters for possible contamination, OHD recommends the use of Lysol® Professional spray to combat airborne pathogens. The spray can be applied directly to the portion of the adapter kit that is exposed to the subject's breathing. This is the exposed flat surface below the rust-colored diaphragm. Spray the Lysol and let stand for a minimum of 30 seconds.

Lysol® Professional:

- Effective against HIV-1 (AIDS Virus) and hydrophilic viruses such as Poliovirus Type 1 and Hepatitis A
- Kills germs and eliminates odors on hard, nonporous surfaces
- Formula is tuberculocidal, virucidal, fungicidal and bactericidal
- Prevents odors and growth of damaging mold and mildew
- Meets AOAC efficacy standards for hospital aerosol disinfectants
- Meets the requirements of the OSHA Bloodborne Pathogen Standard for Decontamination

*Kills 99.9% of staphylococcus aureus (staph) and Klebsiella pneumoniae (K. pneumoniae) on hard, nonporous surfaces in 30 seconds.*

