

1 BEFORE YOU BEGIN...



INSTALL BATTERIES



DETERMINE YOUR CURRENT TOOTH COLOR WITH THE SHADE GUIDE



BRUSH & FLOSS TEETH



TAKE A BEFORE PICTURE

2 YOU ARE NOW READY TO BEGIN TREATMENTS



STEP 1 PRETREAT TEETH WITH ACCELERINSE:

Pour rinse up to the 10mL line in the cap of the rinse bottle. Swish vigorously for 10 seconds. Do not swallow.



STEP 2 WHITEN EACH TOOTH WITH THE STAIN LIFTING SERUM:

Shake the Serum for 3-5 seconds. Apply the Serum onto the surface of each tooth you would like to whiten; avoid the gums, as irritation could occur. Re-dip brush as needed. Ensure the front and edges of your teeth are covered. Wait for 20 seconds for gel to dry before allowing teeth to come in to contact with your lips or tongue.



STEP 3 ACTIVATE THE STAIN LIFTING SERUM WITH THE WHITENING LIGHT:

Position the Whitening Light about 1" in front of your teeth. If you wish to rest your lips on the Light's rubber head, apply a thin coat of petroleum jelly to prevent irritation. To avoid saliva build up, place a cotton ball or small tissue inside your lower lip and/or under your tongue. Press the button to turn on the Light, it will beep twice. Hold the light in place for 2 minutes - it will beep again and automatically turn off when complete.

REPEAT STEPS 1-3 for up to 20 treatments based on your customized whitening schedule. There is no need to rinse with water in between treatments.

3 YOUR CUSTOMIZED WHITENING SCHEDULE

How much do I need to whiten?	# of Treatments	How to complete
Never Whitened? Full Regimen:	10-20	All at once or over 7 days
It's been less than 6 months? Touch Up:	10	All at once or over 3 days
Got Plans? Quick Boost:	5	Back to back, one after the next

4 TRACK AND SHARE YOUR PROGRESS



Locate your whitening treatment tracker on the back of your shade guide to track your progress. Once you complete your desired treatments, don't forget to take an after picture and share your whitening story with the Luster community on LusterProLight.com. You can post a review, talk to us on social media, sign up to receive special offers and coupons, and buy your refill kit. **We want to hear from you!** info@LusterPremiumWhite.com • [@LusterPremWhite](https://www.instagram.com/LusterPremWhite)

5 PREPARE FOR COMPLIMENTS

Refer to LusterProLight.com for: Detailed instructions, how-to videos, satisfaction guarantee, testimonials, more tips and tricks, how to order refills, the full list of ingredients, warnings and more! Luster Premium White is a registered trademark of Dentovations, Inc., Boston, MA 02116, 1-800-823-5272, info@LusterPremiumWhite.com

TIPS FOR YOUR BEST SMILE:

Avoid eating or drinking for 30 minutes after treatments. Brush with one of Luster's everyday whitening toothpastes (we recommend Luster Power White) to help maintain your bright smile in between treatments. And YES, our toothpastes contain Fluoride for everyday mouth health, so you can toss that other toothpaste. Available at LusterPremiumWhite.com

Complete 5 whitening treatments every 2-3 months. Be sure to perform a full 10-20 treatments twice a year to rejuvenate your bright smile.

OUR LAWYERS INSIST WE TELL YOU...

Keep out of the reach of children under 12 years old.

Don't use this product if you are allergic to any of the ingredients.

If you're pregnant (CONGRATULATIONS!), Be sure to consult your doctor before using the product.

Avoid contact of the Serum with the lips or gums.

If gum irritation occurs, discontinue use and rinse with water. Irritation should subside in 3-7 days.

Visit LusterProLight.com to find out about our 30-Day Satisfaction Guarantee.



LUSTER PRO LIGHT QUICK REFERENCE GUIDE

Visit LusterProLight.com for detailed instructions and to view from your mobile device, tablet, or computer.