

SUBSCRIBE AND SAVE PROGRAM

WHITENING GEL REFILLS AND MIXING TIPS

To get more use out of your Vanity Planet Active Whitening Professional Grade Teeth Whitening System, join our Subscribe and Save Program to receive additional Dual-Barrel Active Whitening Syringes and Active Mix Tips every two to three months, without a second thought.

Dual-Barrel Active Whitening Syringes and Active Mix are available at vanityplanet.com

Subscribe to our convenient refill program and save 25% on every refill order.

V|P

ACTIVE WHITENING

professional grade
teeth whitening

#VPSMILE | VANITYPLANET.COM

GOOD DAY PEARLY WHITES

In this box, you'll find a brightly polished product that's eager to charm its way into your dental routine. Whether it's whitening your contagious grin or brushing your chompers clean, we're here to give you an insider's scoop on oral care. After all, it's pretty darn important.

We know your smile is already the talk of the town, so our goal is to refresh your regimen and equip you with a super system of oral hygiene necessities. Enjoy the compliments you're sure to get and smile on, folks!

VANITY PLANET

**DON'T FORGET TO SHOW OFF YOUR GOODS BY USING THE
#MYVP AND #VPSMILE HASHTAGS.**

YOUR ACTIVE WHITENING SYSTEM

Vanity Planet's Active Whitening System gives you the power you need to achieve a radiant smile from the comfort of your own home.

It's no mystery that at-home teeth whitening methods can be effective. Our Active Whitening System takes the process to the next level by separating the key ingredients in a Dual Barrel Syringe until the time of application. This ensures that your teeth are treated with the whitening solution when it's at its strongest, most potent state.


Our VP Active Whitening System is produced right here in the United States and designed with your convenience in mind. It's easy to use and quick to deliver noticeable results. Simply attach the Patented Tip and apply one third of the mixture from the syringe to the Silicone Mouth Piece to begin. Once filled, apply the Silicone Mouth Piece to your teeth and leave on for at least 15 minutes, while the whitening gel works its magic. Say goodbye to stains and say hello to a bold, brilliant smile.

TWO SIMPLE STEPS



STEP 1

Remove the cap and attach an Active Mix Tip to the Dual-Barrel Syringe. Dispense the whitening gel into the Silicone Mouthpiece. Use 1/3 of one syringe for each application. Use a new Active Mix Tip for each treatment.

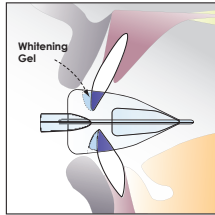

STEP 2

Align the Silicone Mouthpiece with the upper and lower arch of your teeth, and wear for 15 minutes. Rinse your mouth and check for improvements.

Note: Brush your teeth and rinse thoroughly before whitening.

ACTIVE WHITENING CONTENTS

The Active Whitening Professional Grade Teeth Whitening System contains some pretty advanced tools to help you achieve your best smile:

- 2 - Dual-Barrel Active Whitening Syringes
- 6 - Active Mix Tips
- 1 - Silicone Mouthpiece
- 1 - Case for the Silicone Mouthpiece

CAUTION

This product may cause skin irritation. Avoid contact with eyes. Please read directions thoroughly before use.

IMPORTANT

Avoid storing this product in extreme temperatures. Keep out of the reach of children. DO NOT re-use Active Mix tips, use a new Active Mix Tip for each application to insure proper gel activation.

WHITENING GEL INGREDIENTS

Water (Aqua), Glycerin, Hydrogen Peroxide, Potassium Nitrate, Sodium Hydroxide, Disodium EDTA, Sodium Fluoride, Flavor