

Dazzlepro

Say hello to a beautiful smile

DazzleproTM

SMILEPACKSTM - WHITENING SYSTEM

Instruction Manual

DMDP1000



Welcome to

Dazzle**pro**TM

Please take a moment to read these instructions thoroughly so that you can enjoy all the features that make Dazzlepro Smilepacks one of our most effective and easy to use whitening products.

© 2014, Dazzlepro, LLC. All Rights Reserved

Whiten Your Teeth in 3 Easy Steps

1

Remove an upper and lower tray from their packaging.

2

Place the trays on upper and lower teeth, center, and press gently to distribute the gel.

3

Wear trays for 20 minutes each day.

WARNING - Some users may experience sensitivity. This is usually mild and transient. If you experience significant discomfort, stop using this product and consult your dentist. If you have any questions about this product or treatment, please call us at 1.888.358.0330 Do not use if pregnant or breastfeeding. **KEEP OUT THE REACH OF CHILDREN.**