

Gebauer's
PainEase[®]
topical anesthetic skin refrigerant

Exclusive application method cleared by the FDA allows you to apply Pain Ease with cotton.

The cotton application:

- Great for more sensitive patients
- Easy to apply
- Efficacy is equivalent to spraying Pain Ease directly to the skin

Use Pain Ease to temporarily control the pain associated with:

- Immunizations
- Blood draws
- Foreign body removal
- Suturing
- I & D of small abscesses



Pain Ease is available in:

Medium Stream Spray	3.9 fl. oz.	0386-0008-03
Medium Stream Spray	1.0 fl. oz.	0386-0008-04
Mist Spray	3.9 fl. oz.	0386-0008-02
Mist Spray	1.0 fl. oz.	0386-0008-01

IMPORTANT RISK AND SAFETY INFORMATION:

Consult your pediatrician when using on children 4 years old and younger. Do not use on large areas of damaged skin, puncture wounds, animal bites or serious wounds. Do not spray in eyes. Over spraying may cause frostbite. Freezing may alter skin pigmentation. Use caution when using product on persons with poor circulation. Apply only to intact oral mucous membranes. Do not use on genital mucous membranes. The thawing process may be painful and freezing may lower resistance to infection and delay healing. If skin irritation develops, discontinue use. CAUTION: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

The technique is easy!

1. Prep

- Have necessary equipment ready.
- Prep the skin per your protocol.

2. Spray

- Spray 5ml of Pain Ease into a cup & place cotton swab, ball, or gauze into the cup until it is fully saturated with product.

OR

- Spray cotton directly for 10 seconds.

3. Perform

- Hold saturated cotton firmly onto the site for 5-15 seconds.
- Perform the procedure.
- Reapply as needed.

Use this application method on intact skin. If the skin is breached be sure to only use this application method with sterile, disposable cotton balls, cotton swabs or gauze.



4444 East 153rd. Street • Cleveland, Ohio 44128 | 1.800.321.9348 | www.Gebauer.com/painease

© 2018 Gebauer Company. All rights reserved.

1394.1-001