

ACTIVATING BLUETOOTH MODE

1. Press and hold soft key #1 (blue button in upper left corner) to enter settings menu.
2. Using joystick, scroll down to "Bluetooth", right command of joystick to select.
3. Scroll to enable desired iDevice ("iDevice" or "R-Net iDevice2"), right or left command of joystick to turn <on>.
4. Scroll to "Exit" at bottom of screen, right command to "Exit" to settings menu.
5. Scroll to "Exit" at bottom of settings menu, right command to "Exit".
6. Power cycle chair, twice.

PAIRING PERMOBIL JOYSTICK MODULE WITH DEVICE

1. Press and hold soft key #3 (blue button in upper right corner) to enter Bluetooth Mode ("M3"). Option to use power/mode toggle to access Mode 3.
2. Select desired device, if more than one device is active, by scrolling and selecting with right command of joystick.
3. Put in discoverable mode by holding joystick forward for 15 seconds until you hear a beep, repeat by holding joystick in reverse for 15 seconds until you hear a second beep.
4. Enter Bluetooth settings on device being paired, and if required search for new device. Pair with "iDevice 1" or "iDevice 2".



SETTING UP IDevice SWITCH CONTROL ON IOS DEVICE

1. Enable Accessibility Shortcut for Switch Control (Settings > General > Accessibility)
2. Turn on Switch Control
3. Recommend Scanning Style: Manual
4. Add Switches (Settings > General > Accessibility > Switch Control > Switches > Add New Switches > External)
 - Short= 0.1-2.0 sec; Medium= 2.0- 3.5 sec; Long= 3.5-5.0
5. Name Switch and assign command (Example Short Forward = Tap)
6. Turn Switch Control On

EXAMPLE EXTERNAL SWITCH SETUP USING DRIVE CONTROL

Short Command Forward — Tap
 Short Command Right — Move to Next
 Short Command Left — Move to Previous
 Short Command Reverse — Scanner Menu
 Long Command Reverse — Siri

IF USING WITH OMNI DISPLAY

- Follow instructions as outlined above for activating Bluetooth mode and pairing with device.
- "Bluetooth" should show in Omni user menu. User will select "Bluetooth" from user menu through right command of alternative drive control.
- If user only has one device active, that device will be selected and ready for use.
- If more than one device is active, reverse command will scroll down the device list and right command will select device. If user cannot see the Permobil Joystick Module (PJSM), he/she will need to memorize order of devices on the PJSM, and count beeps to know what device is active.