

OBS! WiFi- och e-poståtkomst krävs för konfiguration

Komma igång

The image shows four sequential screenshots of the MyPermobil app activation process:

- 1** Search for the app in the App Store or Google Play. The app is listed as "My Permobil Medical" with an "OPEN" button.
- 2** "Get started" screen with the heading "Select your region". Options include Australia, Canada, Europe, and USA, each with a right-pointing arrow.
- 3** "Get started" screen with a "Bluetooth" permission dialog box. The dialog says: "MyPermobil" Would Like to Use Bluetooth. Bluetooth is needed to be able to activate your wheelchair. Buttons: "Don't Allow" and "OK". Below the dialog, there is a checkbox for "I have read, understood and agree to the End User License Agreement" which is checked.
- 4** "Get started" screen with a checkbox for "I have read, understood and agree to the End User License Agreement" which is checked. Below it is a "Consents and purposes" section with a checkbox for "I give my consent to the usage of my data for product development as described in the privacy notice." which is checked.



MyPermobil



Ladda ned MyPermobil till din Android- eller Apple-telefon


Välj din region

Acceptera Bluetooth™-begäran

Acceptera slutanvändarlicensavtal


OBS! WiFi och åtkomst till e-post krävs för konfiguration

Skapa nytt konto

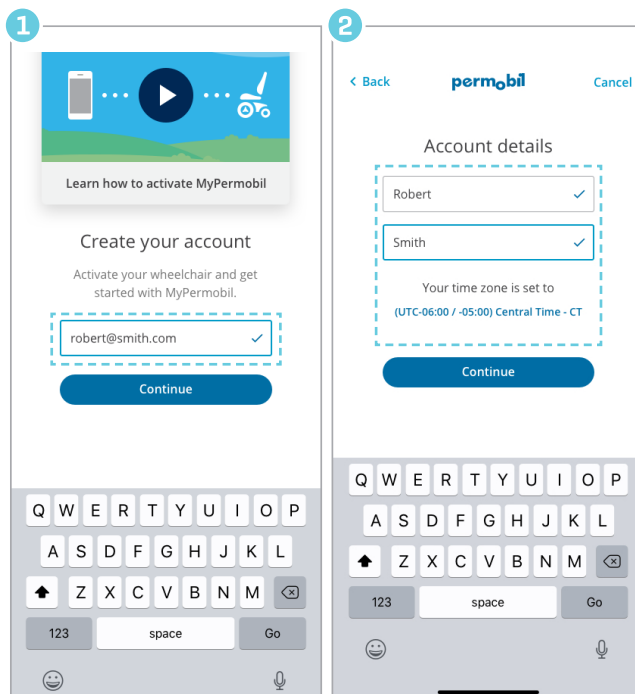


Ange din e-postadress

Ange ditt namn och ställ in din tidszon

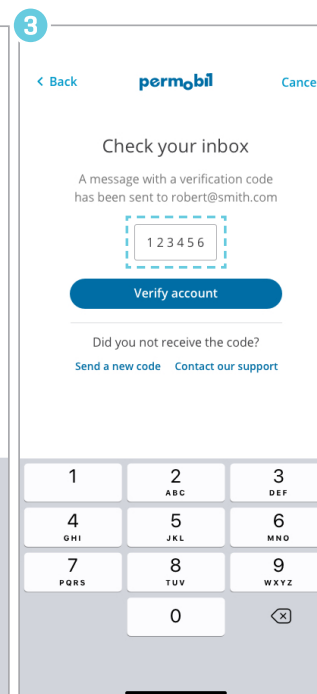


Ange koden du fick via e-post för att verifiera ditt konto

Aktivera

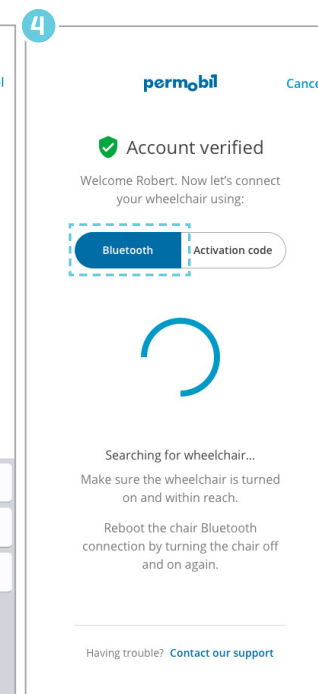


Välj Bluetooth och slå på rullstolen

Om din stol redan är på, stäng av och slå på den igen för att starta om rullstolens bluetooth-anslutning

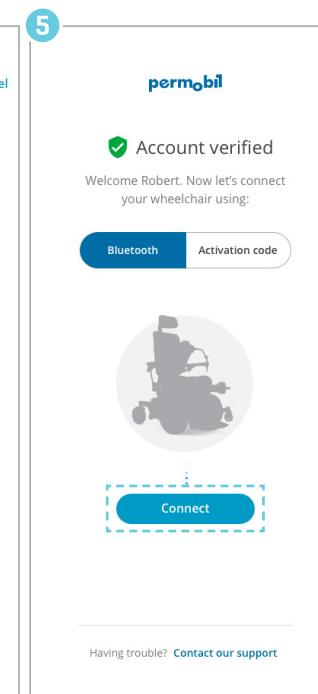

Klicka på "Connect"

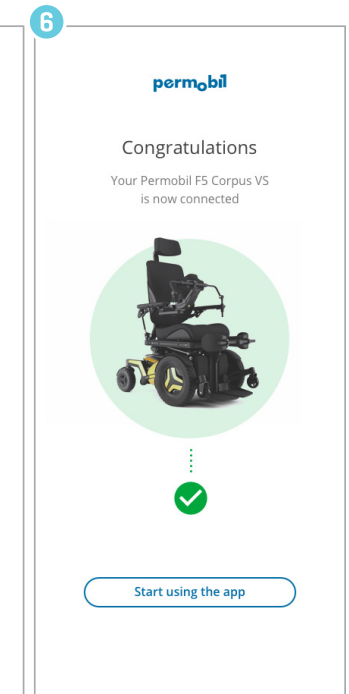

Du är ansluten!