SILVERLON®

safe, strong, simple

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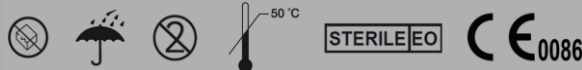
Silverlon® Wound Contact Negative Pressure Dressings

Silverlon® Easy Ag™ Wound Contact Negative Pressure Dressings

EC REP

Emergo Europe, Prinsessegracht 20, 2514 AP The Hague, Netherlands Tel.: +(31) (0) 70.345.8570

LC-CE-IFU-NPD-NPG-WCDNPS ; Rev 07

Silverlon® Negative Pressure Dressings Silverlon® Easy Ag™ Negative Pressure Dressings

English

SILVERLON®

safe, strong, simple

Device Description

Silverlon® Wound Contact Negative Pressure Dressing and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressing contain silver to provide an ancillary, antimicrobial barrier dressings, intended for topical for use in wound management and in negative pressure wound therapy (NPWT), to reduce risk of wound infection.

Available as:

Silverlon® NPD		Silverlon® Easy Ag™	
Code	Size	Code	Size
NPD22	5 x 5 cm	WCD22-NPS	5 x 5 cm
NPD45	10 x 12 cm	WCD45-NPS	10 x 12 cm
NPD58	13 x 20 cm	WCD58-NPS	12 x 20 cm
NPD512	13 x 30 cm	WCD512-NPS	12 x 30 cm
NPD1212	30 x 30 cm	WCD66-NPS	15 x 15 cm
NPG – L	Glove	WCD88-NPS	20 x 20 cm
		WCD1212-NPS	30 x 30 cm

Silverlon® Wound Contact Negative Pressure Dressing and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressing contain silver to provide an ancillary, local antimicrobial effect to reduce the risk of wound infection by organisms sensitive to silver. Ionic silver is a broad spectrum antimicrobial, which has been shown to be effective against a range of pathogens associated with wound infection. Silverlon® dressings have been tested in vitro and found effective against microorganisms such as: Enterococcus faecalis, Staphylococcus aureus, Pseudomonas aeruginosa, Acinetobacter Baumannii, Enterobacter cloacae, Staphylococcus epidermidis and Klebsiella pneumoniae.

Silverlon® Negative Pressure Dressing and Silverlon® Easy Ag™ Negative Pressure Dressing may be used with all types of negative pressure wound therapy porous foam/gauze dressings and equipment.

In vitro testing has shown Silverlon® dressings to be MRI compatible.

Indications

Silverlon® Wound Contact Negative Pressure Dressing and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressing are professional use wound dressings indicated for wound management and also use in conjunction with negative pressure wound therapy (NPWT), for local management of:

- 1st and 2nd degree burns
- Donor and graft site wounds
- Traumatic and surgical wounds
- Diabetic, pressure and venous ulcers
- Infected wounds*

*Silverlon® Wound Contact Negative Pressure Dressing and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressing may be used on those infected wound types listed above. Where used on infected wounds, the infection should be treated as per local clinical protocol.

Silverlon® Wound Contact Negative Pressure Dressings and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressing are indicated for a total duration of contact (i.e. where treatment involves consecutive application of individual dressings) of up to 30 days.

Warnings

- Do not use past expiration date on the product packaging.
- Do not use if pouch is damaged or open.
- Clinicians / Healthcare Professionals should be aware that there are very limited data on prolonged and repeated use of silver-containing dressings, particularly in children and neonates.
- Silverlon® Wound Contact Negative Pressure Dressing and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressings should not be used during pregnancy or lactation because of the lack of data concerning the effects of silver on reproduction.
- Silverlon® Wound Contact Negative Pressure Dressings and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressings are intended for single use only and should not be re-used. Re-use may adversely affect the dressing's performance characteristics and also presents a risk of infection to the patient.
- Do not use petroleum-based ointments or creams under Silverlon® Wound Contact Negative Pressure Dressings and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressings.
- Do not moisten Silverlon® Wound Contact Negative Pressure Dressings and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressings with hydrogen peroxide or povidone iodine.
- Total duration of use of Silverlon® Wound Contact Negative Pressure Dressings and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressings (i.e. where treatment involves consecutive application of individual dressings) should not exceed 30 days.
- Silverlon® Wound Contact Negative Pressure Dressings and Silverlon® Wound Contact Easy Ag™ Negative Pressure Dressing are not intended for direct application to exposed underlying structures (i.e. blood vessels, bone, tendons, nerves).