

Thermometer Instructions for Building Entry

1. After entering the building, **wait at least 5 minutes** before taking your temperature.
2. Press the **Measure/power button** to power on the thermometer.
3. Press and hold the Measure/power button while you **press the thermometer to your forehead and slowly move it from one side to the other for about 2-3 seconds**– make sure the thermometer is touching your skin the entire time.
4. **Release the measure button and remove the thermometer** from your forehead, **you will hear a “beep”** and can now **read the digital temperature**.
 - a. If your temperature reads 99.9° or lower, proceed into the building.
 - b. If your temperature reads 100.0° or higher, wait 2 minutes and then do the test again.
 - c. If your temperature reads 100.0° or higher on the second test;
 - i. **Employees:** contact your manager and EHS (David Steinbacher at 206-375-0427 or Shyam Mahanti at 425-205-7355).
 - ii. **Non-employees:** contact your AGC Biologics contact
5. After use, sanitize the thermometer using the wipes provided.

