



Electric Epidural Positioning Device Quick Start Guide

1) Position patient in an upright sitting position at the edge of the bed



2) Roll the EPD into place and lock the casters



3) Using the UP/DOWN buttons on the hand control, adjust the EPD to proper height



4) Have the patient lean forward, resting on face rest and arm rest



5) Adjust the Face Rest to desired position




6) Adjust the Arm Rest by pressing toggle switch and sliding into position - release to lock



7) Using BOTTOM UP/DOWN buttons on hand control, adjust the foot rest



8) Adjust chest support by pressing down toggle switch and sliding with gentle pressure - making a nice curve of the back

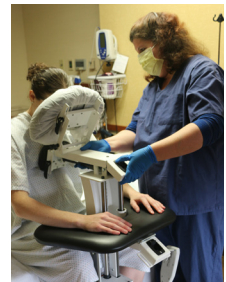

9) After the procedure, have patient lean back into a normal sitting position



10) Unlock the casters by lifting up on brake lever, roll the EPD away from patient

