



PHS Medical
by Pivotal Health Solutions

Epidural
Positioning
Device

Quick Start Guide



QUESTIONS?

phsmedicalsolutions.com

800-743-7738



Epidural Positioning Device

Quick Start Guide

1) Position patient in an upright sitting position at the edge of the bed



2) Roll the EPD into place and lock the casters



3) Adjust the EPD to proper height



4) Have the patient lean forward - adjust the Face Cradle



5) Adjust the Arm Rests



6) Adjust the Foot Rests




7) Loosen the knob to push the chest cushion into the chest - making a nice curve in the back



8) Perform the Epidural procedure



9) Help the patient lean back, off the EPD

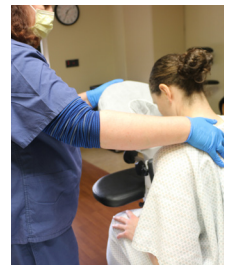

10) Unlock the casters by lifting up on brake lever, roll the EPD away from patient



PHS Medical
by Pivotal Health Solutions

QUESTIONS?

phsmedicalsolutions.com
800-743-7738