

Patient ID entry

Alaris® System



The Alaris® PC unit offers the option to enter and display a patient identifier. Enter the patient identifier at start-up when responding Yes to New Patient or at any time through the System Options menu.

USE LAST 6 DIGITS OF PATIENT ACCOUNT NUMBER:
N5011013546 → enter **013546**

How to enter a patient identifier at start-up on the Patient ID entry screen
(if applicable, select the letter grouping and the individual letter):

1. Enter up to 16 alphanumeric characters
2. Press Page Down to access the Z, hyphen, underscore or space
3. Press Clear to clear an entry
4. Press Cancel to back up one character at a time
5. Press Confirm to verify the entry (A)

How to enter a patient identifier through System Options
(even after an infusion has been started):

Press Options

Select Patient ID (B)

Patient ID Entry	
A	A-E
B	F-J
C	K-O
D	P-T
E	U-Z

>Enter Patient ID and Press CONFIRM

A	EXIT	CONFIRM	PAGE DOWN
----------	------	---------	-----------

System Options 1 of 3	
Display Contrast	
Patient ID	
Clinician ID	
Power Down All Channels	
Anesthesia Mode	

>Select and Option or EXIT

B	EXIT	PAGE DOWN
----------	------	-----------

For product support, contact Customer Advocacy at 888.812.3266 or customerfeedback@carefusion.com.

For technical support, contact Instrument Technical Support at 888.812.3229.

For product orders, contact Customer Order Management at 800.482.4822.

CareFusion
San Diego, CA

carefusion.com

