

SSCOR Duet

A Powerful Substitute for Wall Vacuum in an Emergency



The **SSCOR Duet**® is designed for the hospital crash cart, patient transport, surgi-center backup, long term care facilities and more.



Easily Attached to Walls or Carts
With the optional **SSCOR Duet** retention bracket.



Powerful and Portable
With adjustable vacuum.



Universal Canister Holder
Enables you to use the canister of your choice for standardized inventory control.

SSCOR

Complete information at www.sscor.com.

The **SSCOR Duet** battery care system will automatically charge and monitor itself.

Leave it plugged in all the time. The battery cannot be overcharged.



SSCOR Duet with Retention Bracket - Model 2314B



SSCOR Duet Flat Back - Model 2314

The **SSCOR Duet**® contains a battery care system that is designed to gently hold the battery at optimal charge, without harmful overcharging.

While plugged into AC power, the internal battery control system will:

- Bring the battery to full charge, illuminating the green light on the side of the unit
- Drop back to "float" when the battery reaches full charge, eliminating damage or destruction of the battery caused by overcharging

The battery care system monitors the battery during operation, informing the operator of anticipated running time and will disconnect a depleted battery to protect the battery from destructive deep discharge.

The **SSCOR Duet**'s automatic capacity check will warn you when the battery begins to lose capacity. (Eventually all batteries lose capacity and need to be replaced.)

SSCOR will replace a **SSCOR Duet** battery that does not operate to specifications within three years of a **SSCOR Duet** purchase.

Specifications:

Performance:	>30lpm airflow at the end of a patient tube; <-50mmHg - >-550mmHg
Dimensions:	17" x 9" x 5.25"
Weight:	10.05 lbs. (Model 2314), 10.65 lbs. (Model 2314B)
Charger:	Internal automatic charger offers fast charge up to full and then gentle float to maintain charge. LED's indicate charging status and operation mode.
Power:	12v DC Sealed Lead Acid Battery, 100 - 240V AC Input
Run Time:	45 minutes +/- 10% when battery fully charged at full capacity

Standard items included with product:

Canister, 6' patient tube, HI-D® suction tip, AC power cord and Quickstart Guide

SSCOR

SSCOR, Inc.
11064 Randall St. Sun Valley, CA 91352
+1 818 504 4054
Lit-2314-3 Rev A

e-mail: sales@sscor.com www.sscor.com
© 2018 SSCOR and HI-D are registered trademarks of SSCOR, Inc.
U.S. Patent No. D270948