

i DID YOU KNOW *that instruments that come into contact with mucous membranes or non-intact skin increase the risk of transmission of infectious pathogens and require a higher level of care when reprocessing?*

- Step 1:** Pour fresh solution of Preempt™ HLD5 or CS20 into clean soaking tray, no activation or dilution is required.
- Step 2:** Use AHP2 (for HLD5) or AHP7 (for CS20) test strips to validate the required concentration of hydrogen peroxide during the re-use period up to 14 days.
- Step 3:** Discard of the solution up to 14 days, if the solution is visibly soiled, or if indicator strip fails test.
- Step 4:** Wash and dry instrument soaking tray before filling with new solution.

Product: _____


Lot Number: _____

Bottle Expiry Date: _____

Date Solution Poured

Re-Use Day	Date	Test Time	Test Results	Initial	Comments
1			Pass – Fail		
2			Pass – Fail		
3			Pass – Fail		
4			Pass – Fail		
5			Pass – Fail		
6			Pass – Fail		

Date Solution Poured

Re-Use Day	Date	Test Time	Test Results	Initial	Comments
7			Pass – Fail		
8			Pass – Fail		
9			Pass – Fail		
10			Pass – Fail		
11			Pass – Fail		
12			Pass – Fail		
13			Pass – Fail		
14			Pass – Fail		
 Make NEW Solution!					


 Questions / Comments?
1-800-387-7578 | ViroxProbeauty.com

