

The data generated by CalproSmart™ Crohn's is owned by the patient and their healthcare team.

Nr 1 - November 2017



**TEST YOUR CALPROTECTIN LEVELS
AT HOME** with the CalproSmart™ Home Test Kit



CALPRO AS
Arnstein Arnebergsvei 30
N-1366 Lysaker
Norway
Tel: +47 400 04 279
mail@calpro.no
www.calpro.no



CalproSmart


helps you and your healthcare team to monitor your calprotectin levels more closely between hospital appointments. You can accurately self test your calprotectin levels at home using the CalproSmart™ Home Test Kit and the CalproSmart™ app.

This booklet works with the CalproSmart™ app to show you how to perform the test step-by-step – it is important that you follow the test instructions carefully to get a valid test result. Your healthcare team will advise how often you should do the test.


Each time you perform the test using the CalproSmart™ smartphone app, your result is automatically synched with your calprosmart.com portal and your healthcare team is notified of the result.

Your result history can be viewed at any time in ‘My health measures’ in the CalproSmart™. These charts will allow you to monitor your disease levels at home and can be a valuable aid for your healthcare team in deciding further examination like endoscopy.

However, diagnosis shouldn't be made on one test result alone. You must consult your healthcare team before changing your medication on the basis of any CalproSmart™ test results.

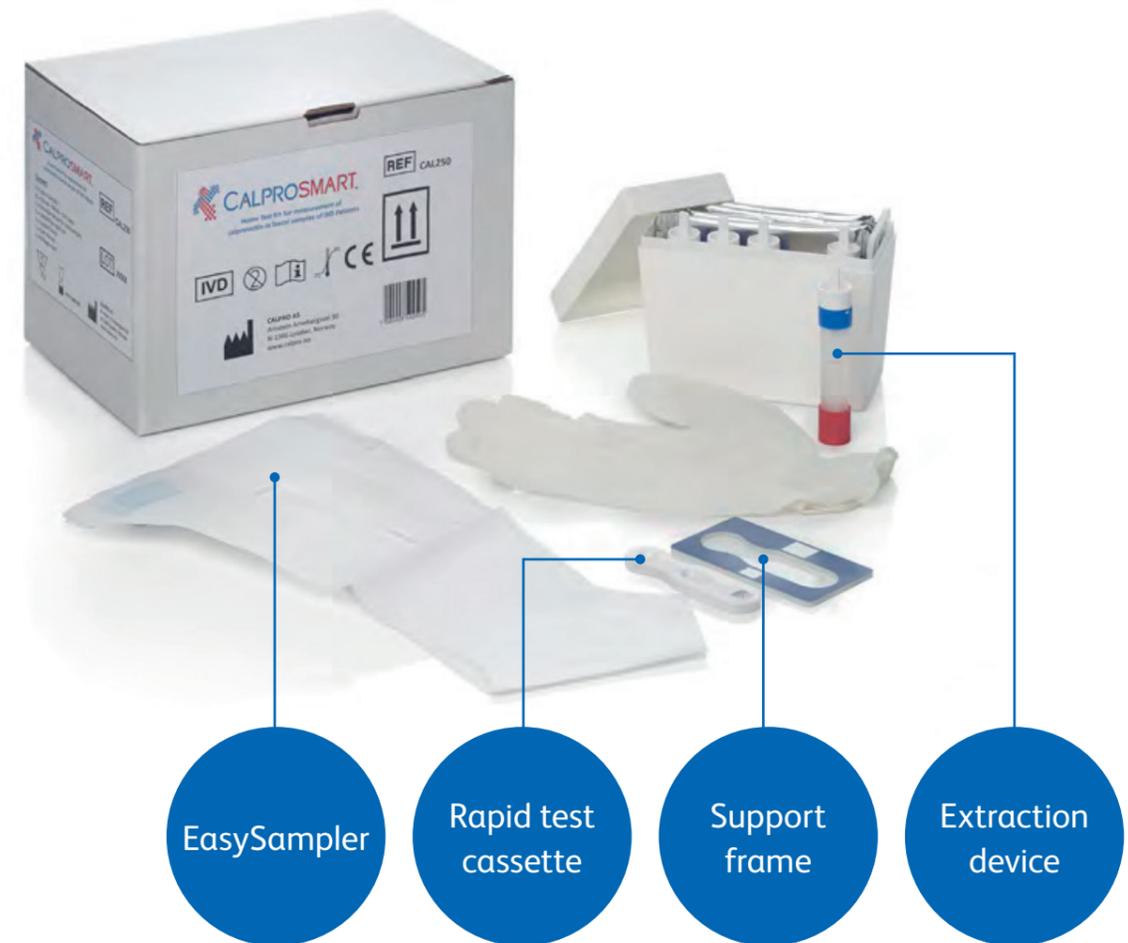

How to set up the CalproSmart app

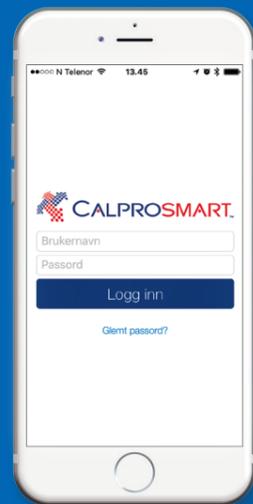

- 1 Your healthcare team will register you for CalproSmart™ through the calprosmart.com portal.
- 2 You will receive an email confirming your new account and a link to set a new password.
- 3 The app can be downloaded directly from from iTunes App Store or Google Play, simply search for 'CalproSmart'.
- 4 You will need to accept the End User Licence Agreement to start using the app.

What you will need

2 clean tissues or toilet paper	To clean the toilet bowl before you do the test and another to wipe the tip of the extraction device.
A cup or glass	To hold the extraction device base when you are extracting a sample.
EasySampler	This disposable paper sampler makes it easier for you to collect a stool sample.
Extraction device and gloves	The faecal extraction device is designed to collect the exact size and type of sample required.
Support frame (reusable)	The reusable plastic frame holds the rapid test cassette and enables the smartphone app to read the test result.
Rapid test cassette (single use)	These individually wrapped, disposable cassettes make it easy to perform the test on your faecal sample.
CalproSmart™ smartphone app	The smartphone app helps you through each stage of the test and will read the final test result.

Store your unopened rapid test cassettes in the fridge (between 2–8°C) and avoid exposure to high temperatures and direct sunlight. Always check the expiry date.