

CONGRATULATIONS!

Your Zirlux Acetal partial has been custom made for you by dental professionals to fit comfortably and look natural. With proper care, your appliance should provide years of use. In order to keep your ***Zirlux Acetal partial*** looking good as new, it is important that you maintain good oral hygiene and take proper care of the appliance.

- ✓ After meals, rinse appliance to remove any food partials.
- ✓ Clean your appliance daily using an antibacterial denture cleanser.
- ✓ Only use a soft bristle brush and scrub gently to avoid damage.
- ✓ Never use any abrasives such as toothpaste or powdered household cleanser to clean your appliance.
- ✓ Handle appliance with care.
- ✓ Contact your dentist if you experience any discomfort. Your dentist will advise you when to come in for a regular checkup.



*Photo courtesy of Dental Arts Laboratories, Inc. (DAL) of Peoria, IL