



	<p>This symbol indicates that dangerous voltage constituting a risk of electric shock is present within this unit.</p>
	<p>This symbol indicates that there are important operating and maintenance instructions in the literature accompanying this unit.</p>

	<p>CAUTION RISK OF ELECTRIC SHOCK DO NOT OPEN</p>	
<p>CAUTION: TO REDUCE THE RISK OF ELECTRIC SHOCK, DO NOT REMOVE COVER. NO USER SERVICEABLE PARTS INSIDE. REFER SERVICING TO QUALIFIED SERVICE PERSONNEL.</p>		


THIS INSTALLATION SHOULD BE MADE BY A QUALIFIED SERVICE PERSON AND SHOULD CONFORM TO ALL LOCAL CODES. READ AND FOLLOW THE SAFETY INSTRUCTIONS BEFORE ATTEMPTING THIS INSTALLATION.

Note to CATV installer: This reminder is provided to call the CATV system installer's attention to Article 820-40 of the NEC that provides guidelines for proper grounding and, in particular, specifies that the cable ground shall be connected to the grounding system of the building, as close to the point of cable entry into the room as practical.

 **WARNING: OXYGEN ENVIRONMENT DO NOT** use in an oxygen tent or oxygen chamber. Such use may cause a fire hazard.




DANGER: ARM RECOIL HAZARD The safety brake pin must remain in the SAFETY BRAKE PIN HOLE whenever the television set is removed from the arm or when the arm is removed from the wall bracket to prevent the arm from springing open.

 **WARNING:** To avoid the hazards of fire or electric shock, DO NOT expose this TV to rain or moisture.

Installing Swivel to Monitor:
 (Cables not shown for clarity)


1. Detach swivel cover from channel by removing two (2) PDIPFHMS83238 SCREWS.
2. Securely attach swivel channel to monitor with four (4) M4 X 6mm TORX screws. Reattach swivel cover using same 2 screws.

PDIPFHMS83238

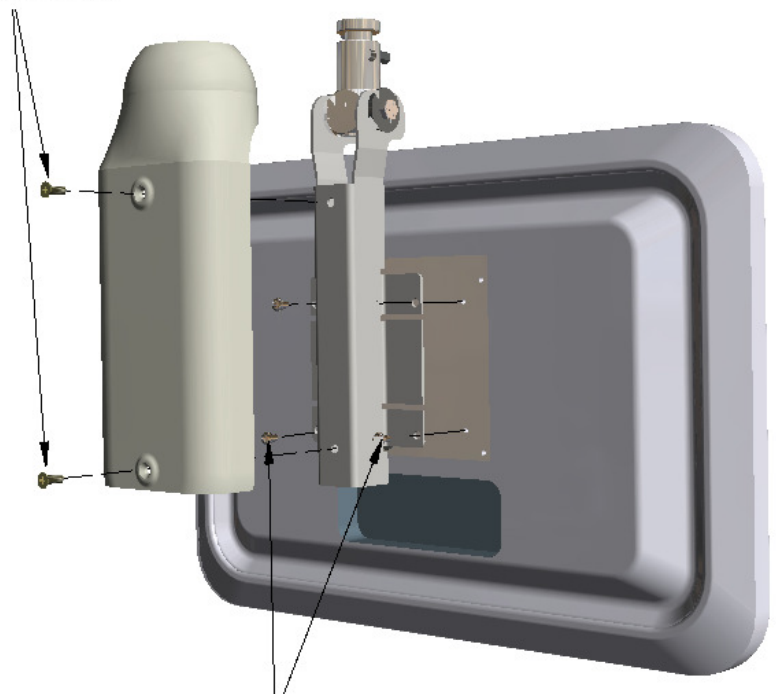


M4 X 6mm TORX

- To attach the monitor unit to the arm, begin by removing two (2) PDIPFHMS83250 screws as shown. Lift nose of the arm unit and also remove the PDI2520AN acorn nut and PDISLW25 washer. This will allow you to remove the PDI-194 retainer from the arm unit.

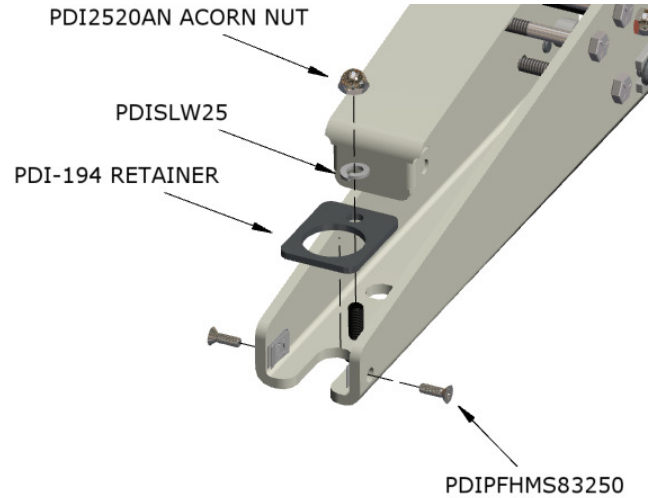

- Slide the monitor unit onto the nose of the arm as shown and securely reattach the retainer to the unit and the arm using the same washer and acorn nut.

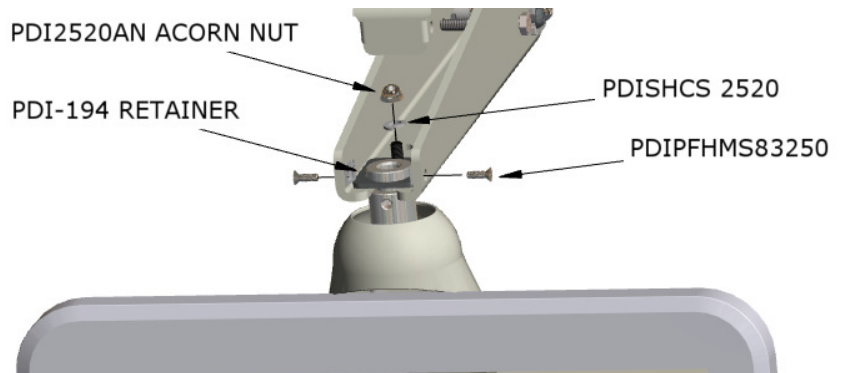

- Close the nose cover of the arm and securely reattach the PDIFHMS83250 screws.

