

Classification of ultrasound probe and appropriate reprocessing between patients prevent infection transmission

- Classification of ultrasound based on use and patient contact site
- Local and regional guidelines form a consensus for when to high level disinfect probes



How to decide when to high level disinfect your ultrasound probe

Risk based classification for disinfection

It is very important that reusable medical devices are safely reprocessed (cleaned, disinfected/sterilised) between each patient to prevent infection transmission.

Classifying ultrasound probes based on risk associated with intended use provides a rational framework for associated decontamination methods (sterilise, high level disinfection and low level disinfection).

One such framework is the Spaulding Classification for determining the associated decontamination method required for reusable medical instruments according to the degree of infection transmission risk.¹

Devices that only contact intact skin are considered 'non-critical' requiring low level disinfection (inactivates bacteria, some fungi and some viruses). Devices that contact unhealthy skin or mucous membranes are considered 'semi-critical' requiring high level disinfection (inactivates bacteria, fungi, viruses with a few spores remaining). Devices which contact or enter sterile tissue, body fluids or vasculature are considered 'critical' requiring sterilisation (inactivates all viable microorganisms).

Guidelines released locally by the Royal Society & College of Radiographers and the British Medical Ultrasound Society, Ireland's Health Service Executive, NHS Scotland and regionally by the European Committee for Medical Ultrasound Safety and the European Society of Radiology, all make use of the widely adopted Spaulding Classification (or derivatives thereof) to form a consensus for when to high level disinfect ultrasound probes based on their use and patient contact sites.²⁻⁷ Table 1 overleaf summarises their recommendations.

While the Spaulding Classification is a good general classification system for devices, ultrasound probes have specific usage and reprocessing factors that also need to be considered (Table 1).

Nanosonics Europe Limited

Unit 2, Linfit Court, Colliers Way
Clayton West, Huddersfield HD8 9WL
United Kingdom
T: 01484 860581
E: ukinfo@nanosonics.co.uk
www.nanosonics.co.uk

Nanosonics Europe GmbH

(EU Representative)
Poppenbütteler Bogen 66
22399 Hamburg
Germany
T: +49 40 46856885
E: info@nanosonics.eu
www.nanosonics.eu

SPAULDING CLASSIFICATION	PROBE CAN CONTACT	DISINFECTION/ STERILISATION REQUIREMENT*	EXAMPLES OF PROCEDURES
Critical	<ul style="list-style-type: none"> • Sterile areas • Tissues • Blood • Body cavities 	<ul style="list-style-type: none"> • Sterilisation + (cover optional. If used, cover must be sterile) <p>OR</p> <ul style="list-style-type: none"> • High level disinfect + sterile cover 	<ul style="list-style-type: none"> • Intraluminal • Intraoperative • Biopsy • Puncture techniques • Non-invasive probe-guided cannulation/ venepuncture** • Non-invasive probe-guided wound assessment**
Semi-Critical***	<ul style="list-style-type: none"> • Internal organs via non-sterile natural orifices • Blood contact expected occurs during procedure • Mucous membranes • Non-intact, broken skin 	<ul style="list-style-type: none"> • High level disinfect + cover (preferably sterile) 	<ul style="list-style-type: none"> • Transoesophageal echocardiography (TOE) • Transvaginal ultrasound (TV) • Transrectal ultrasound (TR) • Non-invasive probe-guided cannulation/ venepuncture • Non-invasive probe-guided wound assessment
Non-Critical (Non-invasive****)	<ul style="list-style-type: none"> • Only healthy, intact skin 	<ul style="list-style-type: none"> • Cleaning and/or low or intermediate level disinfection (cover optional) 	<ul style="list-style-type: none"> • Abdominal ultrasound, healthy skin • Pelvic ultrasound, healthy skin

Table 1. Classification of reusable medical devices and recommended disinfectant method and efficacy according to specific guidelines.

- * According to the Spaulding Classification, critical medical devices should be sterilised. However, ultrasound probes are generally thermolabile so guidelines recommend high level disinfection along with sterile cover use.
- ** HSE Ireland and NHS Scotland classify critical procedures such as cannulation and venepuncture as semi-critical.
- *** Semi-critical ultrasound probes can include those used in semi-invasive as well as non-invasive procedures.
- **** HSE Ireland and NHS Scotland classify non-critical medical devices as non-invasive.

Many ultrasound probes cannot be sterilised and accordingly most guidelines permit high level disinfection in lieu of sterilisation for 'critical' probes so long as a sterile sheath is also used. Both endocavitary and surface probes are often used in conjunction with a sheath or condom. Importantly, guidelines state that use of a sheath (or condom) does not replace the need for disinfection as sheaths (and condoms) can have microscopic tears and can break during use.

It is important that this classification system is applied before the procedure commences. This means that information about what tissues or body sites may be contacted, is required in advance, so that an appropriately disinfected or sterilised probe can be selected. Furthermore, there is a strong preference in guidelines for automated, validated reprocessing procedures.

The need for adequate reprocessing

Ultrasound usage in the United Kingdom has increased significantly with diagnostic procedures in England rising over 24% from 7 million to 9.3 million over five years.⁹

Ultrasound probes, as with all reusable medical devices, carry infection risks and a number of infections have been

attributed to improper use and reprocessing. A fatal hepatitis B infection contracted during a transoesophageal procedure led to a Medicines and Healthcare products Regulatory Agency alert requesting all users to review their decontamination procedures.⁹

Furthermore, a retrospective cohort study by NHS Scotland and Health Protection Scotland demonstrated an infection risk when high level disinfection was not performed for endocaviatry procedures. The study found that patients undergoing transvaginal scans with low level disinfection probes, were 41% more likely to have positive bacterial cultures compared to matched patients not undergoing ultrasound procedures.¹⁰

The need for standardised infection control procedures is paramount to improving patient outcomes in healthcare settings. With increasing applicability of ultrasound and subsequent increase in ultrasound procedures, infection control becomes increasingly more important.

Proper application of the Spaulding Classification system and adherence to local guidelines will ensure an adequate level of disinfection is performed and will protect patients from infection risk.

References

1. Spaulding EH. (1968). Chemical disinfection of medical and surgical materials. In: Lawrence C, Block SS, eds. Disinfection, sterilization, and preservation. Philadelphia: Lea & Febiger, 1968:517-31.
2. The British Medical Ultrasound Society (BMUS). Guidelines for professional ultrasound practice. 2017.
3. Health Service Executive (HSE) Quality Improvement Division. HSE Guidance for Decontamination of Semi-critical Ultrasound Probes; Semi-invasive and Non-invasive Ultrasound Probes: 2017.
4. Health Protection Scotland (HPS), Health Facilities Scotland (HFS), National Services Scotland (NHS). NHSScotland guidance for decontamination of semi-critical ultrasound probes; semi-invasive and non-invasive ultrasound probes. 2016; 24.
5. Kollmann, C. and K. Salvesen (2017). "Best Practice recommendations for cleaning and disinfection of ultrasound transducers whilst maintaining transducer integrity." European Committee for Medical Ultrasound Safety (ECMUS).
6. Nyhsen, C. M., et al. (2017). "Infection prevention and control in ultrasound - best practice recommendations from the European Society of Radiology Ultrasound Working Group." Insights Imaging 8: 13.
7. Health Protection Scotland (HPS). National Infection Prevention and Control Manual: 2017. Accessed: <http://www.nipcm.hps.scot.nhs.uk>.
8. National Health Service England (NHS). Diagnostic Imaging Dataset Statistical Release.
9. Medicines and Healthcare products Regulatory Agency (MHRA). Medical Device Alert: Reusable transoesophageal echocardiography, transvaginal and transrectal ultrasound probes (transducers). 2012.
10. Health Protection Scotland (HPS) (2017). NHSScotland Risk Based Recommendations for the Decontamination of Semi-Invasive Ultrasound Probes: Risk of infection following semi-invasive ultrasound procedures in Scotland, 2010 to 2016: 25.