# Sonex-HL®

ONLY FOR USE WITH THE TROPHON® EPR

**Distributed in US by:**  
**Nanosonics, Inc.**  
11793 Technology Lane  
Fishers, IN 46038, USA  
support@nanosonics.us

**Distributed in Canada by:**  
**Hansamed Ltd.**  
Mississauga, Ontario  
Canada, L5N 8G4  
www.hansamed.net

**USA/ Canada Customer Support:**  
1-844-876-7466

**Manufactured for:**  
**Nanosonics Limited**  
14 Mars Road, Lane Cove  
NSW 2066, Australia  
+61 2 8063 1600  
info@nanosonics.com.au

[www.nanosonics.us](http://www.nanosonics.us)

**MADE IN AUSTRALIA**

REF N00037 -NNA

L00070-WEB 3.0 01/2016

Fig. a


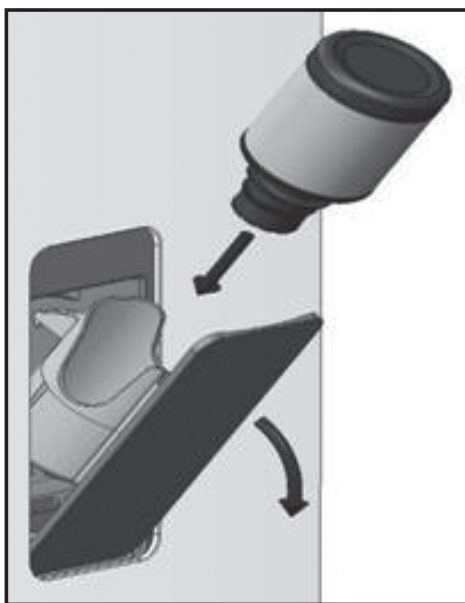
Fig. b
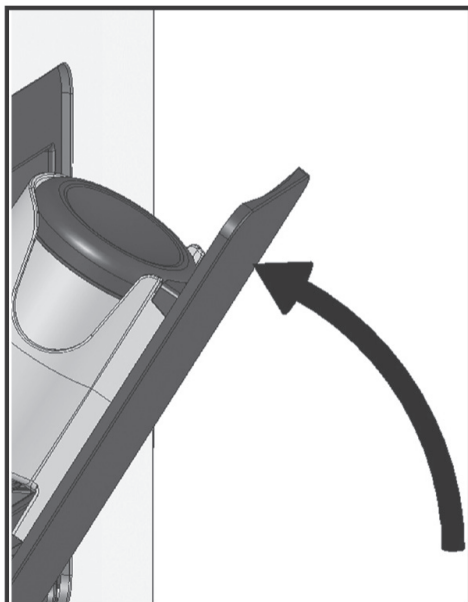
## EN Instructions for Use

### Sonex-HL®

#### Single use disinfectant delivery cartridge, containing multiple doses of disinfectant.

The following must be read in conjunction with the trophon EPR User Manual. This cartridge is specifically designed for use with the trophon EPR to deliver the correct dosage of disinfectant to ensure high level disinfection.

Each cartridge contains 80mL of 35% w/w hydrogen peroxide solution. Use of disinfectants with differing composition or concentration to that specified may result in damage to the device, probe and/or reduced effectiveness of the disinfection process.

#### 1. Warnings (Refer to the Safety Data Sheet for more information)

**IMPORTANT:** Cartridges are pierced at the top and on the side near the bottom upon insertion of a new cartridge into the device. A small amount of disinfectant may remain in the cartridge, even when it has been fully used. Follow the instructions carefully to avoid injury.

- Always wear disposable gloves when handling disinfectant cartridges.
- Avoid contact with eyes and skin.
- Do not manually pierce the cartridge.
- Do not attempt to open or use a damaged or distorted cartridge.
- Do not attempt to refill or reprocess the empty cartridge after removal from trophon EPR. This could lead to personnel injury and/or equipment/device malfunction or damage.
- Strong Oxidizer: Contact with other material may cause a fire.
- Corrosive: Harmful if swallowed or inhaled. May cause eye and skin burns. Prolonged or repeated skin contact may cause dermatitis.

#### 2. Emergency Guidelines

- **Spillage:** Always wear personal protective equipment. Turn OFF the device at the power switch and at the mains power switch. Dilute with plenty of water and discharge according to Occupational Health and Safety Guidelines for your institution.
- **Eye contact:** Rinse immediately with copious amounts of cold water, also under the eyelids, for at least 15 minutes. Do NOT rub or keep eyes closed.
- **Skin contact:** Flush skin with copious amounts of soap and water for at least 15 minutes. Remove contaminated clothing and shoes. Consult a physician if irritation persists.
- **Ingestion:** Do NOT induce vomiting. If victim is conscious and alert, rinse mouth thoroughly with water. Never give anything by mouth to an unconscious person. Call a poison control centre.
- **Inhalation:** Remove person to fresh air. If symptoms persist, call a physician.

#### 3. Warning Symbols

	Caution		Corrosive – 8
	Single Use Only		Oxidiser – 5.1
	No air transport		Wear gloves

#### 4. Inserting the Disinfectant Cartridge

**IMPORTANT:** Do NOT use a distorted cartridge. Always check the expiration date on the cartridge before use.

- Cartridge door will automatically open when the cartridge needs replacing. Do NOT force the cartridge door open as this may damage and render the device non functional.
- Screen message: *CARTRIDGE EMPTY. REPLACE CARTRIDGE NOW.*
- Press the button under OK to open the cartridge door.
- Remove the cap from the cartridge
- Place the cartridge NECK FIRST into the holder. Do NOT force the cartridge into the holder (see figure a).
- When situated in place correctly, the bottom of the cartridge should be in line with the top of the holder (see figure b). The door should close smoothly. Do NOT force the door to close.
- The device is now ready for use. Once the door is closed, it will automatically lock and cannot be reopened until the cartridge is emptied or purged.

#### 5. Cartridge Removal


Cartridge removal is required when the cartridge has expired or is empty. When the cartridge expires, the device will need to be purged to ensure all disinfectant is removed. Refer to Section C3 in trophon EPR User Manual for further information on purging.

- Screen message: *REMOVE AND EMPTY WASTE CONTAINER.*
- Screen message: *LOAD WASTE CONTAINER.*
- Screen message: *CARTRIDGE EMPTY. REPLACE CARTRIDGE NOW.*
- The empty cartridge can now be removed.
- Wear gloves. Lift cartridge out by touching the areas exposed whilst the cartridge is in the holder and avoid touching pierced areas.
- Do NOT shake or change the orientation of the cartridge. For empty cartridge disposal see section 6: Disposal of Used Cartridge.
- Refer to section 4 to replace with a new cartridge.

#### 6. Disposal of Used Cartridge

Follow procedures of your institution for the disposal of CORROSIVE or OXIDIZING materials or dispose of as per local environmental and government regulatory requirements.

#### 7. Storage

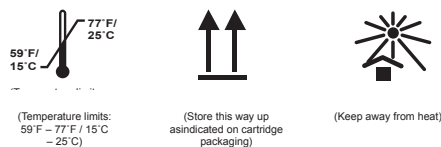- Store cartridge in ALL original packaging until use.
- Store cartridge upright as indicated on cartridge packaging.
- Cartridges may distort or explode if not properly stored.

#### 8. Other Symbol Definitions

	Batch No.		Consult instructions for use
	Product No.		Expires

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