

trophon®

# Companion Wipes

Partner up to streamline your ultrasound probe reprocessing

## Cleaning and drying steps in high level disinfection (HLD) cycle

**READY-TO-USE trophon Companion Wipes** simplify cleaning and drying steps to promote efficient HLD workflows.



STEP  
1



Use the Cleaning Wipes to remove protein, gel, microbial load and other soil from probe. Wipe until visibly clean.

STEP  
2



Dry probe with a Drying Wipe and load into trophon for HLD.

STEP  
3



During room turnover, use the Cleaning Wipes to wipe down the patient environment including ultrasound monitors, keyboards probe holders and cables, patient contact surfaces, as well as the trophon touch screen and handle.

STEP  
4



Remove probe from trophon. Wipe with a Drying Wipe then store in a clean cover.

**Note:** Suggested workflow.

USA & Canada. Nanosonics, Inc. 7205 E 87<sup>th</sup> Street, Indianapolis, IN 46256 USA  
T: 1-844-TROPHON. 1-844-876-7466, W: [www.nanosonics.us](http://www.nanosonics.us), E: [info@trophon.com](mailto:info@trophon.com)

Headquarters. Nanosonics Limited, 14 Mars Road, Lane Cove, NSW 2066, Australia  
T: +61 2 8063 1600, E: [info@nanosonics.com.au](mailto:info@nanosonics.com.au) MM0827-US-FY V01.

**nanosonics**  
Infection Prevention. For Life.