

trophon[®]

Companion Wipes

Multi-purpose low level disinfection

Cleaning and low level disinfection (LLD) for non-critical items

trophon Companion Cleaning Wipes are multi-purpose for efficient cleaning and LLD of non-critical ultrasound probes and equipment such as consoles, monitors, keyboards and probe holders.



STEP
1



Use the Cleaning Wipes to remove protein, gel, microbial load and other soil from probe. Wipe until visibly clean.

STEP
2



Use the Cleaning Wipes to LLD the probe.

STEP
3



During room turnover, use the trophon Cleaning Wipes to LLD ultrasound monitors, keyboards, probe holders, probe cable and plug.

Note: Suggested workflow.

USA & Canada. Nanosonics, Inc. 7205 E 87th Street, Indianapolis, IN 46256, USA
T: 1-844-TROPHON. 1-844-876-7466, W: www.nanosonics.us, E: info@trophon.com

Headquarters. Nanosonics Limited, 14 Mars Road, Lane Cove, NSW 2066, Australia
T: +61 2 8063 1600, E: info@nanosonics.com.au MM0823-US-FY V02.

nanosonics
Infection Prevention. For Life.