

BPH affects more than 500 million men worldwide, with many men suffering from symptoms of enlarged prostate. You no longer have to be one of them!

You have taken the first step in improving your symptoms through a proven approach called the UroLift® System treatment. This booklet will not take the place of a consultation with your physician, but it will answer some of the questions you may have before your UroLift System treatment.


Benign Prostatic Hyperplasia, or BPH, is a very common condition in which the prostate enlarges as men get older. Over 70% of men in their 60s have symptoms of BPH. While BPH is a benign (noncancerous) condition, it can cause loss of productivity and sleep, depression and decreased quality of life.


WHAT IS AN **ENLARGED PROSTATE?**


The prostate is a male reproductive gland, about the size of a walnut, that produces fluid for semen. The prostate surrounds the urethra, which is the tube that carries urine from the bladder out of the body. As the prostate enlarges, it presses on and blocks the urethra, causing bothersome urinary symptoms.

BPH symptoms include:

- Frequent need to urinate both day and night
- Weak or slow urinary stream
- A sense that you cannot completely empty your bladder
- Difficulty or delay in starting urination
- Urgent feeling of needing to urinate
- A urinary stream that stops and starts

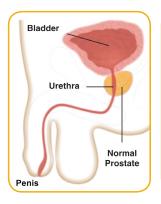

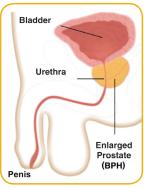

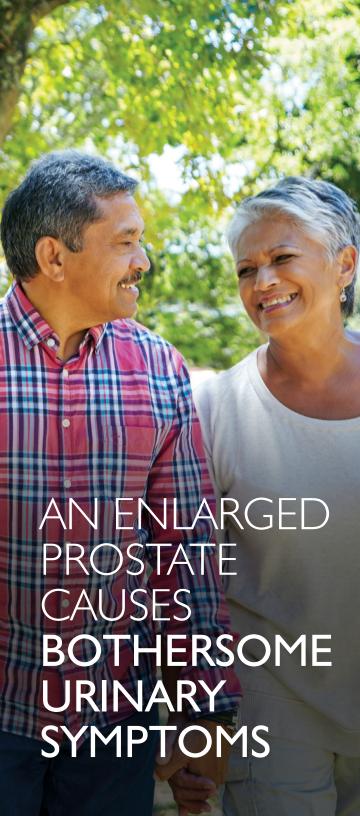

TREATMENT OPTIONS

LESS INVASIVE



Watchful Waiting


Medications

UR()

ADVANTAGES OF THE UROLIFT® SYSTEM

- Rapid symptom relief, better than reported for medications¹
- Risk profile better than surgical procedures such as TURP²
- **⊘** Preservation of sexual function
- Return to normal activity in days not months

^{2.} Sonksen, EU Urol, 2015 BPH6 Study

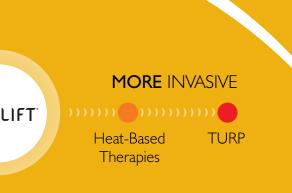

- Increased quality of life
- Ooes not require BPH medications
- Over 20 clinical study publications;5-years published clinical data

Most common side effects are light blood in the urine, some pain or discomfort when urinating, some increased urge to go and discomfort in the pelvis that typically resolve within 2-4 weeks after the procedure.

SIGNIFICANT SYMPTOM IMPROVEMENT WITHOUT SIGNIFICANT DOWNSIDES


L.I.F.T. Randomized Study Results
Compared to AUA Guidelines at 1 Year





EjD

Sexual Dysfunction Symptom Improvement a

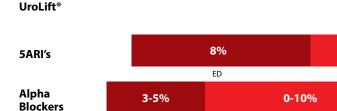

ED

0% Ejaculatory or No instances of de novo, sustained erecti

I Irwig & Kolukila JSM, 2001 http://www.ncbi.nlm.nih.gov/pubmed/21418145

² Roehrborn, J Urol 2013; 190: 2161-2167

³ Sonksen, Eur Urol 2015; 68: 643-652

⁴ AUA Guidelines 2003, 2010, 2014, which address a range of outcomes across alfuzosir 5 L.I.F.T. IDE Study. Roehrborn. | Urology 2013




Erectile Dysfunction le or ejaculatory dysfunction observed.

n, doxazosin, tamsulosin and terazosin for ABs and only finasteride for 5ARIs

THE UROLIFT® SYSTEM HAS HELPED OVER 35,000 MEN AROUND THE WORLD

WHAT IS THE UROLIFT® SYSTEM?

The UroLift® System is a revolutionary approach to treating BPH that lifts and holds the enlarged prostate tissue out of the way so it no longer blocks the urethra. It is the only BPH procedure that does not require ongoing BPH medication, cutting, heating or removal of the prostate tissue.

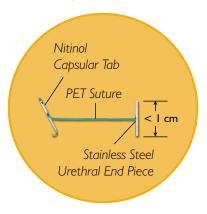

Actual Size of UroLift Implant

HOW THE UROLIFT® SYSTEM WORKS

STEP 1

The UroLift
Delivery Device is
placed through the
obstructed urethra
to access the
enlarged prostate.



STEP 2

Small UroLift Implants are permanently placed to lift or hold the enlarged prostate tissue out of the way and increase the opening of the urethra. The permanent Implants are delivered through a small needle that comes out of the UroLift Delivery Device and into the prostate.

STEP (3)

The UroLift Delivery Device is removed, leaving an open urethra designed to provide symptom relief.



WHAT TO EXPECT DURING THE TREATMENT

During the UroLift® treatment the doctor places small, permanent implants into the prostate to lift and hold the enlarged tissue out of the way.

- During the treatment you will be situated comfortably on a table with your legs slightly elevated. There will be a monitor that the physician watches during the treatment.
- Depending on where you have your treatment performed (in the hospital or doctor's office), your doctor will administer local or general anesthesia to help maximize comfort during the treatment.
- The doctor gently introduces a thin tube containing a tiny camera through the urethra to determine where to place the UroLift implants.
- Once the proper location is determined, the doctor will insert several tiny implants into the prostate. You will hear a "click" during the placement.
- This will be repeated on both sides until the doctor achieves a sufficient opening for urine to pass easily. During the treatment you may feel some pressure and the urge to urinate. This is to be expected.





"SYMPTOM RELIEF IS ACHIEVED RAPIDLY, WITHIN TWO WEEKS, AND AT ONE MONTH IS VERY SIMILAR TO WHAT CAN BE EXPECTED AT FIVE YEARS"

- Claus Roehrborn MD¹



WHAT TO EXPECT AFTER THE TREATMENT

Immediately following the treatment, you will be asked to urinate. If you can successfully urinate, you will be allowed to go home. If you have difficulty urinating, your doctor may insert a catheter to help you void, and you will be given instructions on when to have the catheter removed. Most patients do not require a catheter after having the UroLift System treatment.

Some men may experience discomfort after the treatment. You may have soreness in the lower abdomen, and it may be uncomfortable to sit. You may experience the need to urinate more frequently and with greater urgency. You may have some blood in your urine, including passing an occasional blood clot. These are all normal reactions to the treatment. Most of these symptoms will resolve within 2-4 weeks – this is normal.



Urologist's view of obstructed prostate



Unblocked prostate after UroLift System treatment



Is the treatment performed in an office or hospital?

The UroLift® System treatment can be performed in either an office or hospital. Please speak with your urologist to decide which location would be best for you.

When can I resume my usual activities?

Many men resume normal activities shortly after the treatment. Your doctor will discuss any restrictions and your specific situation after your treatment.

Do I need to continue taking my BPH medications after the treatment?

A benefit to getting a UroLift System treatment is that you may no longer have to take BPH medications after the treatment.

Does my prostate continue to grow even after I get a UroLift System treatment? What if I need another treatment?

Your prostate may continue to grow after the treatment; however, one unique aspect of the UroLift System is that it does not preclude retreatment or other BPH treatments, should that be needed and desired in the future.

How quickly can I expect to have symptom relief?

Patients typically show significant improvement in their symptoms by 2 weeks after the treatment.

How long will my results last?

The UroLift Implant is a permanent implant, and the treatment is intended to be permanent. Durability has been shown to at least 5 years in U.S. clinical data.²

^{1.} Roehrborn 2013 LIFT Study

^{2.} Roehrborn, et al. Can J. Urol 2017

Is my sexual function affected by the UroLift System treatment?

Clinical studies have proven the UroLift System tretament preserves sexual function. This is a unique benefit of the UroLift System treatment compared with other BPH therapies such as TURP, laser, heat-based therapies and even medication.

Does my insurance cover the treatment?

The UroLift System treatment is paid for by Medicare and many private insurers. Contact your insurance provider for your specific coverage information.

How many UroLift System treatments have been performed?

Over 35,000 men worldwide have been treated with the UroLift System over the last 10 years. Over 20 clinical study publications demonstrate that the UroLift System is safe and effective.

What happens if the implants need to be removed?

The implant is made up of standard surgical implantable materials: a nitinol tab, a stainless steel tab, and polyester suture that holds the two tabs together. Your doctor can simply remove the implant, if needed.

I have read that many men don't need a catheter after they receive UroLift implants. Is this true?

Most patients do not require a catheter after having the UroLift System treatment. Your Urologist will determine this if you need one after a voiding test. If a catheter is placed, it is typically only overnight and is removed the next morning.



90%
OF PATIENTS
REPORTED
IMPROVEMENT
IN THEIR
CONDITION AT
ONE MONTH

NOW WHAT?

Are you ready to take control of your BPH symptoms? Don't settle for frustrating urinary symptoms and daily medications. The UroLift® System treatment may be the right choice for you if you are seeking a treatment solution. So now what?

- Talk to your doctor to see if you are a candidate for the UroLift System treatment
- 2. Coordinate with your doctor's office to confirm your insurance coverage
- 3. Schedule your UroLift System treatment
- 4. Enjoy your new life!

UROLIFT°

"I noticed results the same day. It was a very pleasant experience." - Larry White (patient)

"The procedure has allowed me to regain my quality of life. I have no urgency, no frequency and I'm thoroughly satisfied with the results."

- Stephen Richardson, MD (patient)

FOR MORE INFORMATION VISIT www.UroLift.com

Follow us on 9 @UroLift

Find us on facebook.com/UroLift

NeoTract, Inc. is dedicated to developing innovative, minimally invasive and clinically effective devices that address unmet needs in the field of urology. Our initial focus is to improve the standard of care for patients with Benign Prostatic Hyperplasia (BPH), a broadly underserved market. Our first product is the UroLift® System, a minimally invasive device designed to treat lower urinary tract symptoms (LUTS) due to BPH.



NeoTract, Inc. Pleasanton, CA 94588 Tel: (925) 401-0700 • (877) 408-9628