

## BH2012 EZ Rx Drug Keeper Instructions for folding containers

The instructional folding video can be watched at: [www.drugkeepers.com](http://www.drugkeepers.com)

1) Expand so that a square is formed



2) On bottom side, push down panels one and then two until they lock into place




3) Hold down panel three



4) Fold the side ears in on panel four



5) Fold tab on panel four and push through the slot on panel three

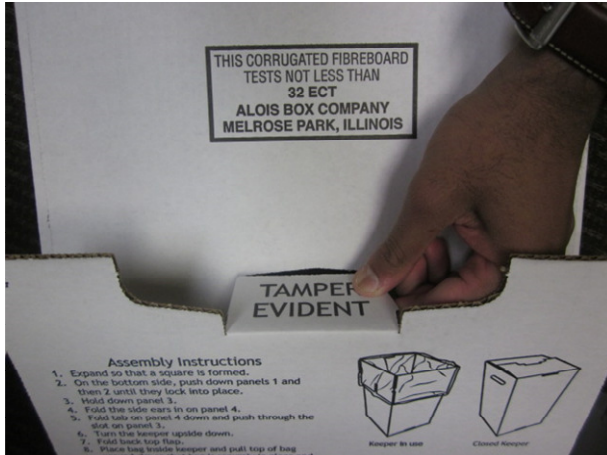

6) Turn the keeper right side up



7) Fold back top flaps



8) Place bag inside keeper and pull top of bag around rim so that bag is securely in place and top flaps stay open



9) Keeper is now ready for use



10) Once keeper is full, collapse bag and repeat steps 2-5 on top side to securely close container



**Reorder Number: BH2012**

**If you have any questions, suggestions, or to order, contact:**

**Whitney Medical Solutions**

**6153C Mulford St., Niles, IL 60714**

**1-800-DEVICES or [info@whitneymms.com](mailto:info@whitneymms.com)**