




Clean Protection for Tablets and Electronics

INSTRUCTIONS FOR USE

Tablet Size Covers



Step 1:
Insert tablet into cover with device facing tape tab.

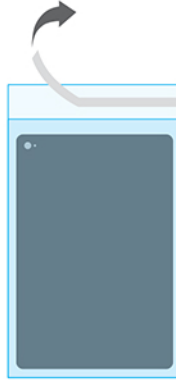

Step 2:
Remove release liner from tape.



Step 3:
Fold tab over device, pull taut, and smooth down tape.

TIP: Pulling tab taut will help remove wrinkles from cover and improve touch screen performance.

CAUTIONS:

- 1) NOT STERILE. Not for use in a sterile field.
 - 2) Do not reuse. Change when cover becomes contaminated.
- WARNING:** Cover limits air flow to the device. Some devices require significant air flow for cooling. Verify that device will not overheat while in Stay-Clean cover before extended use.



"The Experts in Germ-Free-Digital"
Made in USA
800-DEVICES
www.whitneymedicalsolutions.com

TRY OUR OTHER eSHIELD PRODUCTS

eShield Stay-Clean Non-Sterile Covers		eShield Sterile Covers	
Size	Order No.	Size	Order No.
Smartphone	EC3100	Smartphone	EC2100
Small Tablet	EC3310	Digital Camera	EC2200
Medium Tablet	EC3320	Tablet	EC2300
Large Tablet	EC3330	SLR Camera	EC2400