

To provide an effective whitening treatment using the uPhoria Whitening System, all users should become familiar with the MOST EFFECTIVE WAYS to use the uPhoria system. Visible results will follow.

The uPhoria system uses VIBRATION (NOT light) through the gel to create microbubble cavitations (1000's of them every second). These bubbles FIZZ with great force (14500psi) and heat (8500F). Accordingly, the micro-bubbles drive hydrogen peroxide into the stained pores of the enamel. It works AMAZINGLY WELL, if the system is used properly.

Advice:

1. SETUP – Please be sure your Stand is set up properly. All parts tightened, Mouthpiece installed firmly. Use suction for comfort.
2. USE - practice and experience the treatment for yourself; Feeling a consistent tingle, pulsing hiss from the Mouthpiece and a gentle warming sensation are all indications that the system is working as intended;
3. SOAK PAD - assure that foam pad is saturated. Squeezing gel out during placement will reduce whitening efficacy due to reduction of number of micro implosions;
4. CONTACT – CONTINUOUS contact with gentle pressure is the only way to get system will deliver hydrogen peroxide with micro-implosions. Too much pressure applied to the back of the ultrasonic unit (silver piece that is connected to the white mouth piece), can cause the foam pad to dry out in spots and cause excessive heat;
5. Patient selection - my teeth will not get same whitening that the 25yo will get. However, everyone will see some results.

It is recommended that the in-office uPhoria Teeth Whitening Treatment is supplemented by the uPhoria @Home. You can provide this 'take home' consumable product to your customers as an enhancement to their care and maintenance after getting the uPhoria Boost in the office.

uPhoria FAQs

1. How often can I get a uPhoria Ultrasonic Teeth Whitening treatment?

We recommend that you get an Ultrasonic Teeth Whitening treatment once every 6 months after you get your teeth cleaned with your dentist.

2. How can I maintain my white teeth after my treatment?

To extend the effect of your whitening, we recommend that you uPhoria @Home to provide further results and to maintain your new bright smile.

3. Will this cause tooth sensitivity?

The uPhoria Ultrasonic Teeth Whitening System does not cause tooth sensitivity. Because the whitening gel contains alcohol and hydrogen peroxide, you may experience slight irritation on your gums if they are exposed to the whitening gel during your treatment. The irritation should not last long but if you notice any discomfort during your treatment, be sure to alert the hygienist so that they can make proper adjustments to protect your gums.

4. Can you get a uPhoria Ultrasonic Teeth Whitening treatment if you are pregnant?

We recommend that you first consult your doctor before receiving a treatment if you are pregnant.

5. Can you use it with veneers / caps?

The uPhoria Ultrasonic Teeth Whitening System is completely safe to use on veneers and dental restorations. The treatment will work on veneers, but will only whiten to remove any surface stains on the dental restoration.

6. How long does one treatment last?

It is recommended to get one Whitening treatment every 6 months following your cleaning with your dentist.

7. Does this remove deep stains or just surface stains?

The proprietary formulation has been developed to remove both deep stains and surface stains.

8. What testing has been done on this Whitening Treatment?

This product has been through rigorous testing to be FDA Compliant. It was also been through Enamel Safety Testing to ensure that it will not damage or weaken teeth.

9. Are the Treatment Kits reusable?

The treatment kits are for single use only. Please dispose of all parts of the kit once the treatment is complete.

10. Can I do more than one treatment at a time?

We recommend that you only do one treatment per dentist visit. Multiple treatments don't necessarily mean that your teeth will continue to whiten as the initial treatment removes the most stains.

11. Do I need to wait after the treatment before drinking or smoking?

We recommend that you refrain from eating, drinking or smoking for 30 minutes after your treatment is completed.