


Albahealth Anti-Embolism Stockings are color-coded to make proper sizing easy. Color coding is found in the following locations:

- **Toe Port**
  - **Packaging**
  - **Tape Measures**
  - **Quick Reference Chart Column**
- Small      ● Medium      ● Large  
● X-Large      ● 2X-Large      ● 3X-Large

### KNEE HIGH Measurement Guide

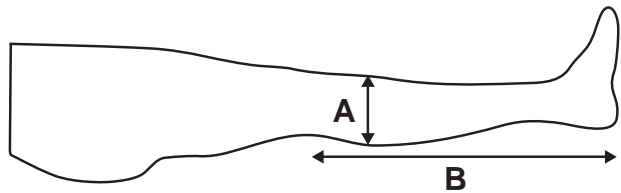

**Size**  
Circumference of "A"

**Length**  
Distance of "B"


**CAUTION: DO NOT APPLY** these stockings if any measurement exceeds the maximum recommendation. Consult a physician. Federal law restricts this device to sale by or on the order of a physician.

KNEE LENGTH		Small	Medium	Large	X-Large	2X-Large	3X-Large
SHORT (Yellow Top)	Calf	10" to 12"	12" to 15"	15" to 17"	17" to 19"	19" to 21"	—
	Length	12" to 14"	13" to 15"	14" to 16"	15" to 17"	16" to 18"	—
REGULAR (White Top)	Calf	10" to 12"	12" to 15"	15" to 17"	17" to 19"	19" to 21"	21" to 24"
	Length	14" to 16"	14" to 16"	16" to 18"	16" to 18"	17" to 19"	17" to 19"
LONG (Blue Top)	Calf	10" to 12"	12" to 15"	15" to 17"	17" to 19"	19" to 21"	—
	Length	16" to 18"	16" to 18"	18" to 20"	18" to 20"	19" to 21"	—

**INDICATIONS:** The use of Anti-Embolism Stockings is indicated in situations where there is likely to be impaired venous return from the lower extremities as in: Surgical patients, Pregnancy, Varicose veins, Geriatric/bedfast patients, Chronic venous insufficiency, Edema secondary to cast removal, Patients recovering from phlebitis

**CONTRAINDICATIONS:** Anti-embolism stockings are not recommended for patients with: severe arteriosclerosis or other severe ischemic vascular diseases, including absence of pedal pulses; local skin disorders such as dermatitis, open ulcerations, infections, recent vein ligation, skin graft or gangrene in which stockings would interfere; massive peripheral edema associated with pulmonary edema; and severely deformed extremity.

### THIGH AND WAIST HIGH Measurement Guide

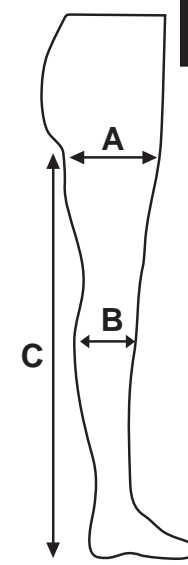

**Size**  
Circumference of "A" and "B"

**Length**  
Distance of "C"

**CAUTION: DO NOT APPLY** these stockings if any measurement exceeds the maximum recommendation. Consult a physician. Federal law restricts this device to sale by or on the order of a physician.

THIGH LENGTH		Small	Medium	Large	X-Large	2X-Large	3X-Large
SHORT (Yellow Top)	Calf	10" to 12"	12" to 15"	15" to 17"	17" to 19"	19" to 21"	—
	Thigh	20" to 25"	20" to 25"	20" to 25"	25" to 32"	25" to 32"	—
	Length	up to 29"	up to 29"	up to 29"	up to 29"	up to 29"	—
REGULAR (White Top)	Calf	10" to 12"	12" to 15"	15" to 17"	17" to 19"	19" to 21"	21" to 24"
	Thigh	20" to 25"	20" to 25"	20" to 25"	25" to 32"	25" to 32"	30" to 36"
	Length	29" to 33"	29" to 33"	29" to 33"	29" to 33"	29" to 33"	29" to 33"
LONG (Blue Top)	Calf	10" to 12"	12" to 15"	15" to 17"	17" to 19"	19" to 21"	—
	Thigh	20" to 25"	20" to 25"	20" to 25"	25" to 32"	25" to 32"	—
	Length	33" to 36"	33" to 36"	33" to 36"	33" to 36"	33" to 36"	—
WAIST LENGTH		Small	Medium	Large	X-Large	2X-Large	3X-Large
REGULAR (White Top)	Calf	10" to 12"	12" to 15"	15" to 17"	17" to 19"	19" to 21"	—
	Thigh	20" to 25"	20" to 25"	20" to 25"	25" to 32"	25" to 32"	—
	Length	29" to 33"	29" to 33"	29" to 33"	29" to 33"	29" to 33"	—
LONG (Blue Top)	Calf	10" to 12"	12" to 15"	15" to 17"	17" to 19"	19" to 21"	—
	Thigh	20" to 25"	20" to 25"	20" to 25"	25" to 32"	25" to 32"	—
	Length	33" to 36"	33" to 36"	33" to 36"	33" to 36"	33" to 36"	—