RESMED

ClimateLine™ / ClimateLine^{MAX}™ / ClimateLine^{MAX}™ Oxy

AIR TUBING

User Instructions

English • Deutsch • Français • Italiano • Español • Português • Nederlands • Svenska • Dansk • Norsk • Suomi • 简体中文 • 繁體中文


English

Thank you for choosing the ClimateLine, ClimateLine^{MAX} or ClimateLine^{MAX} Oxy (hereafter 'ClimateLine') air tubing. The ClimateLine is heated air tubing that delivers the desired temperature at the mask and enables you to use Climate Control. Climate Control maintains humidity at comfortable levels. Please read this entire user instructions before using your air tubing.

Note: The ClimateLine is designed for use only with the H5i. If you are using the S9 without the H5i, please use your SlimLine or Standard air tubing.


Setup

1. Remove the rubber cover from the rear of the H5i.
2. Align the ClimateLine with the ClimateLine connection port so that the lock symbol is facing up. Push the ClimateLine firmly onto the air outlet.
3. Rotate the ClimateLine clockwise until it clicks into place.
4. Connect the assembled mask system to the free end of the ClimateLine air tubing. Refer to your S9 Welcome Guide for general setup information.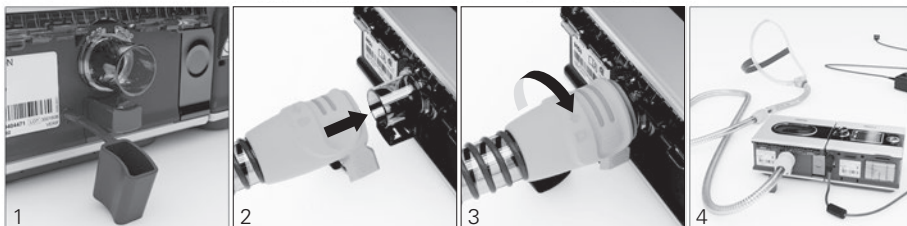

If you use the ClimateLine^{MAX} Oxy with supplemental oxygen, perform the following steps in addition to the ones described above:

1. Remove the cap from the oxygen connector.
2. Connect the oxygen supply tubing to the oxygen connector. Ensure that it is attached firmly.

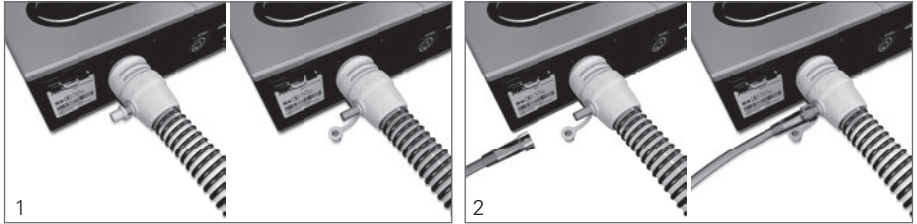Notes:

- Before adding oxygen, familiarize yourself with the specific warnings relating to the use of supplemental oxygen. These can be found at the end of your S9 Information Guide.
- If you use the ClimateLine^{MAX} Oxy without supplemental oxygen, ensure that the cap of the oxygen connector is attached firmly.

Climate Control

Climate Control is an intelligent system that controls the H5i and the ClimateLine to deliver constant, comfortable temperature and humidity levels during therapy. Designed to prevent dryness of the nose and mouth, it automatically maintains the temperature and relative humidity while you sleep.

Climate Control modes

Climate Control Auto

The recommended (default) setting for Climate Control is Auto mode. In Auto mode, you have the ability to adjust air temperature (61-86°F (16-30°C)) depending on preference. Based on your selected air temperature, Climate Control automatically adjusts the humidifier output to maintain a constant, comfortable humidity level of 80% relative humidity while protecting against raintout. In Auto mode the default air temperature is set to 80°F (27°C).

Climate Control Manual

If enabled by the clinician, Climate Control can also be set to Manual. In this mode, air temperature and humidity can be set independently, providing you greater flexibility but without the rainout protection guaranteed in Auto mode. In this mode, air temperature (61–86°F (16–30°C)) and humidity level (0–6) can be set independently. If rainout occurs, either increase air temperature or decrease the humidity setting.

Note: Images depict the US product. Temperatures will be displayed in celsius for all other regions.

Cleaning and maintenance

You should regularly carry out cleaning and maintenance as described in this section.

Daily

Disconnect the air tubing and hang it in a clean, dry place until next use.

Notes:

- Do not hang the air tubing in direct sunlight as it may harden over time and eventually crack.
- Do not wash the air tubing in a washing machine or dishwasher.

Weekly

1. Remove the air tubing from the H5i and the mask.
2. Wash the air tubing in warm water using mild detergent.
3. Rinse thoroughly, hang and allow to dry out of direct sunlight.
4. Before next use, reconnect the air tubing to the air outlet and mask.

Technical specifications

	ClimateLine	ClimateLine ^{MAX} / ClimateLine ^{MAX} Oxy
Maximum recommended pressure	25 cm H ₂ O	30 cm H ₂ O
Inner diameter	0.6" (15 mm)	0.75" (19 mm)
Length	6'6" (2.0 m)	6'3" (1.9 m)
ClimateLine temperature range	60°F–86°F (16°C–30°C)	60°F–86°F (16°C–30°C)
ClimateLine temperature cut-out	≤ 106°F (≤ 41°C)	≤ 106°F (≤ 41°C)
Material	Flexible plastic and electrical components.	
Electromagnetic compatibility	Refer to the S9 Information Guide.	

Symbols which appear on the air tubing

Limited warranty

Please refer to your S9 Information Guide.

WARNINGS

- Read the entire instructions before using this device.
- In the US, Federal law restricts this device to sale by or on the order of a physician.
- If damage to the air tubing results in air leaking, stop using the air tubing.
- The ClimateLine is designed for use only with the H5i.

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© 2017 ResMed Ltd. 368880/2 2017-08
ClimateLine, H5i and S9 are trademarks of ResMed Ltd and ClimateLine and S9 are registered in U.S. Patent and Trademark Office.
Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia.