

Mirage Micro™

NASAL MASK

Fitting Guide




- Turn the MicroFit dial to position 24, where the forehead support is fully extended.



- Disconnect one headgear clip, place the mask on the nose and pull the headgear over the head.



- Bring the lower straps below the ears.
- Connect the lower headgear clip ensuring it clicks into place.

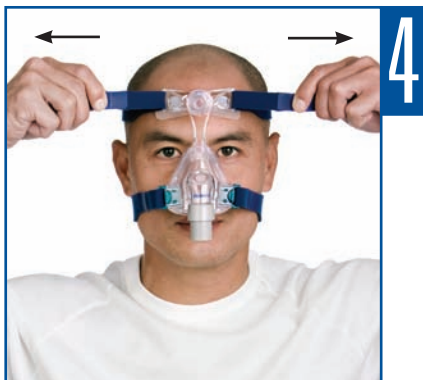

- Unfasten the upper Velcro™ tabs, pull the headgear straps evenly until comfortable, then reattach the Velcro.
- Do not overtighten the straps.



- Repeat Step 4 with the lower headgear straps.
- Connect one end of the air tubing to the swivel and the other end to the flow generator.
- Turn on the flow generator.



- While lying down, turn the MicroFit dial until the mask is evenly and comfortably positioned on the face.
- If any air leaks are felt, readjust the dial and/or headgear.

⚠ For details, refer to the *Mirage Micro Nasal Mask User Guide*.

ResMed Corp Poway, CA, USA +1 858 746 2400 or 1 800 424 0737 (toll free), ResMed Ltd Bella Vista, NSW, Australia +61 (2) 8884 1000 or 1 800 658 189 (toll free). Offices in Austria, Brazil, China, Finland, France, Germany, Hong Kong, India, Japan, Malaysia, Netherlands, New Zealand, Norway, Singapore, Spain, Sweden, Switzerland, United Kingdom (see website for details). Protected by patents: AU 710733, AU 741003, AU 766623, AU 775051, AU 777033, AU 785376, CA 2261790, EP 0956069, EP 1187647, EP 1187648, EP 1187649, EP 1187650, EP 1314445, EP 1479406, HK 1057714, JP 3686609, NZ 513052, NZ 526165, NZ 526166, NZ 526167, NZ 526168, US 6112746, US 6119693, US 6357441, US 6374826, US 6439230, US 6463931, US 6532961, US 6557556, US 6581594, US 6581602, US 6634358, US 6691708, US 6823865, US 6860269, US 6871649, US 6997188, US 7011090, US 7159587, US 7178527, US 7188620, US 7216647, US 7234466, US 7243651. Other patents pending. Protected by design registrations: AU 304309, CN 200530117106.1, EP 366844, JP 1264474, NZ 406357, US D532512, US D547439, US D545960. Others pending. Mirage and Mirage Micro are trademarks of ResMed Ltd and Mirage is registered in U.S. Patent and Trademark Office.
© 2007 ResMed Ltd. 1012275/1 07 12