

Less mask. **More you.**



Bringing light, unobtrusive design to a full face mask, Quattro™ FX provides an effective alternative to conventional full face masks and more choice in full face therapy.

Benefits for you ...

It's as easy as ... Size. Swing. Seal

Quattro FX gives you the tools to achieve the best fit for your patients—first time, every time. The custom-made sizing tool and **Swing Seal** fitting technique simplify the sizing, fitting and sealing process ... saving you time, money and additional patient visits.

Time and cost savings

The suspension-like combination of the Spring frame and Spring Air™ cushion means forehead support is unnecessary. Eliminating an additional point of adjustment, fitting Quattro FX is fast and straightforward.

Therapy acceptance

The comfort, performance and feeling of openness that Quattro FX provides, help users accept and continue therapy ... at titration and beyond.

... and your patients

Quality and performance retained

New technologies in the Spring Air cushion and Spring frame offer the stability and angular adjustment provided by conventional forehead support. Quattro FX: quality and performance in a less obtrusive full face mask.

Freedom to be

Fewer points of contact with the face reduce the chances of skin irritation; a clear line of sight helps break down the physical and emotional barriers associated with full face therapy.

A good night's sleep

High-tech Spring Air cushion distributes pressure evenly and absorbs even the slightest user movements, so users can enjoy a good night's sleep, confident that their seal is safe.



Quattro™ FX
FULL FACE MASK



Technology highlight

Size. Swing. Seal **NEW**

Sizing, fitting and sealing Quattro FX is fast and straightforward.

The custom-made **sizing tool** provides an easy and accurate sizing process.

Once sized correctly, fitting the mask is easy too, with the **Swing Seal** technique:

- 1 Firstly position the cushion at an angle on the nasal bridge ...
- 2 ... then **swing** the lower half of the mask down against the face to **seal**.

Quattro FX: fits right, seals securely, first time, every time.

Clear line of sight

and feeling of openness created by a full face mask that starts mid-way on the nasal bridge; a smaller frame also makes side-sleeping more comfortable

Built-in vent design

disperses air gently and quietly

Flexible, durable Spring frame

strong enough to stabilise the mask on the face, soft enough to accommodate user movement during sleep

360° elbow rotation


positions the tubing for ease of use

Swivel

offers a convenient way to connect and disconnect air tubing

New, tension-reducing headgear design

features a contoured crown strap that sits snug at the top of the head and avoids the sensitive area around the back of the neck

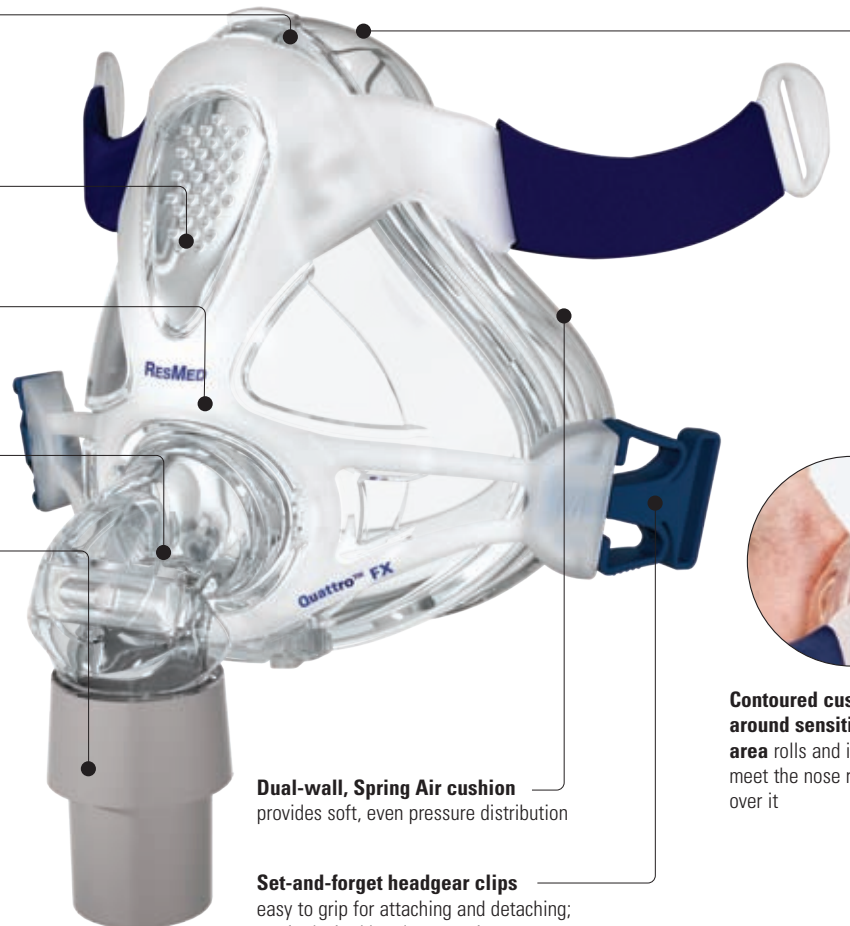

Dual-wall, Spring Air cushion
provides soft, even pressure distribution

Set-and-forget headgear clips
easy to grip for attaching and detaching;
retain desired headgear settings

Contoured cushion membrane around sensitive nasal bridge area rolls and inflates to gently meet the nose rather than stretch over it

Product Codes

	Asia Pacific
Small	61763
Medium	61764
Large	61765

Quattro™ FX
FULL FACE MASK

www.quattrofx.com

Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia. Distributed by: ResMed Corp 9001 Spectrum Center Boulevard San Diego CA 92123 USA. ResMed (UK) Ltd 96 Milton Park Abingdon Oxfordshire OX14 4RY UK. See www.resmed.com for other ResMed locations world-wide. Quattro and Spring Air Technology are trademarks of ResMed Ltd and Spring Air Technology is registered in U.S. Patent and Trademark Office. © 2010 ResMed Ltd. Specifications may change without notice. 1014330/1 2010-10