

RESMED

S9 Escape Auto™

POSITIVE AIRWAY PRESSURE DEVICE



Welcome Guide

English

S9™ Series
More. Comfort.

S9[™] | Series
More. Comfort.

Welcome

Thank you for choosing the S9 Escape Auto. Before operating this device, please read the entire Welcome and Information Guides.



S9 at a glance

The S9 system comprises the following elements:

- S9 device
- SlimLine air tubing
- 90W or 30W power supply unit
- S9 travel bag
- SD card
- S9 SD card protective folder.


Optional components include:

- H5i heated humidifier
- H5i water tub
- Standard air tubing
- ClimateLine heated air tubing
- ClimateLine^{MAX} heated air tubing
- 90W power supply unit
- 30W power supply unit (does not support H5i)
- Power Station II battery pack
- DC/DC Converter 24V/90W.

Traveling tips

When traveling with your S9 only:

- Ensure you pack the SlimLine or Standard air tubing as the ClimateLine or ClimateLine^{MAX} heated air tubing is not designed to connect directly to the S9 device.
- Please ensure you purchase and travel with the approved power cord for the region where you will be using the S9 device.

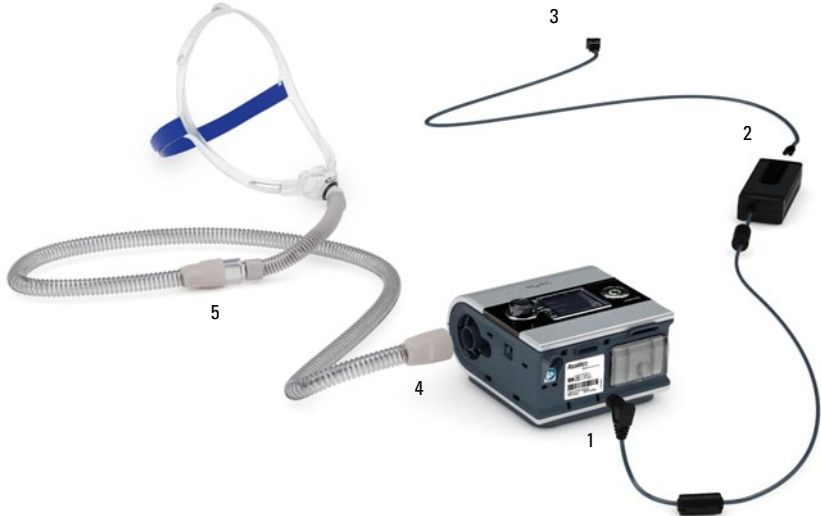

Setup

1. Connect the DC plug of the power supply unit to the rear of the S9.
2. Connect the power cord to the power supply unit.
3. Plug the other end of the power cord into the power outlet.
4. Connect one end of the air tubing firmly onto the air outlet.
5. Connect the assembled mask system to the free end of the air tubing.

Notes:

- For more information on assembling your mask see your mask user guide.
- Recommended masks are available on www.resmed.com on the Products page under Service & Support.

S9 basics



Start/Stop button
Starts or stops
treatment.

Home menu

LCD screen

Info menu*
Allows you to
view your sleep
statistics.

Setup menu*
Allows you to
make changes
to settings.

Push dial
Turning the dial allows
you to scroll through
the menu and change
settings. Pushing the
dial allows you to
enter into a menu and
confirm your choice.

Key:

Home

Humidity
level

Ramp

Heated tube

Climate
Control

Start/Stop

Push dial

Setup menu

Info menu


*The Info and Setup menus are disabled if
S9 Essentials has been enabled by your clinician.

S9 Essentials





S9 Essentials is designed to make device interaction and menu navigation easier for you. If enabled by the clinician, S9 Essentials disables the Info and Setup functionality so that you can simply start and stop therapy and adjust ramp, humidification and Climate Control.


Navigating the menus

In general, to navigate the menus:





The image shows four sequential screenshots of the RAMP menu on a device. Each screenshot has a dark blue background with a white house icon in the top left corner and the word 'RAMP' in white at the top. A small green triangle icon is in the top left of the main display area. The main display area shows a green triangle with a white line indicating a ramp level, and a numerical value followed by 'min.' to its right. The background of the main display area is either blue or orange.

1. Turn  until the parameter you require is displayed in blue.
2. Press . The selection is highlighted in orange.
3. Turn  until you see the setting that you require.
4. Press  to confirm your choice. The screen returns to blue.



Getting started

1. Make sure the power is connected.
2. Adjust the ramp time if required.
3. Fit your mask as described in your mask user guide.
4. To start therapy press .
5. Lie down and arrange the air tubing so that it is free to move if you turn in your sleep.
6. To stop treatment at any time press .

Notes:

- Once therapy has started a treatment screen is displayed.
- If power is interrupted during treatment, the device automatically restarts therapy when power is restored.

Cleaning and maintenance

Refer to your mask and humidifier user guides for detailed instructions regarding the care of this device.

Daily

Remove the air tubing by pulling on the finger grips on the cuff. Hang it in a clean, dry place until next use.

Notes:

- Do not hang the air tubing in direct sunlight as it may harden over time and eventually crack.
- Do not wash the air tubing in a washing machine or dishwasher.

Weekly

1. Remove the air tubing from the S9 device and the mask.
2. Wash the air tubing in warm water using mild detergent.
3. Rinse thoroughly, hang, and allow to dry.
4. Before next use, reconnect the air tubing to the air outlet and mask.

Monthly

1. Wipe the exterior of the S9 with a damp cloth and mild detergent.
2. Check the air filter for holes and blockage by dirt or dust.

Replacing the air filter


Replace the air filter every six months (or more often if necessary).

1. Remove the air filter cover from the back of the S9 device.
2. Remove and discard the old air filter.
3. Insert a new ResMed air filter ensuring that it is sitting flat in the air filter cover.
4. Replace the air filter cover.

Notes:

- Ensure the air filter and air filter cover are fitted at all times.
- Do not wash the air filter. The air filter is not washable or reusable.

Air filter cover

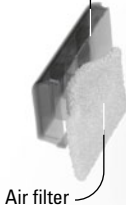

Air filter



SD card

An SD card has been supplied to gather therapy data from your S9 device and provide settings updates from your clinician. When instructed to do so, disconnect the S9 device from the power outlet, remove your SD card, insert it in the protective folder and send it to your clinician.

Removing the card

1. Push in the SD card to release it.
2. Remove the card.
3. Insert the card into the protective folder.
4. Send the protective folder back to your clinician as instructed.

For more information on removing and inserting your card refer to the S9 SD Card Protective Folder provided with your device. Please retain the S9 SD Card Protective Folder for future use.



Notes:

- For Troubleshooting information refer to the Information Guide.
- For more product information see www.resmed.com.

S9[™] | Series
More. Comfort.

368872/2 2017-09
S9 Escape Auto

WELCOME
APAC MULTI



Manufacturer: ResMed Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia.

See www.resmed.com for other ResMed locations worldwide.

For patent information, see www.resmed.com/ip.

S9, H5i, SlimLine and ClimateLine are trademarks of ResMed Ltd and S9 and ClimateLine are registered in U.S. Patent and Trademark Office.
© 2017 ResMed Ltd.