






## How to Cut & Sew OLSON MASKS:

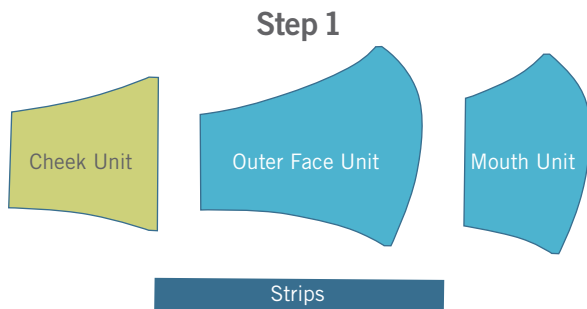

Cut 2 of each shape, giving 6 mask pieces and 2 strips.

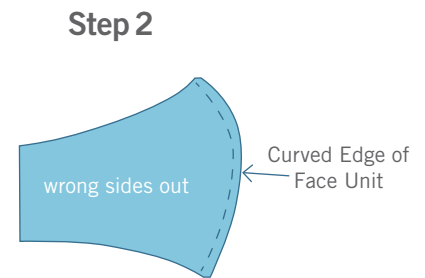


Placing right sides of face unit together, matching curved edge to curved edge. Sew only around curved edge. Set aside.

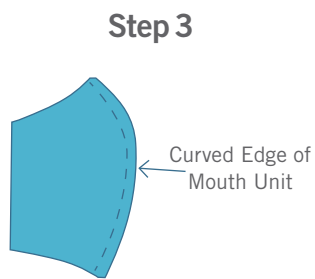


Lay the 2 mouth units on top of each other, right sides together, matching curved edge to curved edge, and straight edge to straight edge.

Sew only around the curved edge.

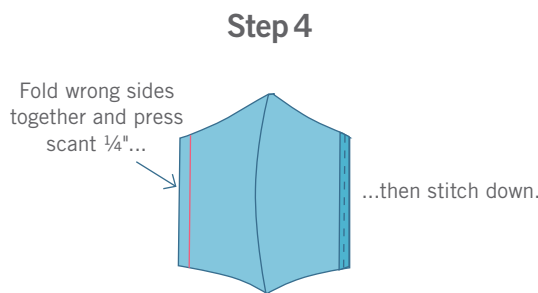

Lay your mouthpiece open and fold the straight edges over about 1/4" and sew each side. Then set aside.

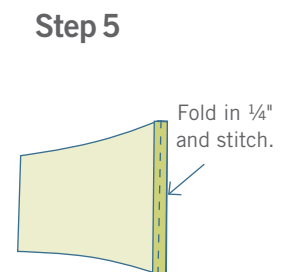


On your cheek pieces, take your larger straight edge, fold over 1/4" and sew. Then set aside.

Continued on back

## How to Sew OLSON MASKS: continued




**Step 6**

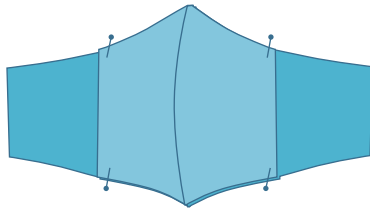


Place the Outer face unit in front of you, right side up. Lay the mouth unit on top, right sides down, matching the center seams. Pin both layers together at the top and bottom.

**Step 7**

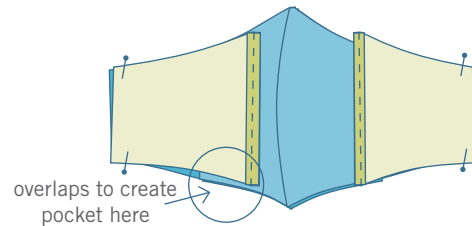

Now, lay the cheek unit on either side, also right side down. You will want to line up the outer corners and pin around the edges. Note that the finished edge of the cheek unit will be overlapping the finished edges of the mouth unit section, creating the pocket. Sew  $\frac{1}{4}$ " seam around the outside edge of the mask.


**Step 8**

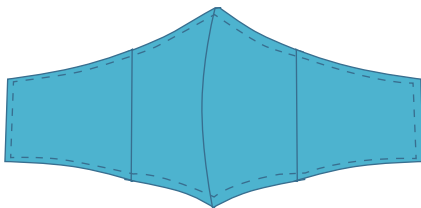

Turn the mask right side out through one of the mouth unit/cheek unit openings. Use only 1 opening.

Sew around the entire perimeter of the mask.

**Step 9**

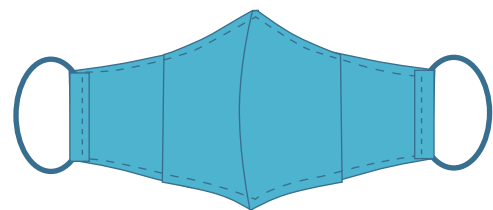

Fold each strip piece in half lengthwise with wrong sides together. Open each strip and fold the raw edges in to the fold, pressing as you go. Sew down each strip approx.  $\frac{1}{8}$ " from the double folded edge. Fold short edges over the ties/elastic and stitch down that short edge as shown.

\*Please note that the CDC does not consider homemade masks personal protective equipment (PPE) and will only be used by professionals as mask covers or when no other alternative is available.