



# cologuard<sup>®</sup>

## HOW TO RETURN YOUR KIT

**PLEASE RETURN YOUR KIT TO EXACT SCIENCES LABORATORIES VIA UPS ACCORDING TO THE FOLLOWING INSTRUCTIONS.**

If you need to contact Exact Sciences Laboratories, please call us at 1-844-870-8870. Our specialists are available 24 hours a day, 7 days a week to answer any questions.



[www.cologuardtest.com](http://www.cologuardtest.com)  
Rx Only

LBL-10034rev3

*Cologuard<sup>®</sup> colorectal cancer screening test is a registered trademark of Exact Sciences Corporation.*

# PREPARE YOUR KIT FOR SHIPMENT



a.

a. Locate the shipping label on top of your box.


- Shipping labels on the box may differ.



b.

b. If the shipping label has a green tear strip, it must be removed.

- Lift the left corner of the tear strip and pull it off.
- Lift the bottom edge of the label and pull up.
- Tear the top label off the box.
- A new label now shows.
- You are ready to ship your box.
- Continue with the next step.

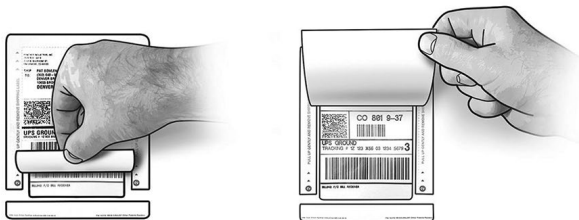

c. If the shipping label has **no** green tear strip, you are ready to ship your box.

# SHIP YOUR SAMPLES USING UPS

**Samples must be shipped to arrive at the lab within 3 days.**

- The lab **must** receive samples within 3 days for samples to be tested. The lab will not test samples not received in time.
- UPS will ship your samples to arrive at the lab the next day from the day it shipped.
- Ship your samples **within a day** of collection for quickest delivery to the lab.

## **SHIP YOUR SAMPLES WITHIN A DAY OF COLLECTION.**

We recommend the following shipping schedule.

**If you collected  
your samples on:**

**You must ship  
your samples by:**

Sunday .....	Tuesday
Monday .....	Wednesday
Tuesday.....	Thursday
Wednesday.....	Friday
Thursday.....	Friday

**Make sure a holiday will not delay your shipment**

If you have questions about when to ship your samples, call our specialists at 1-844-870-8870.

## WAYS TO SHIP THE SAMPLES

You have the following options for shipping your samples:

- Drop off at a UPS location. Check [www.ups.com](http://www.ups.com) for UPS drop-off locations.
  - UPS Stores
  - UPS Customer Center
  - UPS Authorized Shipping Outlet
  - Any UPS Driver
- Call UPS at 1-800-742-5877 for pick up or to find a drop off location near you (M-F 7:30am to midnight, Sa 7:30am-9:00pm Eastern Standard Time).
  - You can also schedule a pick up online at [www.ups.com](http://www.ups.com) any time.
  - Be sure to provide the tracking number shown on your prepaid return shipping label, which starts with 1Z.

Once you have shipped the samples to the lab, you are finished!  
Your doctor will contact you with your result.

## REMINDER: STORING YOUR SAMPLES BEFORE SHIPPING

To store the samples before you return the box:

- Store your boxed and sealed samples at room temperature.
- Keep the samples away from heat and direct sunlight.
- Keep samples out of the reach of children.
- Keep the box upright.