

PKA™ PRAFO® Orthosis

Positioning and Walking Ankle Foot Brace for Custom KAFO or AFO Adaptation



1. Prior to the PKA™ PRAFO® attachment the positive mold was modified in a customary manner.



2. Place the PKA™ PRAFO® (APU® shown) on the calf section of the mold to establish the mounting location.




3. Glue or nail the spacer bar in place to form the PKA™ PRAFO® channel.
1½" x 4½" x 5/8"
Molding Dummy



4. Use plaster to smooth the transition from the calf to the spacer bar prior to vacuum forming.



5. Pull stocking over mold, vacuum form and finish plastic to standard.

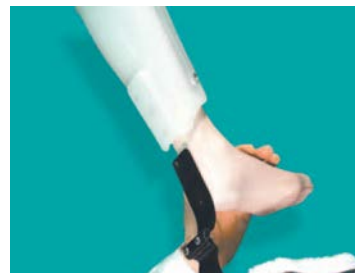

6. Slide PKA™ PRAFO® into channel and attach with retainers and screws.



7. Alternatively, the heel connecting bar can be mounted on the outside to increase relief of the Achilles tendon and heel areas if needed.



282SKG PKA™ Adult PRAFO®/APU® shown. Pediatric (153SKG) and Infant (050SKG) versions available. Also available with any bar or black liner system.

PRAFO® Orthosis is a registered trademark of Anatomical Concepts Inc. #1,793,550
APU® Articulated AFO is protected under US Patent No's 5,908,398 and 5,944,679 and is a registered trademark of Anatomical, Concepts, Inc.



www.AnatomicalConceptsInc.com

Toll Free: 1-800-837-3888

Phone: (330) 757-3569 • Fax: 1-800-657-7236

1399 E. Western Reserve Road • Poland, Ohio 44514