

Disinfection and Cleaning Instructions



WARNING

- Confirm that the respiratory density of disinfectant including solvent is under legal regulation. Certain disinfectants may damage health. When using a disinfectant, follow instructions supplied by the manufacturers.
- Do not use the following disinfectants or sterilizers at the time of disinfection. Quality, performance and safety of each device cannot be assured.
 - Chloric disinfectant which is strongly corrosive to metals and rubber parts.
 - Disinfectant whose uses on metals, plastics, rubber parts and coating are forbidden according to the instructions supplied with the disinfectant.
 - Formalin gas and disinfectant sprays that may get inside the equipment.Disinfectant ethanol is recommended for disinfection. Carefully read the instructions and cautions supplied with the disinfectant before use. For details on the disinfectant, contact a FUJIFILM dealer or the service representatives at the agency from which you purchased the disinfectant.



CAUTIONS

If the sensor unit of the flat panel sensor is not disinfected, it may lead to secondary infection. Clean the sensor unit of the flat panel sensor with ethanol for disinfection, etc. for each patient to prevent infection.

Wireless Communication



CAUTIONS

- Do not place devices generating electromagnetic wave (CT, MRI, diathermy, RFID etc.) near the DR-XD 1000 (FDR AQRO).
- We recommend not to use any other wireless devices such as cellular/smart phones, portable phones, microwave ovens, WAPs, etc. within 2m (6.6 ft) of the DR-XD 1000. When other wireless devices are used within 2m (6.6 ft), wireless data communication may be delayed. (Data will not be lost; if a timeout occurs a retry can be performed after the cause of interference has been removed)
- Do not cover the Flat Panel Sensor (DR-ID 1201SE/DR-ID 1202SE/DR-ID 1211SE/DR-ID 1212SE/DR-ID 1213SE/DR-ID1271SE/DR-ID1272SE/DR-ID1273SE/DR-ID1274SE/DR-ID1275SE) with a shield such as a metallic plate as this will interfere with a wireless communication.

- ① For details, see “Flat Panel Sensors DR-ID 1201SE/DR-ID 1202SE/DR-ID 1211SE/DR-ID 1212SE/DR-ID 1213SE Operation Manual”, “Flat Panel Sensors DR-ID1271SE/DR-ID1272SE/DR-ID1273SE/DR-ID1274SE/DR-ID1275SE Operation Manual” and “Access point Operation Manual”.

Flat Panel Sensor

- ① For details, see “Flat Panel Sensors DR-ID 1201SE/DR-ID 1202SE/DR-ID 1211SE/DR-ID 1212SE/DR-ID 1213SE Operation Manual” and “Flat Panel Sensors DR-ID1271SE/DR-ID1272SE/DR-ID1273SE/DR-ID1274SE/DR-ID1275SE Operation Manual”.

Image Processing Unit

- ① For details, see “DR-ID 300CL Operation Manual”.