

## Luxottica Eyewear Sanitation Guidelines

In order to make your staff and your patients feel at ease while showing and touching your products we are providing you with basic guidelines on how to sanitize frames while protecting your health, with no damage to the frames.

1. We recommend sanitizing the frames after each use/customer visit
2. We recommend using latex gloves during your patient visit or at any time when touching the product
3. Use latex gloves before starting the sanitizing procedure
4. The frames can be cleaned with **hydrogen peroxide wipes**. If hydrogen peroxide wipes are not available, a 50/50 3% hydrogen peroxide / water solution mixed in a spray bottle can be used, with a regular eyewear cleaning cloth
  - a. **Please note**, hydrogen wipes contain a different substance than regular disinfectant wipes (ex: Lysol/Clorox brand). Please **DO NOT** use other disinfectant wipes, as they can damage the product (no acetone)
5. After cleaning the frames, please discard the wipes (do not re-use them). The cleaning cloth can be machine washed and used again

These eyewear sanitation guidelines were provided by **Luxottica and are intended solely for their product**. Please contact the manufacturer of your frames to determine if this recommendation is suited to their materials.

Links to Help:

1. [Optix 55 Pre-Moistened Lens Wipes](#)
2. [CareTouch Lens & Screen Cleaning Wipes](#)
3. [AAwipes Eyeglass Cleaning Cloth](#)
4. [Framar Midnight Mitts Black Nitrole Gloves](#)