

TOKUYAMA **UNIVERSAL BOND**

EVERYTHING & THE...



ACTIVE CHEMISTRY SELF-CURE

provides worry free polymerization when curing lights cannot reach



TRUE UNIVERSAL

all substrates without the need for additional activators and primers



QUICK & EASY

25 second application, no need to agitate, no wait time

Tokuyama UNIVERSAL BOND is a two component self-cured dental adhesive system for direct and indirect restorations. Tokuyama Universal Bond satisfies **ALL** three application definitions listed by The Dental Advisor. May-June 2017 Vol. 34, No.3

1. Compatible with different etching techniques: total-, self-, or selective-etch mode
2. Compatible with dual- and self-cured materials without the use of a separate activator
3. Can be used as a primer for silica-based and metallic restorations

FEATURES OF Tokuyama UNIVERSAL BOND



TRUE UNIVERSAL

- Compatibility with self-etch, total-etch, and selective-etch techniques
- Compatibility with light-curing, dual curing, and self-curing
- Applicability to direct and indirect restoration
- Use as a primer for silica-based, zirconia based and metallic restorations



QUICK & EASY APPLICATION

- No need to apply separately for tooth and restoratives
- No need to wait after bond application
- No need to light cure



RELIABLE


- High bond strength to various dental surfaces & substrates
- Cavity adaption without voids
- Performs with wide mixing ratios & margin for error
- Worry - free polymerization when curing lights cannot reach




INDICATIONS




- Direct anterior and posterior restorations with light-curing, dual-curing, and self-curing composite materials

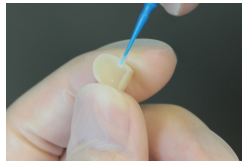

- Intraoral repair of composite restorations, metal, porcelain fused to metal, and all ceramic restorations without an additional primer

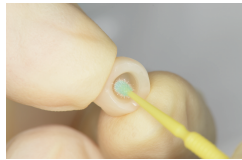

- Cementation of indirect restorations & veneers when combined with light-cure, dual-cure, and self-curing resin cements

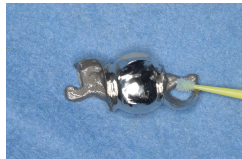

- Bonding and repair of denture resin to metal bases, clasps or attachments
- Bonding of opaque resin to a metal base in the fabrication of resin-faced stainless steel crowns



- Bonding of core build-ups made of core build-up materials

UNIVERSAL USE

Applications	TOKUYAMA UNIVERSAL BOND	Scotchbond Universal Adhesive	G-Premio Bond	Futurabond U	All-Bond Universal	Clearfil Universal Bond	Prime & Bond Elect	Xeno Select	Adhese Universal	iBond Universal
Total-Etch, Self-Etch, Selective Etch	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Compatible w/ all light-curing, dual-curing, or self curing composites	✓	△ *1	✗ *2	✓	✓	△ *6	△ *3	✗	✓	✓
Direct Restorations	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Indirect Restorations	✓	△ *1	△ *3	✓	✓	△ *6	△ *3	✗	✓	✓
Intraoral Repair	✓	✓	△ *4	✓	△ *4	✓ *7	△ *4	✗	△ *8	△ *4
Primer for Prosthesis	✓	△ *1	✗	✗	✓ *5	△ *6	✗	✗	✗	✓ *5

*1 Requires Dual Cure Activator (DCA) unless it is used with Rely X Ultimate

*2 Bonding of dual-cured core build up composites to tooth structure as long as these materials are light-cured

*3 Requires DCA

*4 Requires Primer

*5 Requires light-curing

*6 Requires DCA and light-curing unless it is used with CLEARFIL DC CORE PLUS or PANAVIA SA CEMENT

*7 Primer recommended


*8 Only composite repair

SIMPLE HANDLING


SHORT CHAIR TIME DIRECT & INDIRECT RESTORATIONS

- Keep the bottles as vertical as possible while dispensing the drops into the mixing well.
- Application time is 3 minutes after mixing.
- In the case of intraoral repair of restorations. If the adherent surface includes tooth structure and restorations (ceramics, porcelain, metals or composite materials), apply Tokuyama UNIVERSAL BOND to the entire adherent surface at once. No extra primers are needed.

1) Direct Restorations and Intraoral Repair of Restorations with Composite Resin

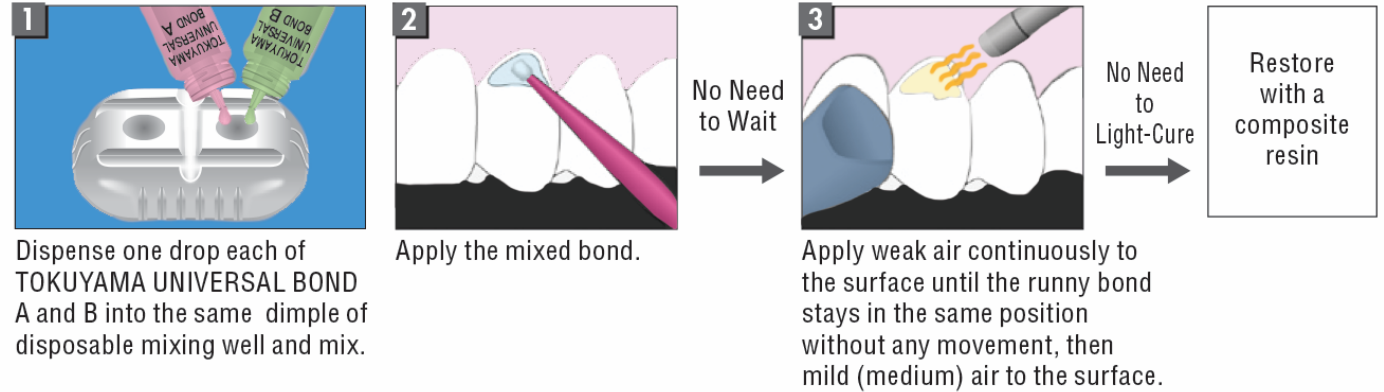

2) Cementation of Indirect Restorations

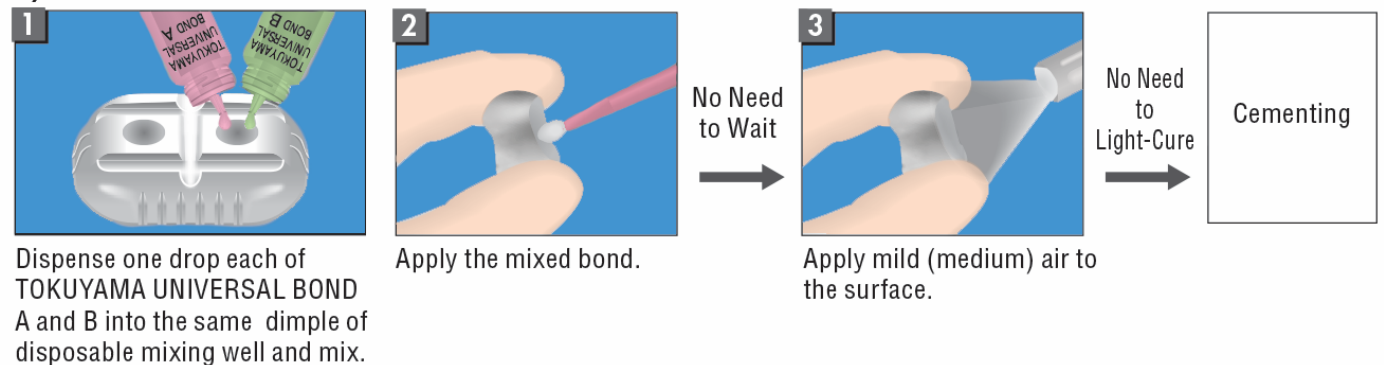


SIMPLE HANDLING EASY TECHNIQUE COMPARISON


DIRECT RESTORATIONS

**TOKUYAMA
UNIVERSAL BOND**


🕒 25 SEC

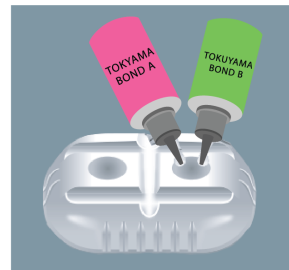


Dispense one drop of each into the same dimple

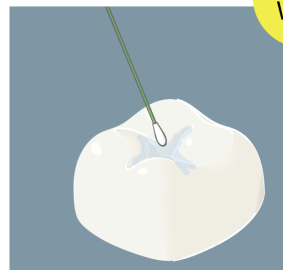

Apply mixed bond.

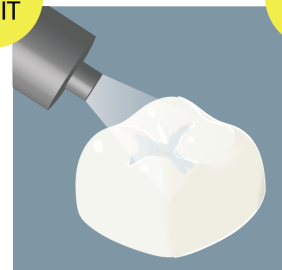

Apply weak air.
Then mild air.

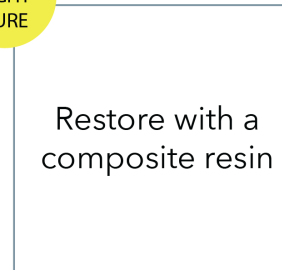


Restore with a
composite resin

COMPETITORS


🕒 ≥35 SEC




Dispense one drop of bonding agent

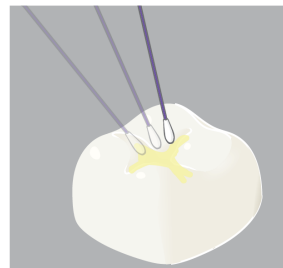


Apply and Agitate

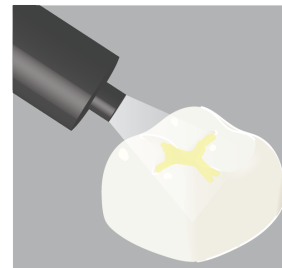

Air dry to
evaporate solvent

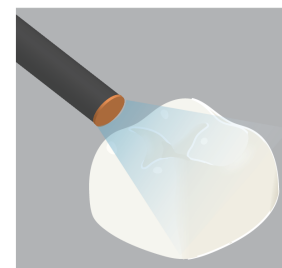

Light cure
bonding agent

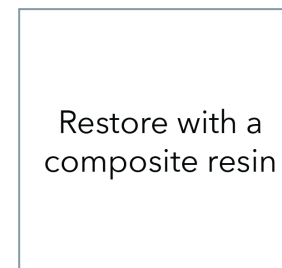


Restore with a
composite resin

SIMPLE HANDLING EASY TECHNIQUE COMPARISON


INDIRECT RESTORATION

**TOKUYAMA
UNIVERSAL BOND**


🕒 25 SEC

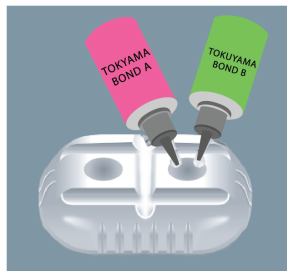

Dispense one drop of each into the same dimple

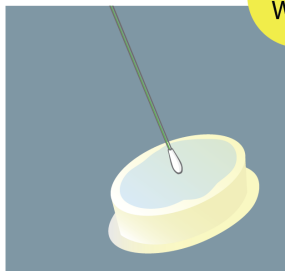

Apply mixed bond.

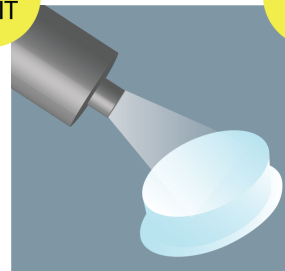

Apply mild air.

NO
WAIT

NO
LIGHT
CURE


Cementing or
Intraoral repair

COMPETITORS

🕒 ≥40 SEC




Dispense and mix
additional activator

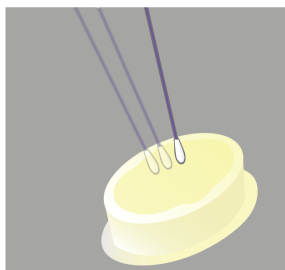

Apply and Agitate



Air dry to
evaporate solvent

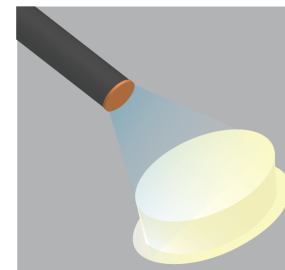


Light cure
bonding agent

Cementing or
Intraoral repair


SIMPLE HANDLING MIXING RATIOS

Maintains performance under a wide range of mixing ratios, offering a wide margin for error.

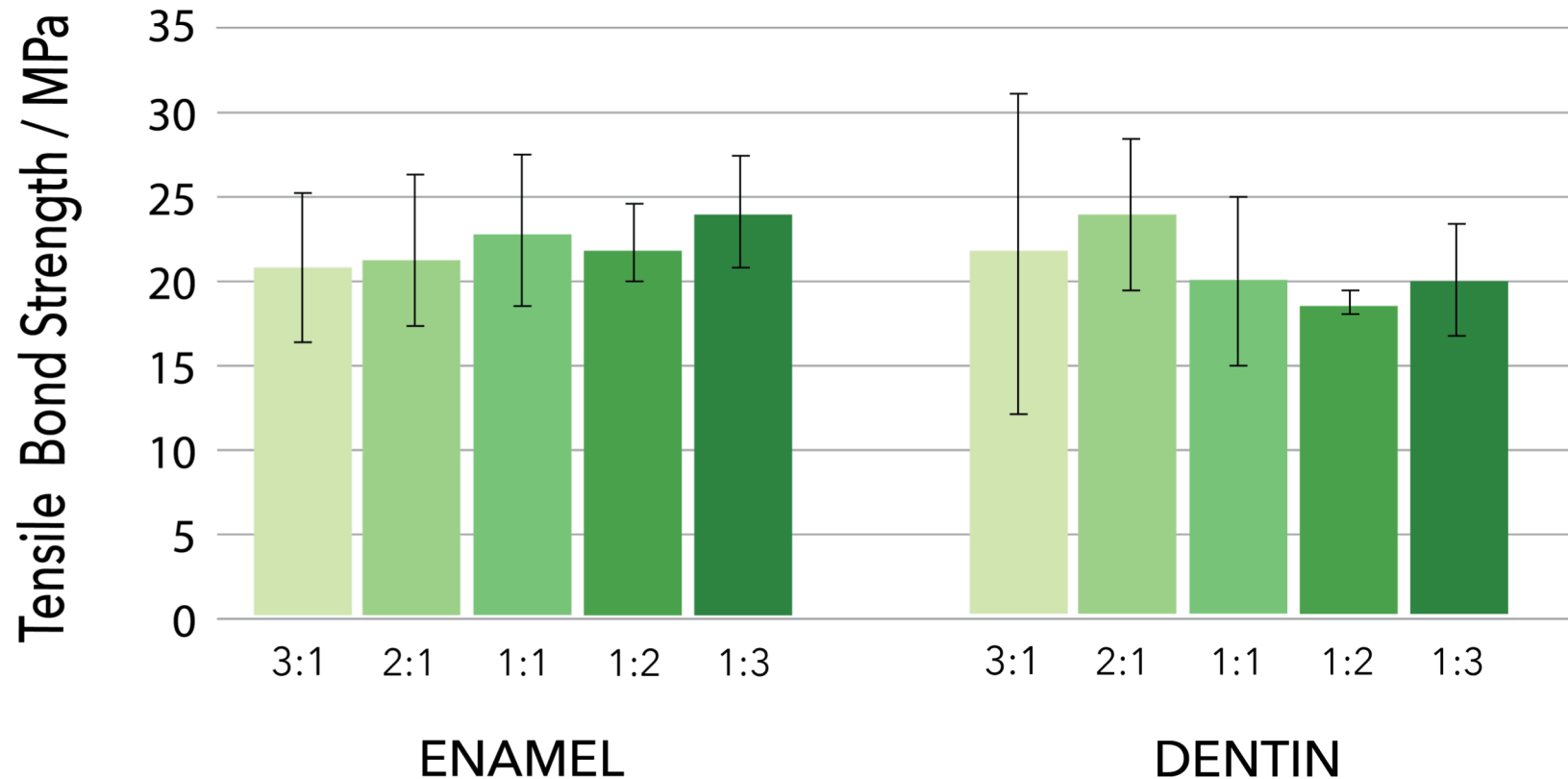

SHORT CHAIR TIME


INDIRECT RESTORATION - INTRORAL REPAIR CERAMICS

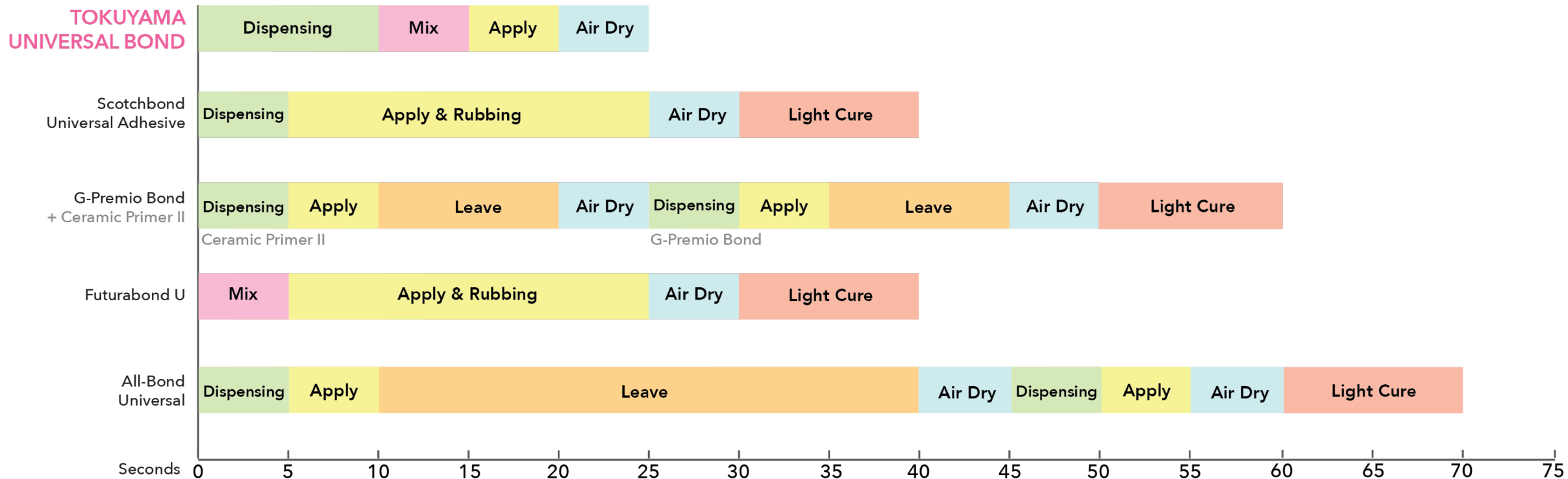


SHORT CHAIR TIME

DIRECT RESTORATION - COMPOSITE RESIN

**TOKUYAMA
UNIVERSAL BOND**

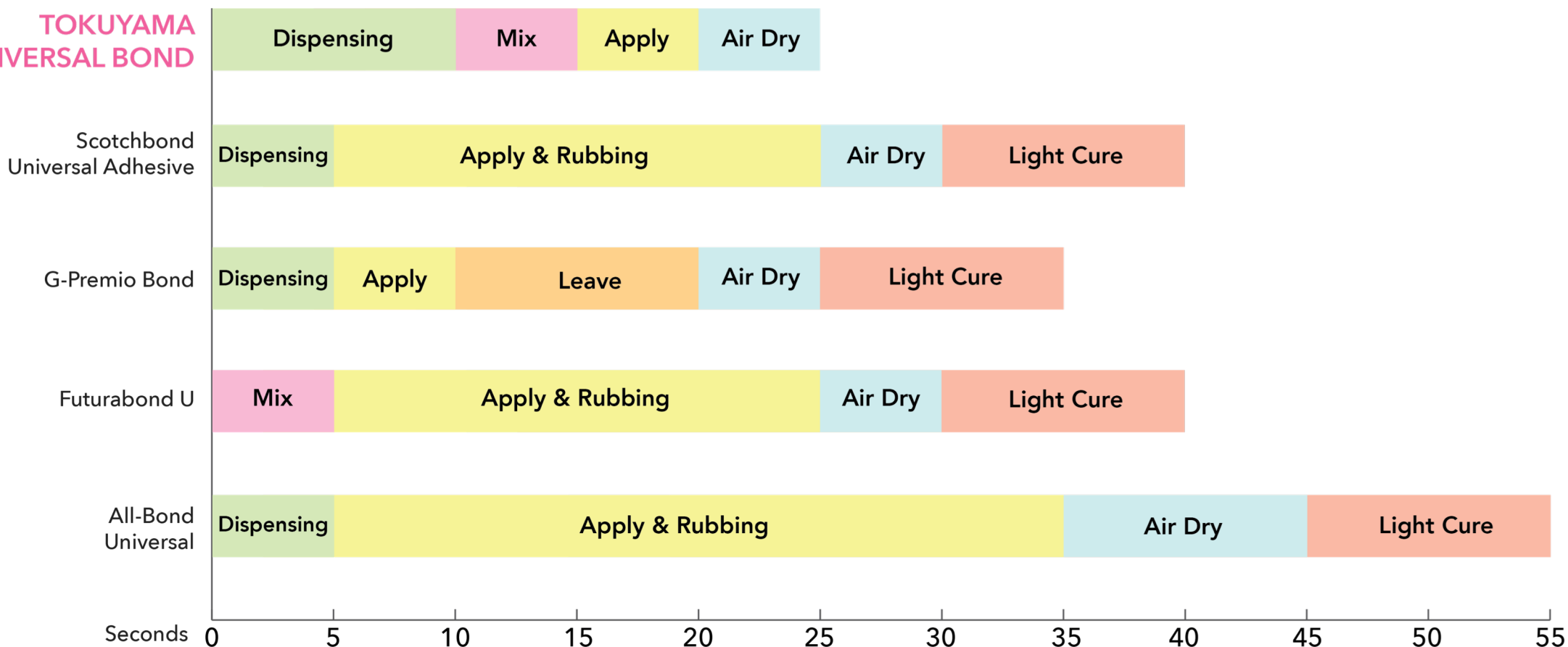


RELIABILITY CAVITY ADAPTION IN DIRECT RESTORATIONS

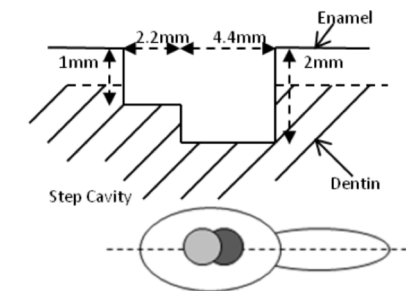

**Tokuyama
UNIVERSAL BOND**

CORNER

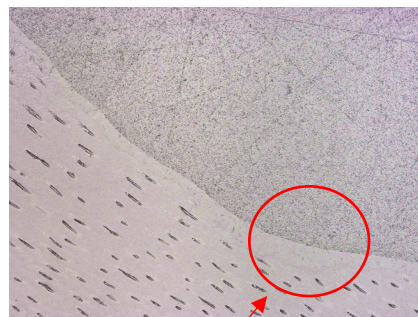


NO GAP


SIDE



EDGE

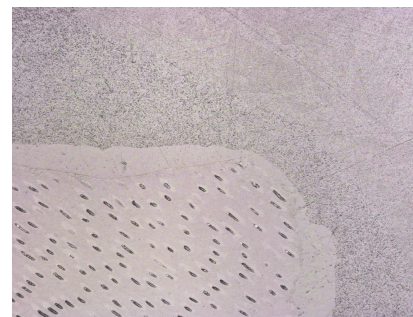



FLOOR

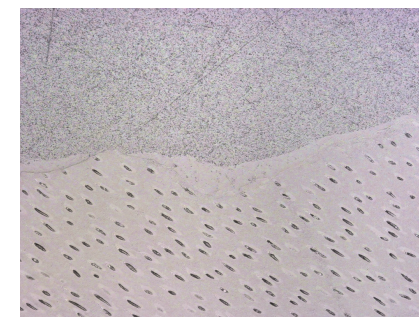

**Scotchbond
Universal Adhesive**




GAP

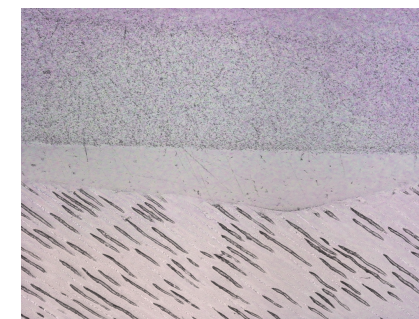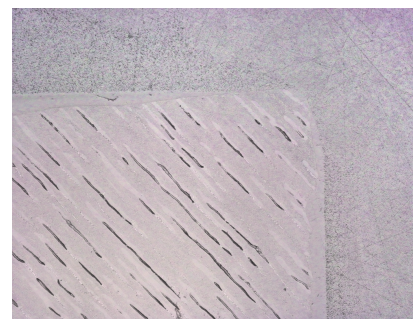

COMPATIBILITY ETCHING PROTOCOL

Shear Bond Strength to Enamel

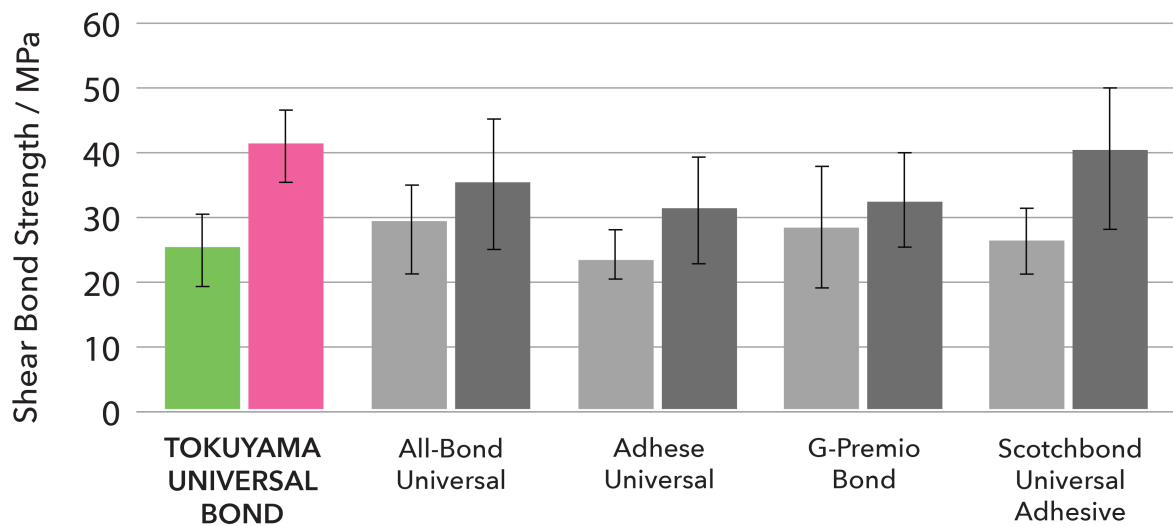


Self-Etch after 24hrs



Total-Etch after 24hrs


Shear Bond Strength to Dentin

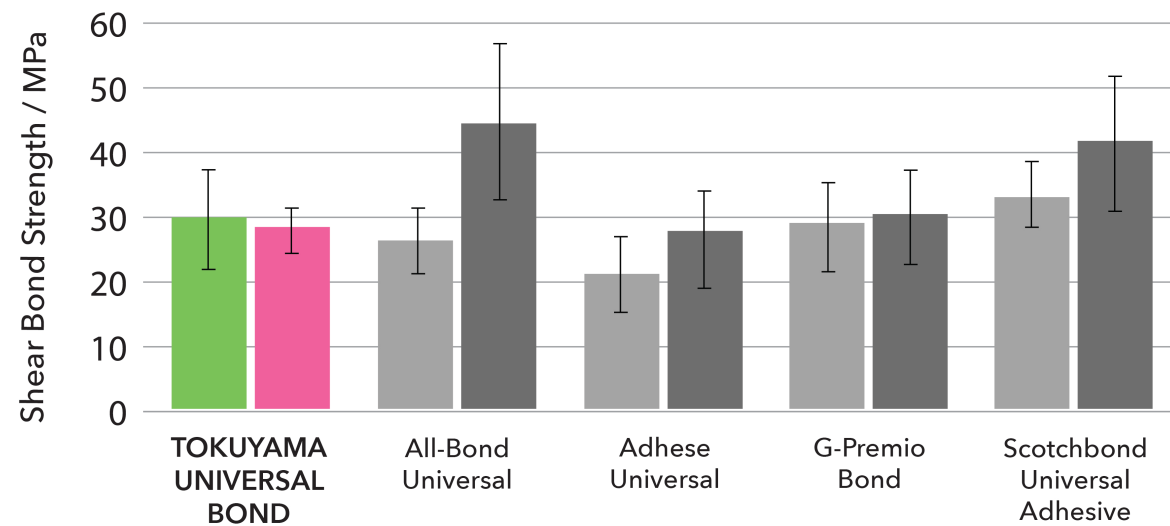


COMPATIBILITY

TENSILE BOND STRENGTH TOOTH

Total-Etch / Enamel

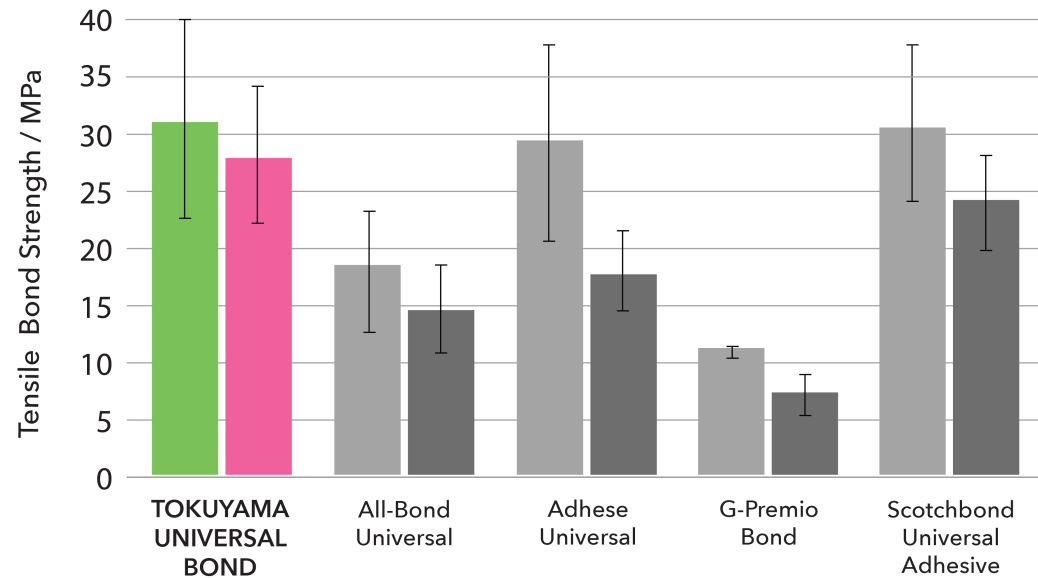

Total-Etch / Dentin

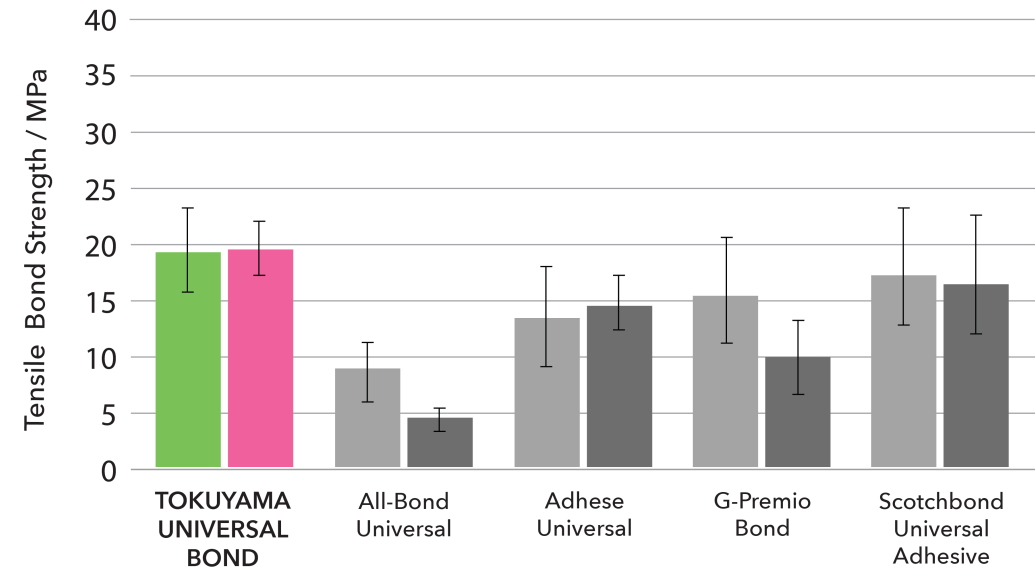


After 24hrs




After 3,000 thermo-cycling

COMPATIBILITY TENSILE BOND STRENGTH TOOTH

Self-Etch / Enamel

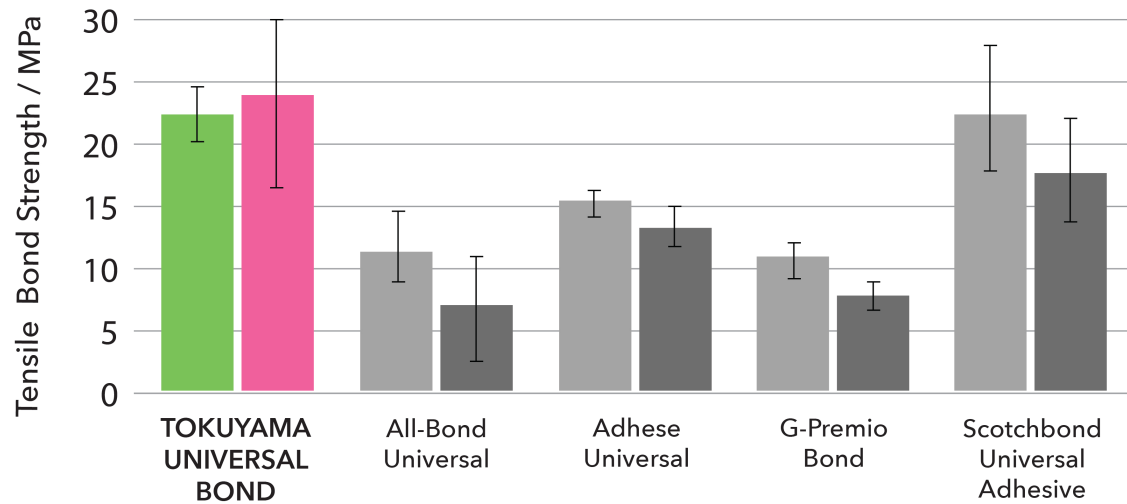

Self-Etch / Dentin

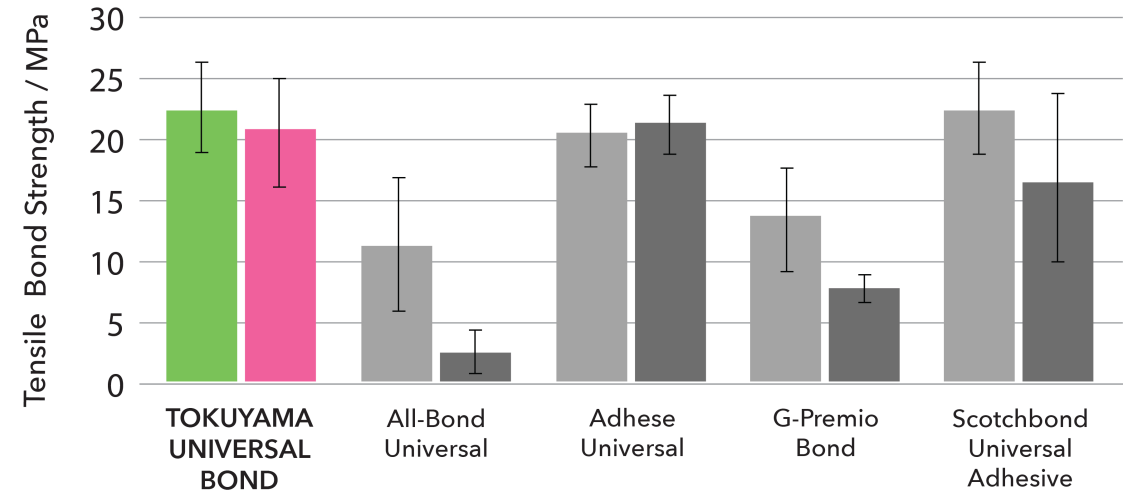

After 24hrs




After 3,000 thermo-cycling


COMPATIBILITY

Tokuyama UNIVERSAL BOND

Light-Curing Universal Composite

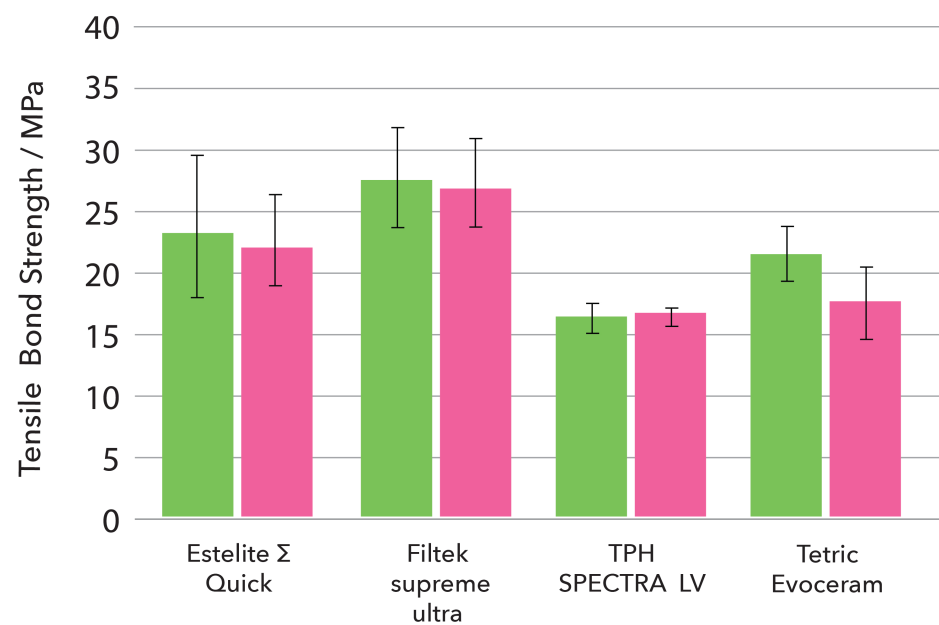

Light-Curing Flowable Composite

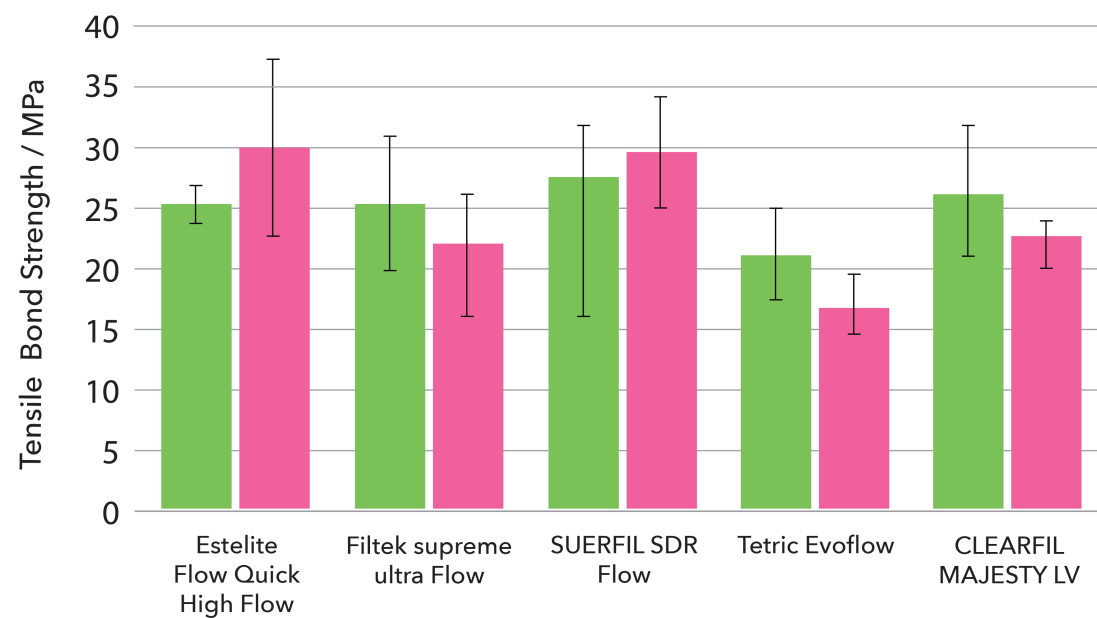


● Enamel after 24hrs

● Dentin after 24hrs

COMPATIBILITY

Tokuyama UNIVERSAL BOND


Dual-Curing Composite / Dentin

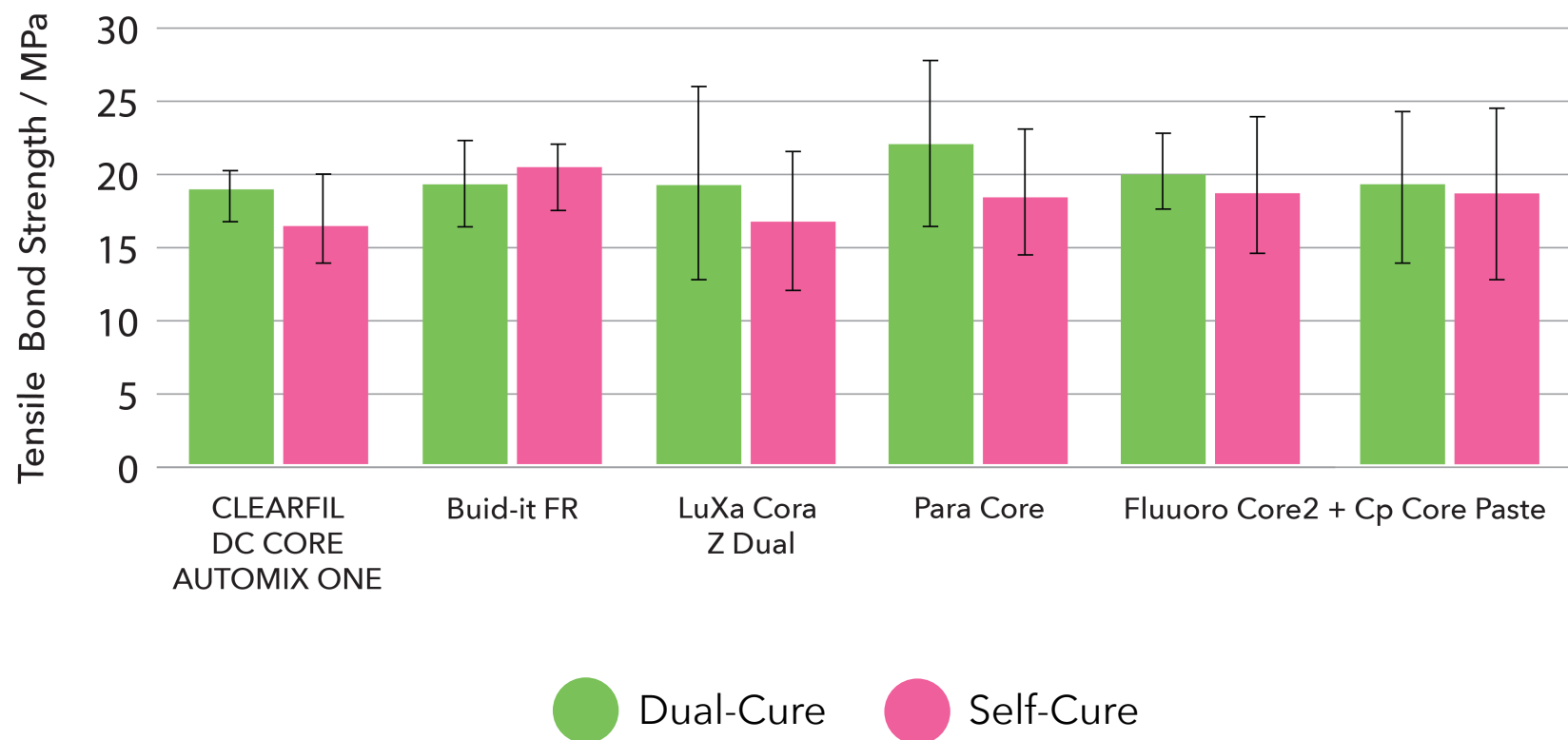


INTRAORAL REPAIR

SHEAR BOND STRENGTH

Lithium Disilicate Silica Sandblasting

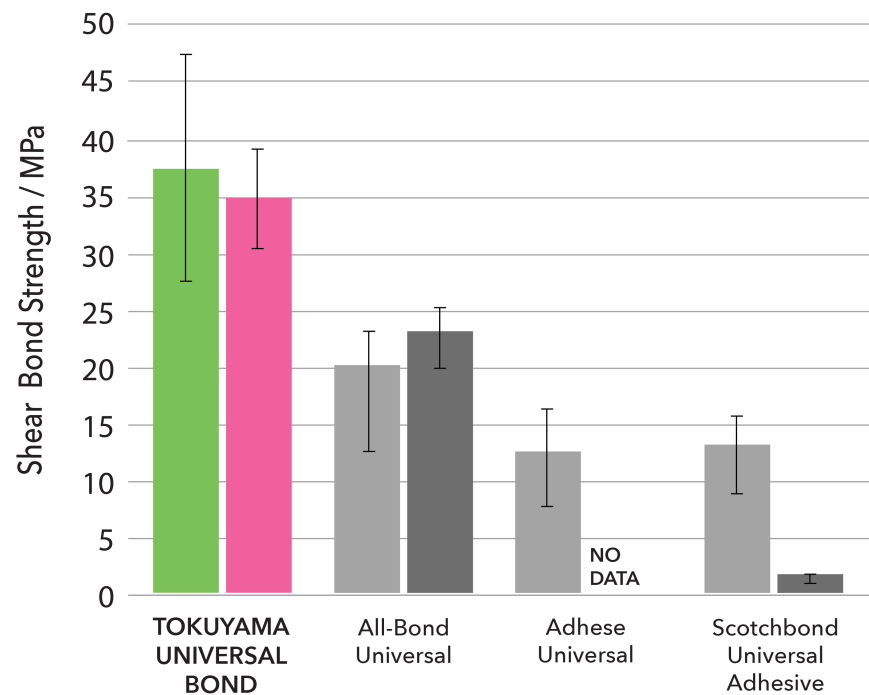


Zirconia Silica Sandblasting

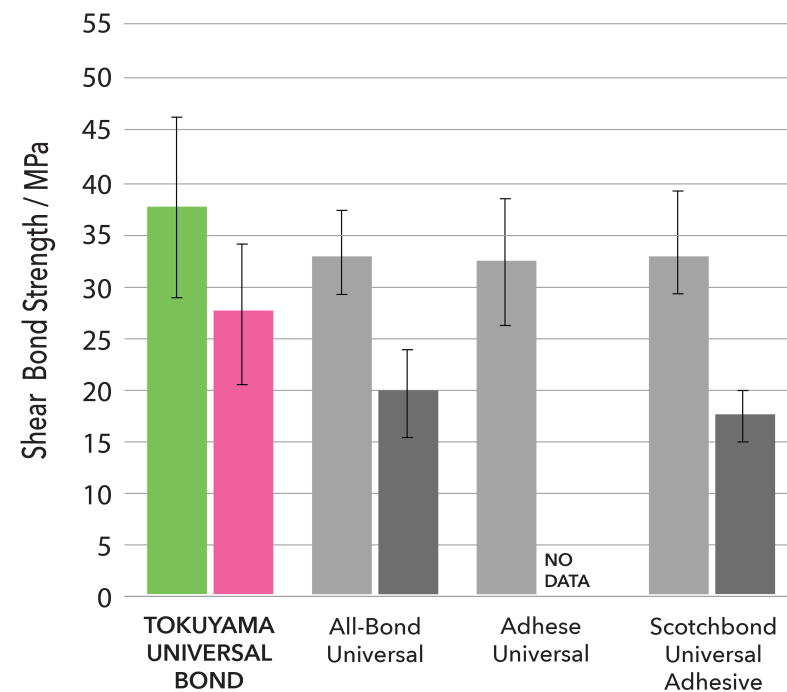


INTRAORAL REPAIR

TENSILE BOND STRENGTH METAL

Precious Metal

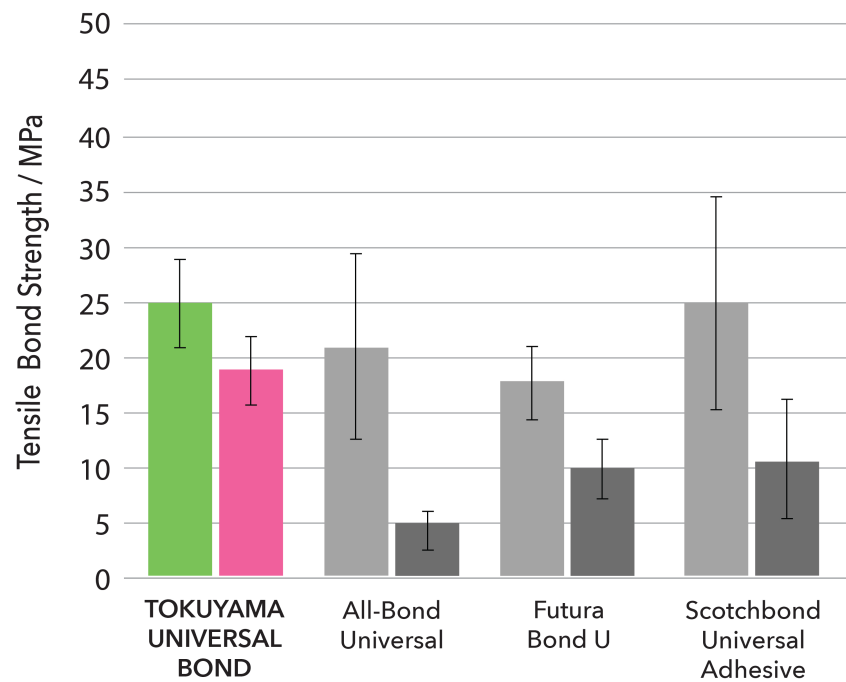

Non-Precious Metal

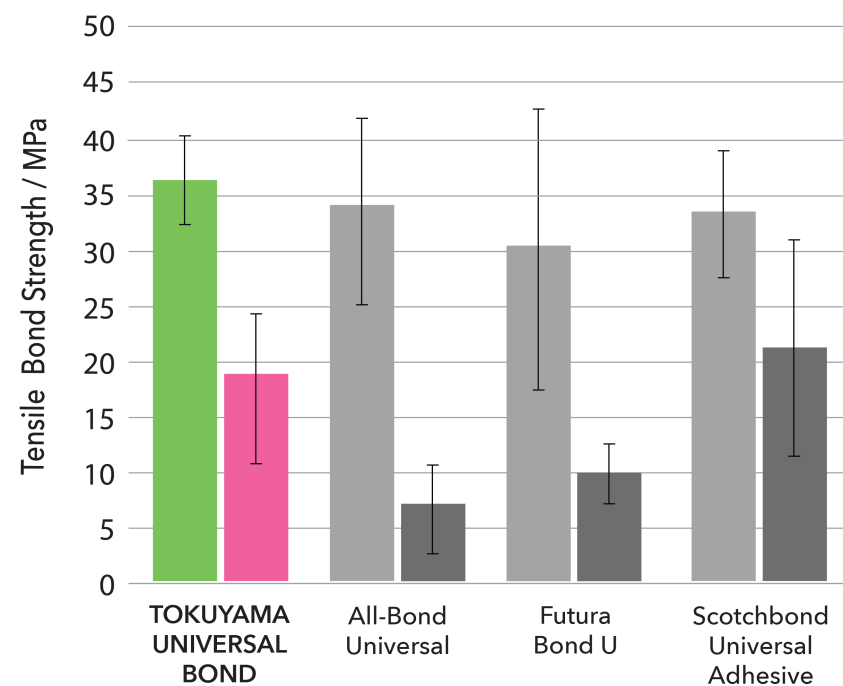

After 24hrs




After 3,000 thermo-cycling


INTRAORAL REPAIR

TENSILE BOND STRENGTH

Ceramics

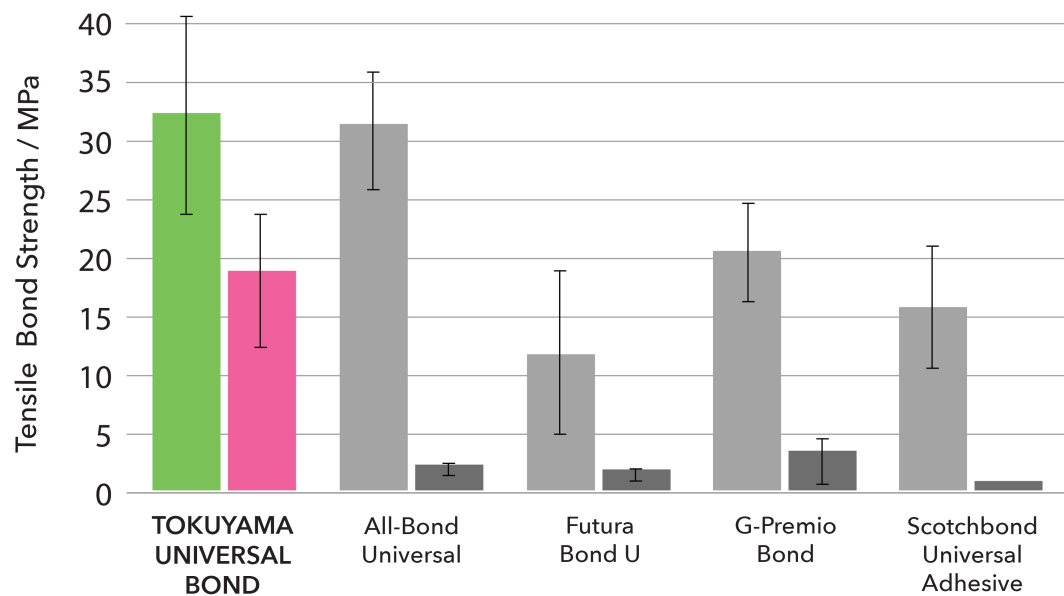

Indirect Composite

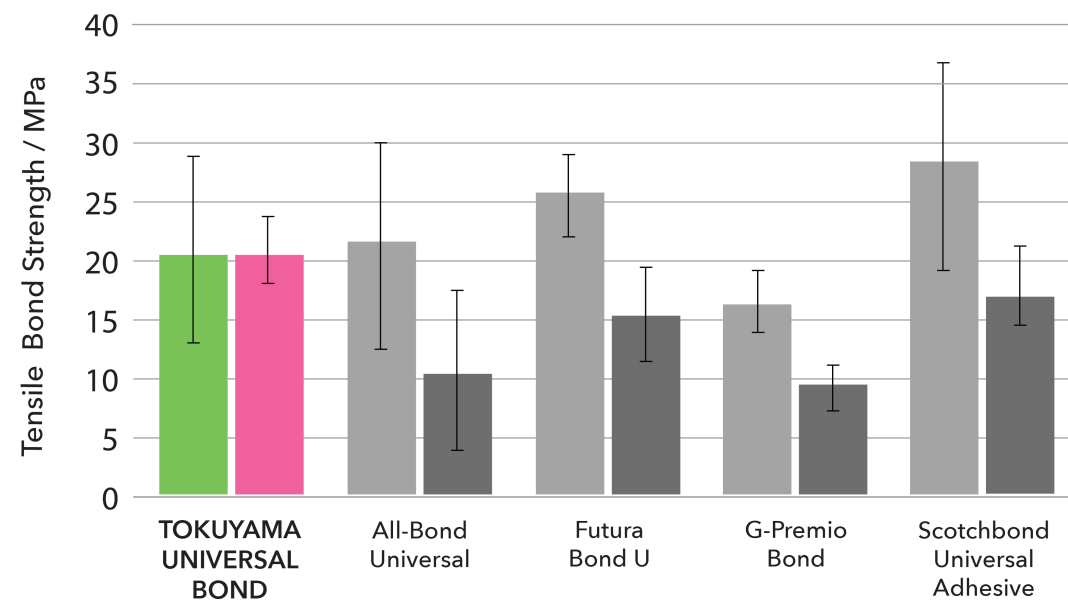

After 24hrs



After 3,000 thermo-cycling

COMPOSITION

Tokuyama UNIVERSAL BOND



Basic Components

Function

Phosphoric acid monomer
(New 3D-SR monomer)

Formation of bonding layer.
Adhesion for tooth, zirconia,
alumina, and non-precious metal

MTU-6

Adhesion for precious metal

HEMA

Penetration into the tooth sub-
stance Formation of bonding layer

Bis-GMA

Formation of bonding layer

TEGDMA

Formation of bonding layer

Acetone

Solvent



Basic Components

Function

γ -MPTES

Adhesion for glass ceramics
and resin composite

Borate

Polymerization catalyst

Peroxide

Polymerization catalyst

Acetone

Solvent

Isopropyl alcohol



Solvent

Water

Solvent


MECHANISM OF ADHESION TO TOOTH

3rd Generation 3D-SR Monomer

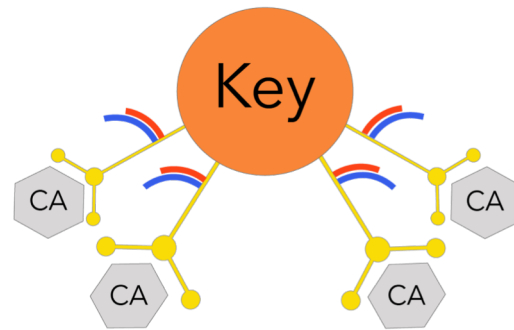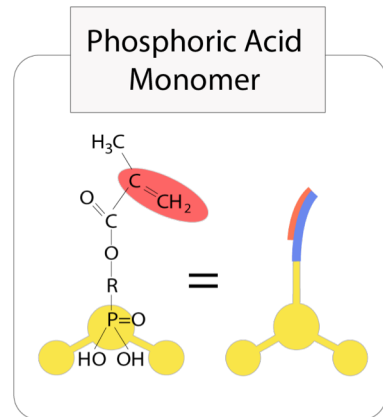


1. Strong bonding to tooth structure surface through multi- point bounding.

2. Improved adhesion layer strength through three- dimensional cross-linking reactions.

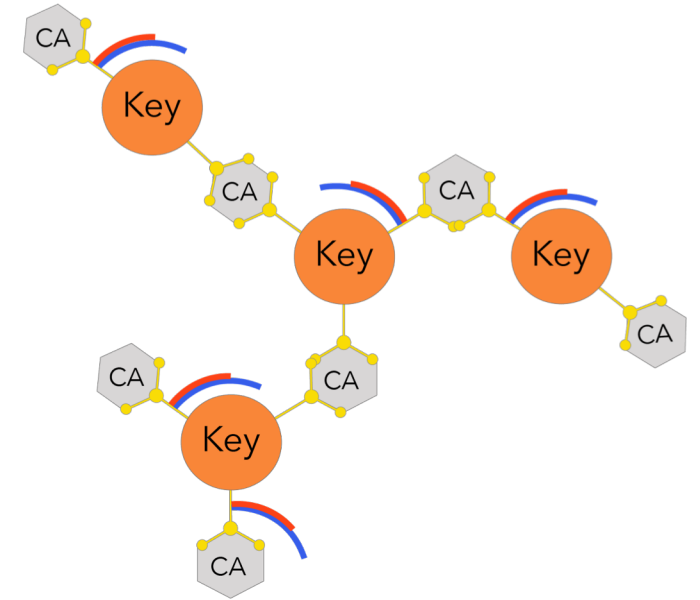


The 3D-SR monomer have several functional groups that can interact with calcium and polymerizing groups per molecule. **Tokuyama UNIVERSAL BOND** has an enhanced response to tooth calcium and durability by using a new 3rd generation 3D-SR monomer.

MECHANISM OF ADHESION TO PRECIOUS METAL

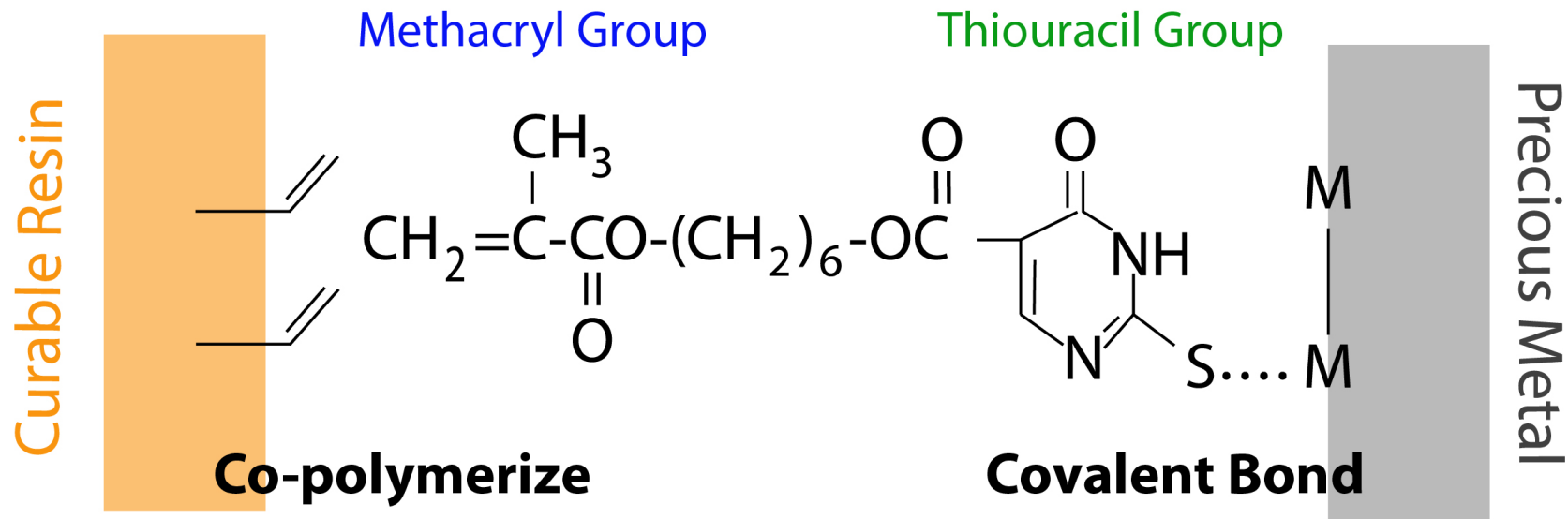



The sulfur atom in the thiouracil group of MTU-6 interacts with precious metal (covalent bond) and additionally, the methacryl group co- polymerizes with monomers in dental- curable materials.

MECHANISM OF ADHESION TO NON-PRECIOUS METAL

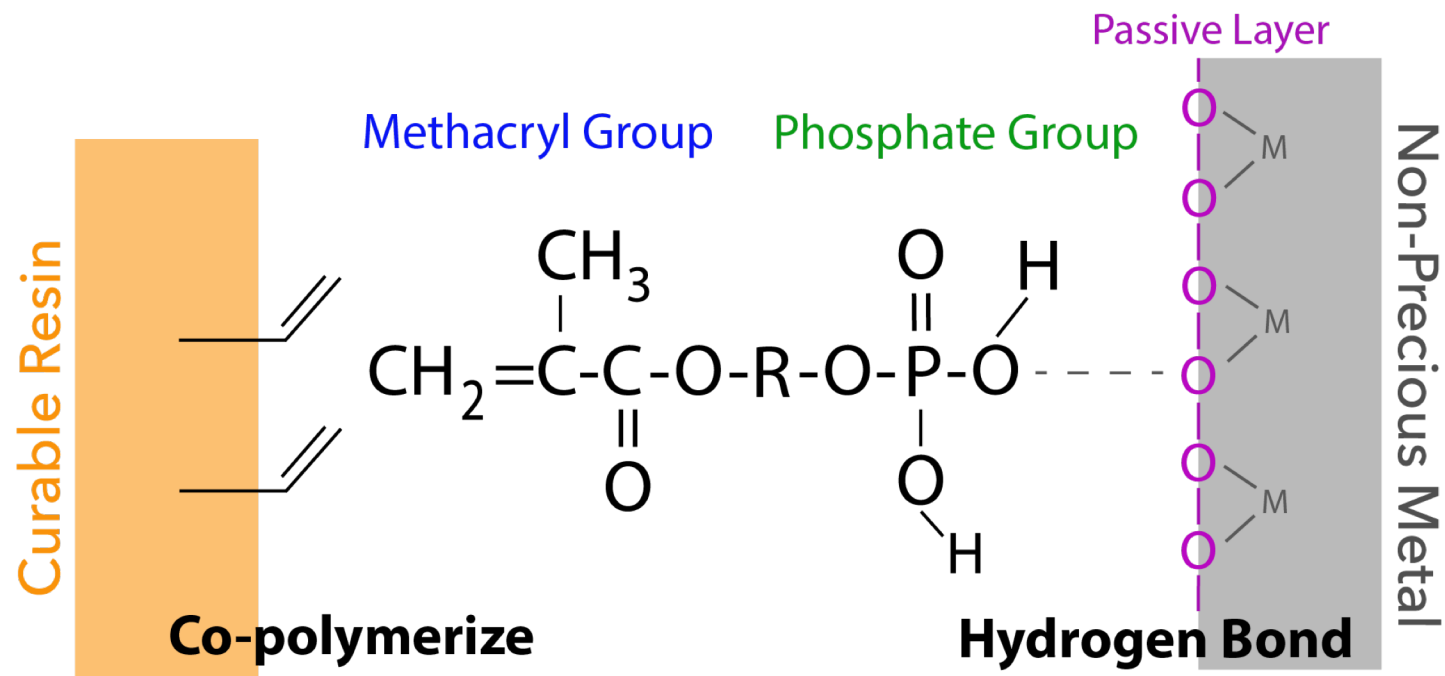


The phosphate group of new 3D-SR monomer interacts with the oxygen atom of the passive layer of a non-precious metal surface (hydrogen bond) and additionally, the methacryl group co-polymerizes with monomers in dental curable materials.

MECHANISM OF ADHESION TO GLASS-CERAMICS/RESIN

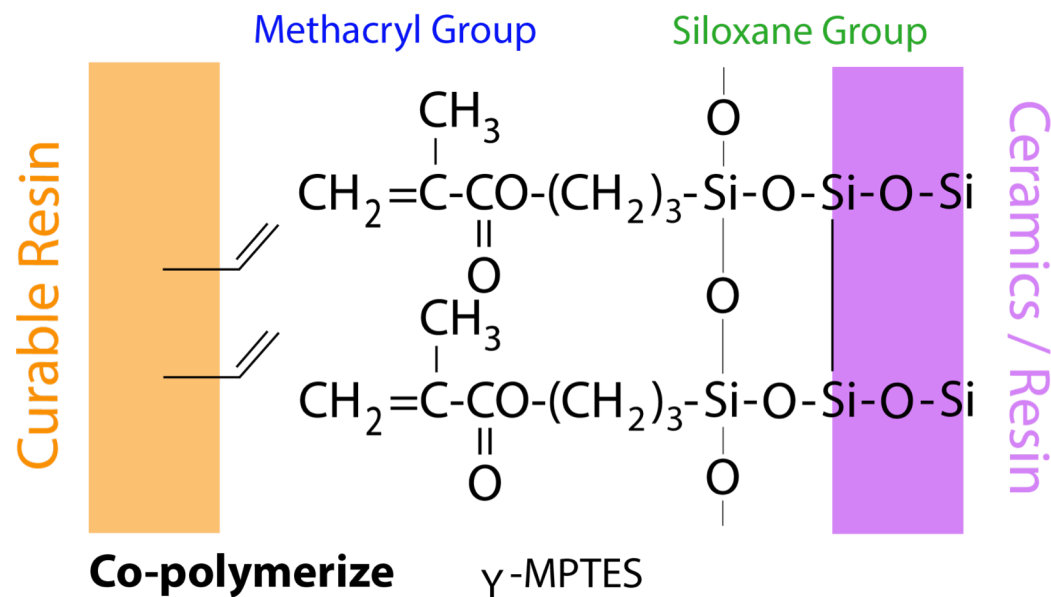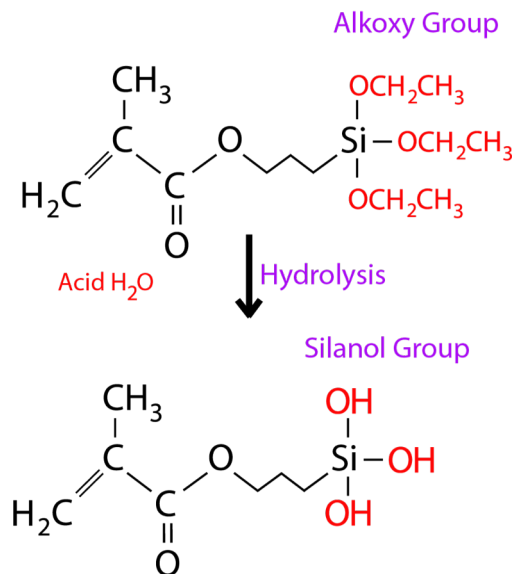

The alkoxy group in γ -MPTES reacts with water to form a silanol group and next, a siloxane bond is formed by a dehydration and condensation reaction with the silanol group on the ceramic surface. Additionally, the methacryl group co-polymerizes with monomers in dental curable materials. Since the new silane coupling agent, γ -MPTES is more stable in the bottle than the conventional one (γ -MPS), the adhesion effect lasts for a long time.


MECHANISM OF ADHESION TO ZIRCONIA/ALUMINA

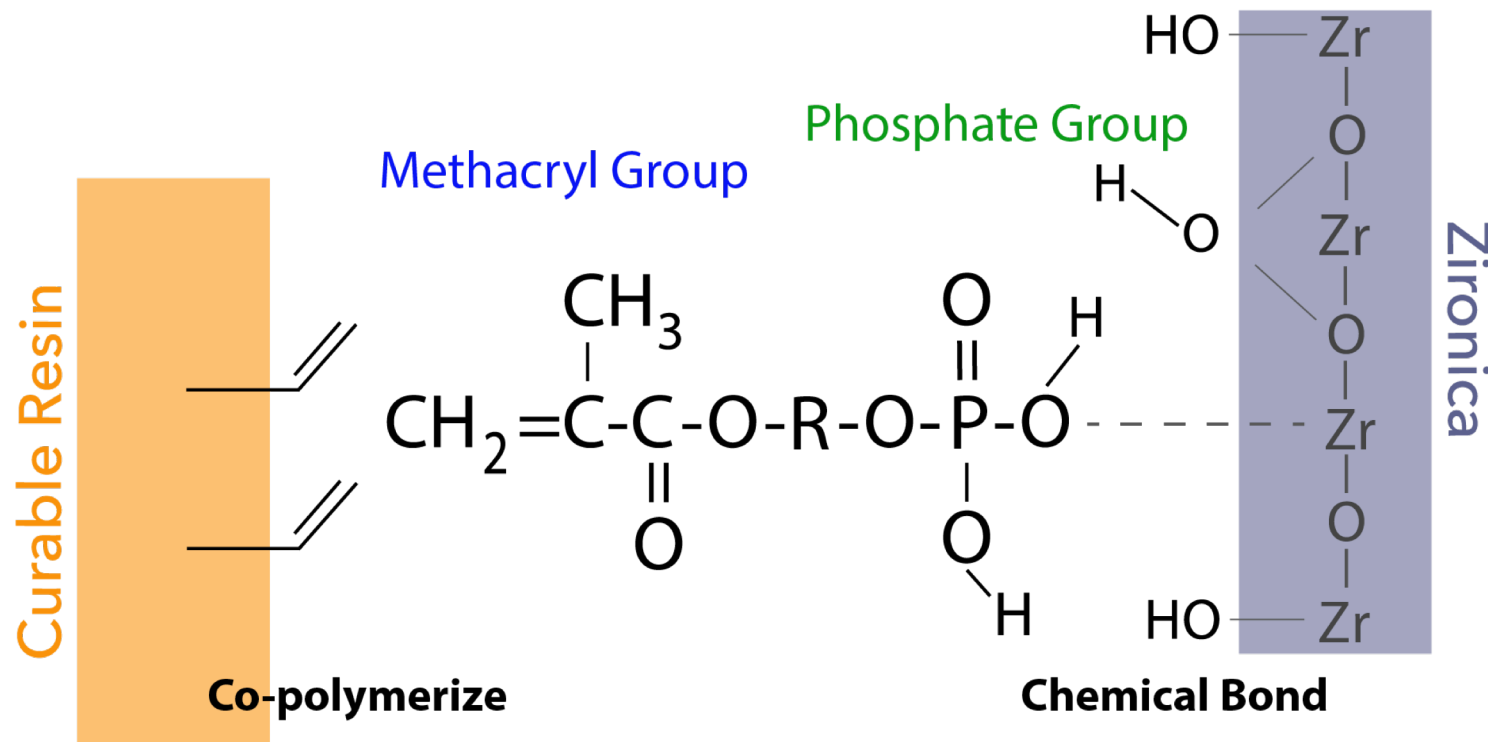

It is believed that the phosphate group of the new 3D-SR monomer forms chemical bonds with the zirconia/alumina surface for adhesion.

MECHANISM OF POLYMERIZATION INITIATOR

Active-Chemistry Technology

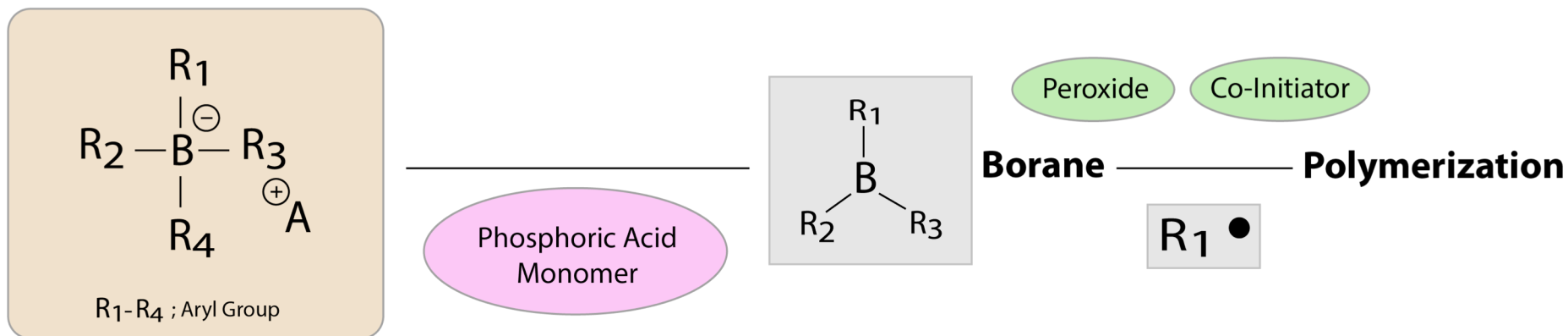

Active - Chemistry technology is superior to the conventional chemical polymerization initiator, a benzoyl peroxide/amine system, because it exhibits high catalytic activity under strongly acidic conditions. A thin bonding layer formed after air blow becomes hard because of rapid progression of polymerization and curing on its adhesive interface (Contact Cure), when it comes into contact with resin materials such as composite resin.