
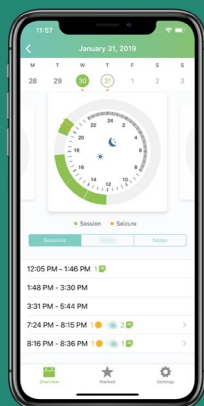


# Try these out

1. Use the Settings menu to adjust the minimum duration of the flashlight when a seizure is detected.
2. Epihunter detects absence seizures automatically. If seizures are missed, press the large button on the main screen to log seizure start and end manually.
3. The top right icon on the main screen (headset symbol) indicates the quality of the EEG signal. Make sure to adjust the headband so that this icon is green. This ensures the best possible signal and will be confirmed with two short beeps.
4. The video functionality can be enabled/disabled in Core under Settings. Recorded videos will be made available in Companion.

Detect, log, film and signal seizures with the epihunter Core app

Search for epihunter Core in

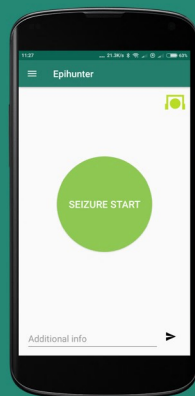

Consult seizure data and grant family members access in the epihunter Companion app

Search for epihunter Companion in



# Headset notifications


Blue light	Module is turned on and is connected to the smartphone
2 beeps	When the module turns on or when the sensors are well connected with the forehead
Red & blue light	Module is turned on but has no connection with the smartphone
1 beep	Module is turned off
4 beeps	Module battery is empty
Red light	Device is charging

Headset

Module



Charging cable

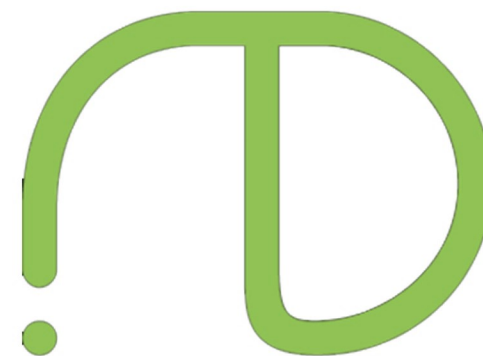

# Instructions

# Charging the headset

## Charge before first use

Battery time of the module is estimated at 3-4 hours.

1. Make sure to use the special charging cable. You can find it under the foam material in the headset box.
2. Gently remove the module from the headset and connect the charging cable.
3. Please allow 30-40 minutes for a full charge.
4. The smartphone can be charged while using epihunter. Placing the smartphone in the supplied smartphone holder in landscape mode allows for easy cable access.

# Headset configuration

## First time only

The epihunter Core app is available only in Google Play store.

1. Make sure the module is charged and the smartphone has enabled Bluetooth.
2. Press the blue button on the module until you hear two beeps to turn on the headset.
3. Go to the epihunter Core app and press "Configure". You might need to grant epihunter certain permissions.
4. "BrainLink\_Lite" should appear on the headset list.
5. Press "Connect".
6. Place the headset on your head with the module sitting over your left ear and the three dry sensor on the left side of your forehead.
7. Two beeps confirm that the sensors are well connected with the forehead and recording brain waves!
8. You're all fine, seizures will be detected.

# Don't throw me away

Watch the video tutorial on our Youtube channel:  
Epihunter



Watch the video tutorial on our Youtube channel:  
Epihunter

