

What to expect. A surgeon's perspective.

by **Envoy Medical Staff Member**,
on November 23, 2018

Can you imagine having your cake and eating it too? That might be what you can expect with the Esteem® hearing implant. The Esteem is a hearing solution that alleviates your hearing loss, but without the external device and without the [hassles or frustrations of hearing aids](#) – an all day, everyday hearing solution that works with you and your lifestyle.

If you're curious about what to expect from the Esteem procedure, you're in luck. Here to share insight into the procedure is Dr. Jack Shohet, a neurotologist who has completed over 300 Esteem cases, more than any other active surgeon in the U.S.



1. What is the Esteem implant procedure like?

Surprising to some, the Esteem implant procedure is actually an outpatient procedure, meaning patients come in and out of the office on the same day. The whole process takes only several hours - patients are administered general anesthesia, we place the implant components and then patients are free to go.

2. Can you describe the average recovery process?

After the initial procedure, patients return either to me or their primary doctor for a quick follow up. The patients then wait about two months for the wound to heal before activating the Esteem implant. The entire process is very quick. The best part is patients don't have to come back to the office every two to seven months to repeat the process because the battery lasts five years on average and can last up to eight.

3. How have your patients responded to hearing with Esteem?

Overall, most patients have had a positive experience with the implant and realize a significant improvement in not only their hearing, but in their overall lifestyle. The device is aptly called Esteem. My patients with hearing loss tend to be withdrawn and isolated because it is difficult for them to be fully engaged in conversations and other social activities. When they can hear, I can sense their personalities changing. They become more extroverted and are more confident to participate in conversation. The device truly gives patients better esteem.

4. Why should a patient consider Esteem over other hearing devices?

Esteem is truly unique. It is the only totally implanted hearing device in the U.S., which means there are no external components. This feature allows for a significant improvement in overall lifestyle – patients don't have to keep track of a device, worry about cleaning it or take anything off when they participate in activities such as swimming. Patients with Esteem can enjoy life without having to even think about their implant.

Ready to move forward in your journey toward a better life? **Get in touch** to speak with an Esteem Counselor today.

Federal Law restricts the Esteem device to sale by or on the order of a physician. The practitioners who may implant this device require specific training. Risks include those typical for ear surgery. Taste disturbance is possible, but is almost always temporary. Side effects related to facial movement are possible, although rare. It is possible that additional surgery may be required to enhance performance or resolve complications. Individual results may vary. For additional information on the indications, contraindications and safety information regarding the Esteem, please go to the Important Safety Information link at <https://www.envoymedical.com/safety-information>

Envoy Medical Corporation
4875 White Bear Parkway
White Bear Lake, Minnesota 55110
United States of America
877-900-EARS (3277)
www.envoymedical.com

Esteem and Envoy Medical
are trademarks of Envoy
Medical Corporation.

All rights reserved.
2020 © Envoy Medical
Corporation or its affiliates.

Ref. # 100067-001 Rev. 01

ENVOY MEDICAL