




# SysLoc<sup>®</sup> MINI **Troubleshooting Guide**


JMS North America Corp.




# SysLoc<sup>®</sup> MINI Diagram

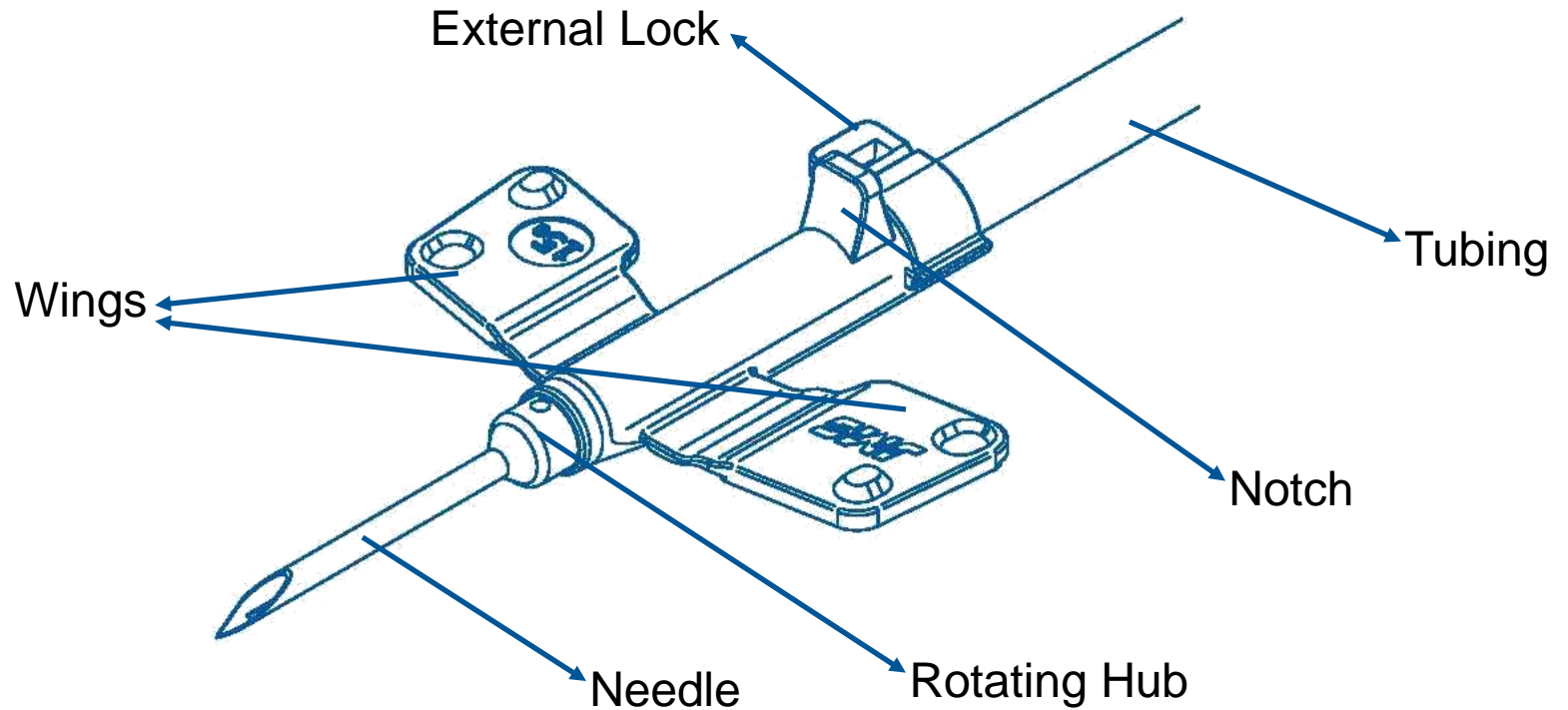

# Depth of Insertion Do's & Don'ts

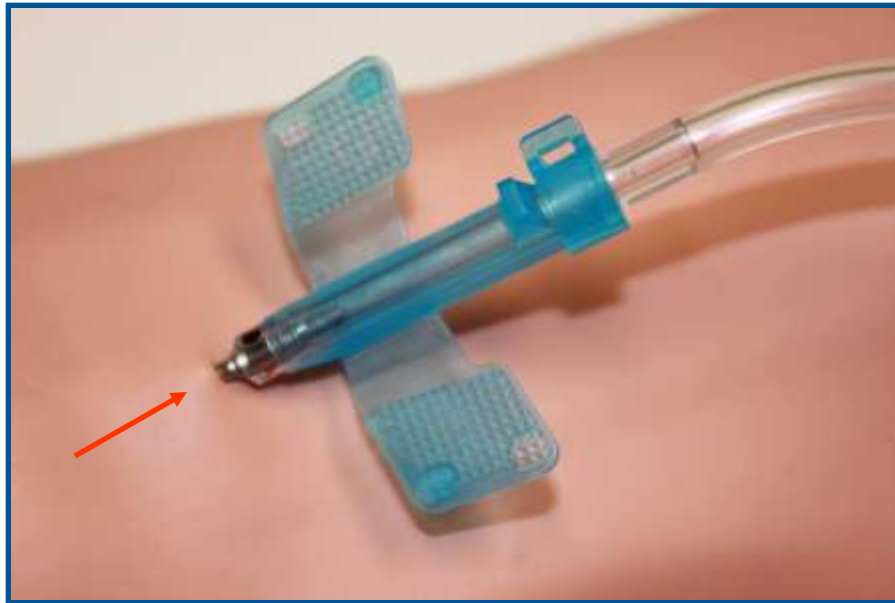

Leave a small amount of the needle visible to prevent oozing.

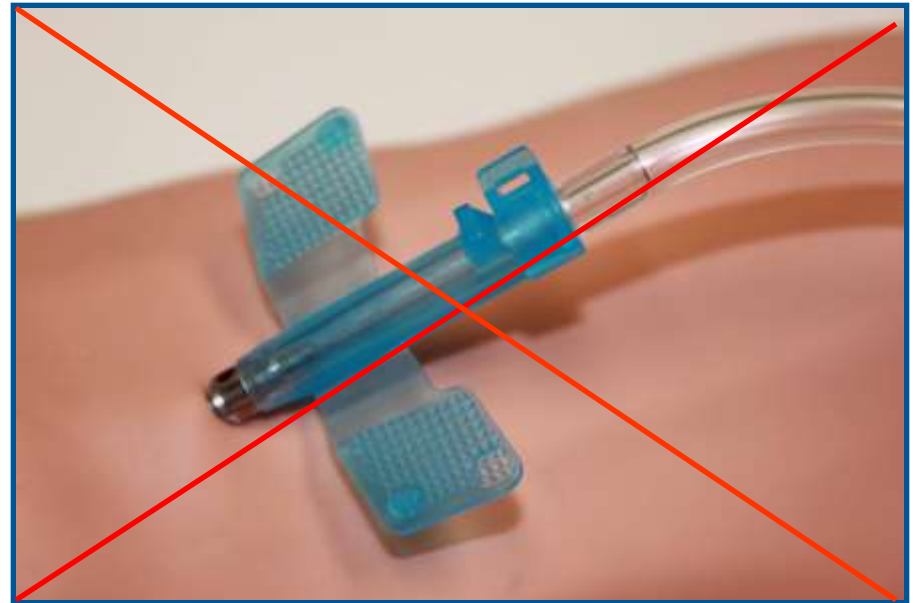

Inserting too far will cause the tissue to spread and may cause unnecessary oozing.





# Preparing for removal

- ❖ Keep last piece of tape in place until external lock is unlocked.
  - This will maintain needle stability and prevent unnecessary discomfort to the patient.



# Needle Removal: Unlocking the external LOCK

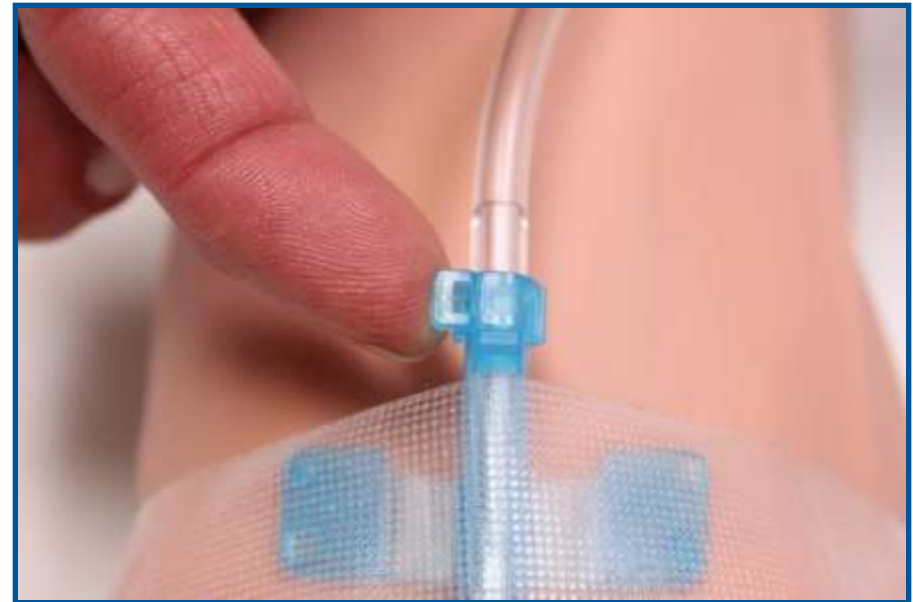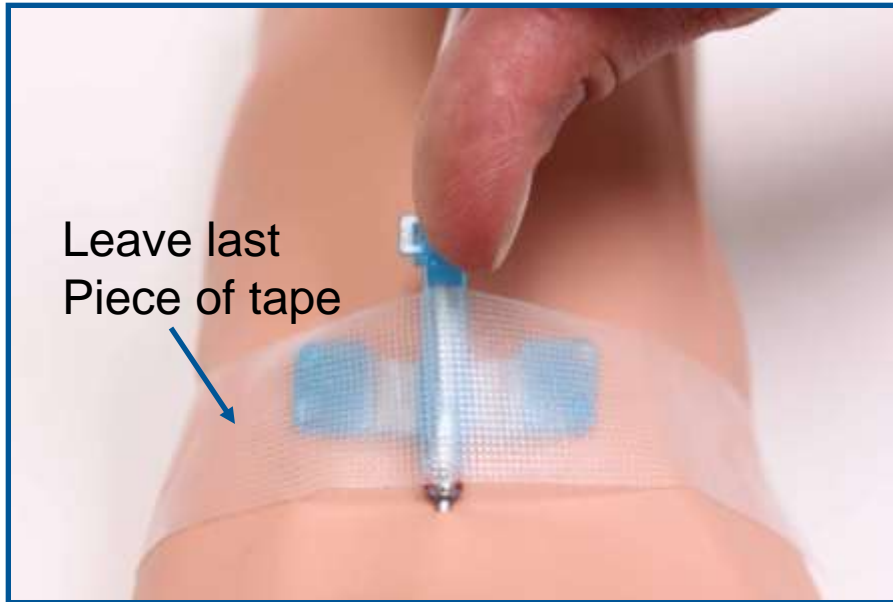

**1:** Place thumb on external notch

**2:** Place index finger on external locking mechanism



# Needle Removal:

## Unlocking the external LOCK - Cont.



**3:** Using your thumb and index fingers, unsnap external lock

**4:** Move external lock out of your way




# Needle Removal:

## Unlocking the external LOCK - Cont.




**5:** Remove last piece of tape

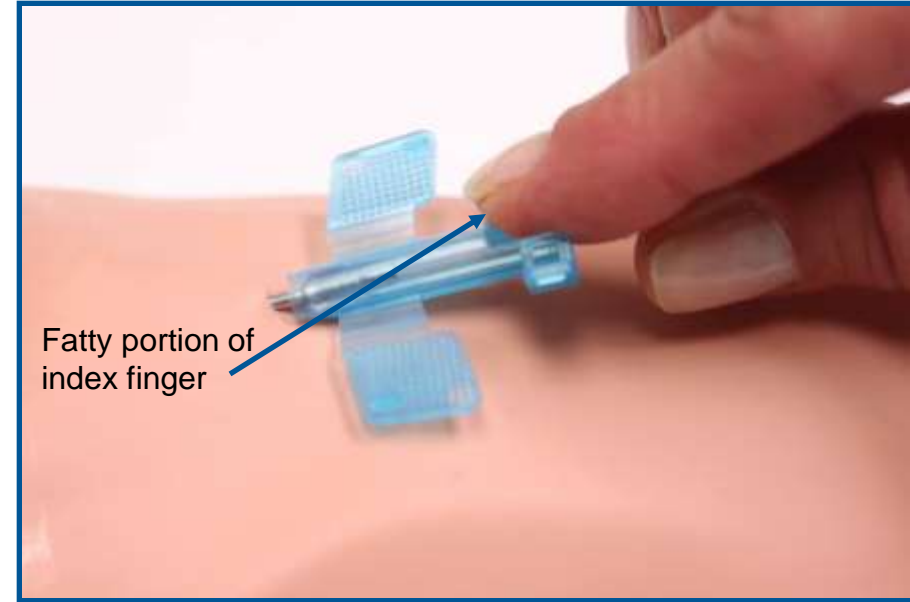

**6:** Prepare for retraction place thumb and middle finger on the clear plastic tubing directly behind the external lock. Use fatty portion of index finger rather than nail for better control.



# Needle Removal: Unlocking the external LOCK - Cont.

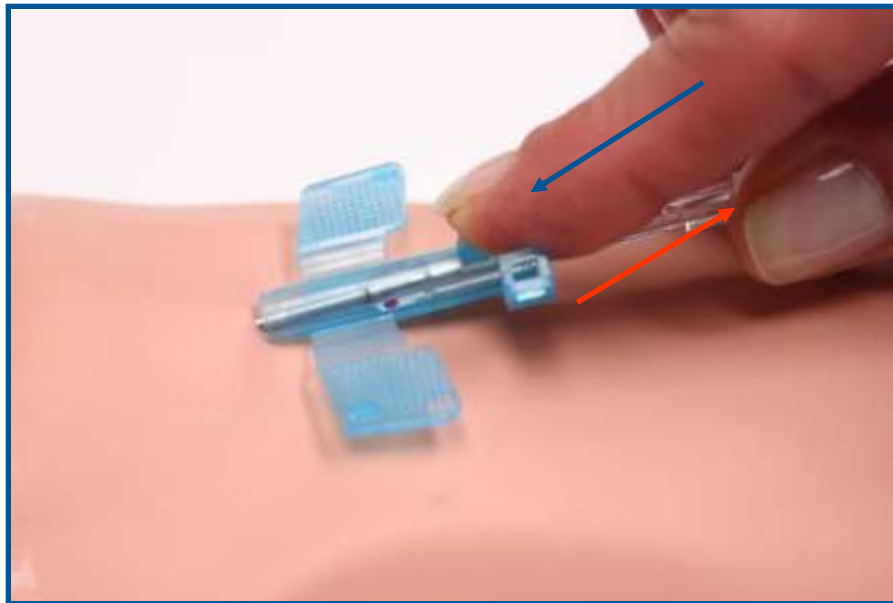

**7:** Start retraction by pushing index finger **forward** while simultaneously **pulling back** with thumb and middle finger.



**8:** To ensure complete retraction has been achieved continue to pull straight back until audible **click** is heard.

\*TIP: you may find that applying a slight amount of downward pressure facilitates complete retraction.





# Contact Us!

❖ For more information or questions please contact us at:

- P: 954 689 9280
- F: 954 689 9289
- [customerservice@jmsna.net](mailto:customerservice@jmsna.net)

## JMS North America Corp

