



# Harmony<sup>®</sup> Needle User's Guide

Main Office  
22320 Foothill Blvd.  
Suite 350  
Hayward, CA 94541  
Ph: 510 888 9090  
F: 510 888 9099  
customerservice@jmsna.net

Sales & Marketing  
480 Sawgrass Corporate Pkwy.  
Suite 120  
Sunrise, FL 33325  
Ph: 954 689 9280  
F: 954 689 9289  
www.jmsna.net



Integrated cap for aseptically removing the scab.



Clean button-hole site according to policy.



Tear pouch, remove Harmony<sup>®</sup> needle from package and inspect for damage or particles.



Without removing the cap, grasp the wings and cap together with the thumb and the forefinger.



Using the tip of the cap, remove button-hole scab.

**Repeat disinfection procedure according to policy before attempting cannulation.**



Remove cap.



Align Harmony<sup>®</sup> needle cannula with bevel facing up over the button-hole site.



Cautiously insert Harmony<sup>®</sup> needle into the established button-hole track ensuring the same angle and depth of insertion are used with each cannulation.



**Observe pulsation of blood in the plastic tubing when the Harmony<sup>®</sup> needle is in the button-hole access.**



Securely tape according to clinic policy and start treatment per facility procedures.

