



Care@Home

CMS Fall Test Technical Note

ESUGSC105
Version 1.0
January 2019

Managing EPA Fall Testing

The fall test is a method for testing an EPA's fall detection. Refer to ESUGSC032 Care@Home EPA User Guide. If enabled by the administrator, fall tests generate a medical alarm (MA).



NOTE: This technical note refers to Care@Home™ CMS version 6.04 or later, released with Care@Home™ version 6.0.6

To enable/disable fall testing:

1. Log on to CMS.
2. From the **Main** menu, click **Control Panel**. The **Control Panel** screen appears.
3. Click **More Options**. The **Control Panel Configuration Extension** screen appears.
4. Click **EPA Config**. The **EPA Config** tab appears.
5. Select/deselect **Enable Free Fall**.
6. Click **Update**.

Legal Notice

Usage of this document, and all information (including product information) provided within, are subject to the following terms and conditions, and all applicable laws. If you do not agree with these terms, please do not access or use the remainder of this document.

This document contains highly confidential information, which is proprietary to Essence SmartCare Ltd. and/or its affiliates (hereafter, "Essence"). No part of this document's contents may be used, copied, disclosed or conveyed to any third party in any manner whatsoever without prior written permission from Essence.

The information included in this document is intended for your knowledge and for negotiation purposes only. Essence makes no implicit representations or warranties with respect to such information.

The information included in this document is subject to change without notice. Any decision to rely on the information contained herein shall be at your sole responsibility, and Essence will not accept any liability for your decision to use any information or for any damages resulting therefrom.

Certain laws do not allow limitations on implied warranties or the exclusion or limitation of certain damages. If these laws apply to you, some or all of the above disclaimers, exclusions, or limitations may not apply to you.

By using the information contained herein, you agree that the laws of the State of Israel, without regard to principles of conflict of laws, will govern any dispute of any sort that might arise between you and Essence regarding the information contained herein, and any such dispute shall be settled exclusively in the competent courts of Tel Aviv-Jaffa, Israel.

All registered or unregistered trademarks, product names, logos and other service marks mentioned within this document are the property of Essence, or their respective owners. Nothing contained herein shall be construed as conferring by implication, estoppels, or otherwise any license or right, either express or implied, under any patent or trademark of Essence or any third party. No use of any trademark may be made without the prior written authorization of Essence.

This document and all of its contents are protected intellectual property of Essence. Any copying, reprinting, reuse, reproduction, adaptation, distribution or translation without the prior written permission of Essence is prohibited.

Please check your End User License Agreement (EULA) for terms and conditions.

© 2019 All rights reserved to Essence SmartCare Ltd.

For more information, please contact:

Essence SmartCare Ltd.
12 Abba Eban Avenue,
Ackerstein Towers Bldg. D
Herzliya Pituach, 4612001 Israel
www.essence-grp.com
Tel: +972-73-2447777
Fax: +972-9-7729962