

BANDGRIP®

MICRO-ANCHOR SKIN CLOSURE

SPEED • SIMPLICITY • STRENGTH



HEALING BEGINS WITH A SECURE CLOSURE.

With a firm grip and lasting hold, BandGrip® Micro-Anchor Skin Closures hold surgical incisions and lacerations edges closed with simplicity and strength. Patented curved Micro-Anchors are gently secured to the skin and the wound edges are slightly everted as BandGrip is pulled across the defect. BandGrip is transparent and not made with natural rubber latex.

- **Non-Invasive.** Closes the skin without needles or staples, reduce concerns of negative cosmetic outcomes and the risks associated with needle sticks.
- **Simple and Intuitive.** Provided in a easy to handle 9 x 4 cm size, BandGrip can be successfully applied by a wide range of health care professionals in addition to doctors.
- **Efficient and Economical.** BandGrip, used alone or in multiples, is a timesaving alternative to current closure modalities. Closure of a small incision was timed at <30 seconds* compared to sutures. BandGrip does not require a return visit for removal if determined by the patient's doctor.

* Data on file

Ask for Bandgrip® Micro-Anchor Skin Closure | www.bandgrip.com

311 S. Wacker Dr. Suite 650 Chicago, IL 60606 | Tel: 844.968.6322 | info@BandGrip.com

A DIFFERENCE PATIENTS CAN SEE AND FEEL



85-year-old male 5 1/2 weeks post total knee arthroplasty

Micro-Anchors Make The Difference

Unique, curved Micro-Anchors gently and securely grip the skin to keep the wound edges closed during the critical healing period. By design, common problems associated with conventional closure materials are potentially avoided.

A Difference Patients Can See and Feel

BandGrip Micro-Anchor Skin Closures can improve patient experience both in recovery and at home on their own.

- **Familiar Look and Feel.** BandGrip can bring patients a sense of normalcy and confidence.
- **Supports Ambulation.** Unlike the raised surfaces of staples and zippers, and the “whiskers” associated with sutures, the smooth surface of BandGrip Micro-Anchor Skin Closures don’t catch on dressings or clothing; patients can shower with them on after 24 hours.

| Product Code | Size | Attributes |
|--------------|--------------|--------------------------------------|
| BG1001 | 9 x 4 cm | 2 Micro-Anchor Strips |
| BG1002 | 9 x 4 cm | 4 Micro-Anchor Strips |
| BG1002F | 9 x 4 cm | 4 Micro-Anchor Strips and perforated |
| BG1003 | 6.5 x 2.5 cm | 2 Micro-Anchor Strips |
| BG1003F | 6.5 x 2.5 cm | 2 Micro-Anchor Strips and perforated |

Application Instructions

- 1 Clean wound according to facility protocols.
- 2 Ensure the application area is clean and dry.
- 3 Open sterile pouch using aseptic technique.
- 4 When taking the backing off, make sure gloves are dry to maintain the integrity of the adhesive.
- 5 Press and apply one side of the BandGrip firmly on skin, adjacent to wound edge.
- 6 With other hand, pinch and hold skin edges closed. Forceps may also be used.
- 7 Gently stretch BandGrip across pinched/closed skin edges and press BandGrip down on the other side of the wound to secure in place.
- 8 BandGrip can be tiled or applied along the length of the wound approximately 3mm apart.

Removal Instructions

- 1 Release the edges of BandGrip.
- 2 Support the skin adjacent to the wound as BandGrip is removed.
- 3 Gently pull back one side of BandGrip in the opposite direction of the wound to release the Micro-Anchors and to minimize any damage to the epidermis. Pull across to the other side to remove completely.

Notes: BandGrip Micro-Anchor Skin Closures are designed for continuous use for up to 15 days. Tiny red pin holes marks may be visible upon removal and may last up to 2 weeks.

Speed * Simplicity * Strength

Healing begins with a secure closure.



BandGrip is a registered trademark of BandGrip, Inc.

BandGrip®, Inc.
311 S. Wacker Dr. Suite 650
Chicago, IL 60606

Tel: 844.968.6322
info@BandGrip.com