

# Current guidelines on how to set the ventilator for (COVID-19) ARDS patients

23.03.2020 📅

Author: Munir Karajaghli, Reviewer: Bernhard Schmitt

In this article you will find a concise list of guidelines\* on setting the ventilator for ARDS patients.

- Tidal volume: 4 – 6 ml/kg predicted body weight (PBW) 1,2
- Maintain plateau pressure < 30 cmH<sub>2</sub>O
- Maintain driving pressure < 14 cmH<sub>2</sub>O 3
- To target lung protective ventilation settings it may be necessary to apply the concept of permissive hypercapnia (pH > 7.15)
- Oxygenation goal: PaO<sub>2</sub> 55 - 80 mmHg, SpO<sub>2</sub> 88 - 95%, PEEP and Oxygen (FiO<sub>2</sub> in percent) settings according the ARDSnet tables (currently for COVID-19 the higher PEEP table is recommended<sup>2</sup>)

## Lower PEEP/higher FiO<sub>2</sub>

FiO <sub>2</sub>	0.3	0.4	0.4	0.5	0.5	0.6	0.7	0.7
PEEP	5	5	8	8	10	10	10	12

FiO <sub>2</sub>	0.7	0.8	0.9	0.9	0.9	1.0
PEEP	14	14	14	16	18	18-24

## Higher PEEP/lower FiO<sub>2</sub> (currently recommended for COVID-19 patients)

FiO <sub>2</sub>	0.3	0.3	0.3	0.3	0.3	0.4	0.4	0.5
PEEP	5	8	10	12	14	14	16	16

FiO <sub>2</sub>	0.5	0.5-0.8	0.8	0.9	1.0	1.0
PEEP	18	20	22	22	22	24

\* ARDSnet: [http://www.ardsnet.org/files/ventilator\\_protocol\\_2008-07.pdf](http://www.ardsnet.org/files/ventilator_protocol_2008-07.pdf)

## References

1. Papazian, Laurent, et al. "Formal guidelines: management of acute respiratory distress syndrome." *Annals of intensive care* 9.1 (2019): 69.
2. Kluge, S., Janssens, U., Welte, T. et al. Empfehlungen zur intensivmedizinischen Therapie von Patienten mit COVID-19. *Med Klin Intensivmed Notfmed* (2020).  
<https://doi.org/10.1007/s00063-020-00674-3>
3. Amato, Marcelo BP, et al. "Driving pressure and survival in the acute respiratory distress syndrome." *New England Journal of Medicine* 372.8 (2015): 747-755.

For additional information on COVID-19:

WHO guidelines: <https://www.who.int/emergencies/diseases/novel-coronavirus-2019/technical-guidance>

ESICM information: <https://www.esicm.org/resources/coronavirus-public-health-emergency/>

Current evidence about COVID-19:

<https://jamanetwork.com/journals/jama/pages/coronavirus-alert>

Centers for Disease Control and Prevention, CDC: <https://www.cdc.gov/coronavirus/2019-ncov/hcp/clinical-guidance-management-patients.html>

Date of Printing: 31.03.2020

Disclaimer:

The content of this Knowledge Base is intended for informational purposes only. Hamilton Medical AG provides no warranty with respect to the information contained in this Knowledge Base and reliance on any part of this information is solely at your own risk. For detailed instructions on operating your Hamilton Medical device, please refer to the official Hamilton Medical Operator's Manual for the respective device.