EN	
All components should be cleaned of debris with a water-damp paper towel and disinfected with a paper towel saturated with an EPA-registered low-level (HIV/HBV claim) to intermediate-level (tuberculocidal claim) hospital disinfectant. Afterwards, the chemical residue should be wiped off with a water-damp paper towel and air-dried. Utilize the surface disinfecting protocol of the disinfectant manufacturer. Consult www.CDC.gov for the most current version of the "Guidelines for Infection Control in Dental Health-Care Settings".	
<b>Parkeℓℓ® LC Base Light-Cured, Composite Monolithic Restorative</b> <b>Store products tightly sealed, away from direct sunlight, extreme cold (below 34°F/1°C) or high heat (above 86°F/30°C).</b> Refrigeration is acceptable but not essential. Do not freeze. Bring chilled material to room temperature before use. Do not store near eugenol-containing materials.	
<b>Product Description</b> Parkeℓℓ's LC BASE is a resin-based, light-cured, bulk-fill composite restorative material for posterior and anterior restorations, where precise shade matching is not critical. LC BASE may be used for both direct and indirect restorations in bulk increments of up to 4mm per layer. It may be cured with any dental curing device (LED, Halogen, Plasma Arc (PAC) or Laser) that can produce light from 430-480nm at an intensity of >600 mW/cm <sup>2</sup> . Sample curing times and depths are presented in the following table. Test your curing light before use.	
<b>Curing time and Curing Depth of Parkeℓℓ LC BASE (Dentin Shade)</b> Conventional Halogen: 40 sec / 4,2 mm High Output halogen: 20 sec / 4,0 mm Plasma Arc: 9 sec / 4,3 mm High Output LED: 10 sec / 4,0 mm	
LC BASE contains BisMPPEF, polyglycol diacrylate, and aromatic urethane diacrylate. The product is approximately 70% filled by weight, with a range of inorganic filler particles from 0.007µm – 8.0µm.	
LC BASE bonds well to any of Parkeℓℓ's bonding agents, including Parkeℓℓ® Universal Adhesive, Brush&Bond®, Brush&Bond® Universal, Amalgambond® or C&B-Metabond. It may also work with bonding products from other manufacturers. Check instructions before use.	
<b>Indications and Intended uses</b> LC BASE is indicated for use in direct and indirect applications in both posterior and anterior restorations, where high compressive and tensile strength is required, but where precise shade match is not. The material cures to a depth of 4mm per increment, and flows well to conform to the cavity preparation. In restorations where interproximal contacts are being restored, special matrixing techniques should be used to ensure tightness.	
LC BASE (Dentin Shade) is highly flowable, and adapts well to irregular cavity preparations. It is indicated for use as the initial increment for deep restorations. It requires veneering with a composite restorative material (such as LC FILL) to complete the external contour of the restoration.	
<b>Contraindications, Precautions and Warnings</b> • LC BASE is for dental use only, and is not for internal consumption, other than as cured restorative materials. It should be kept away from children. • Avoid accidental ingestion while applying. If ingested, have patient seek medical attention. • The product contains acidic methacrylate monomer, and is contraindicated for use on or by persons who are sensitive to acrylates or methacrylates, or who have a history of hypersensitivity to these monomers. • Contact between this product and skin and/or sensitive or inflamed human soft tissues (mucosa, eyes, nose, etc.) may cause irritation, dermatitis or inflammatory reactions. If this occurs, discontinue use, flush the area with copious amounts of water and obtain professional medical care. • Use a rubber dam, proper intra-oral evacuation, and other suitable isolation methods to prevent contamination of restorations by saliva, blood or other fluids. • Use of some dental medications, desensitizers, liners, etc., or products with eugenol, under these materials is not recommended, as they may reduce bond strength or resin curing. • Avoid contamination of product and packages. Do not mix components with those of other materials. Do not return dispensed components to their original container. • To prevent cross infection, single-use disposable capsules and needle tips are for use on one patient only. Dispose of partially-used capsules, needle tips or empty syringes on LC BASE in accordance with all federal, state and local regulations. • Do not force the needle tip into the soft tissues, as it is not an injector. • Working and setting times and activity of these products will vary based upon temperature, humidity, age and storage conditions. Shelf life will be reduced if not stored properly. • Consult the online Safety Data Sheet (SDS) at www.parkell.com for advice on safe handling. • See our package for expiration date. Do not use after expiration date.	
<b>Kits include:</b> <b>Parkeℓℓ's LC Base Capsule Kit (S244):</b> • 15 Single-use, disposable capsules (0,25gm each / total 3,75g) in Dentin Shade. <b>Parkeℓℓ's LC Base Syringe Kit (S246):</b> • 1 Multi-use, hand-dispensed syringe (2.5ml / 4.8gm) in Dentin Shade. • 10 Luer-Lock angled dispensing needle tips (18 gauge).	
<b>General Clinical Hints on Using LC Restorative Products</b> • As with any dental treatment, the clinical requirements of the individual patient must be considered by the dental professional prior to selecting restorative materials. • LC BASE is sensitive to light. Excessive light exposure may shorten working time and result in premature polymerization. Take appropriate precautions to protect the restoration from excessive light until ready to light-cure. • If the cavity is very deep, incremental restoration or use of LC BASE and LC FILL together may be required.	
<b>Clinical Instructions for Use as a Direct Restorative</b> 1. After caries removal and tooth preparation are complete, rinse the prep with water and dry using oil-free compressed air. Disinfect and de-oil the preparation using approved cleansers and solvents. 2. Apply pulp protection to deep areas of the preparation if needed. 3. Prepare the dentinal and enamel surfaces using technique of choice, Selective Etch, Total Etch or Self-Etch. Fully cure the bonding adhesive according to the product instructions prior to placement of the composite resin. 4. Apply the desired matrix system and tooth separation device and burnish the band to allow a broad, tight interproximal contact between the tooth being restored and adjacent teeth. 5. <b>Using LC Base:</b> • If the preparation is deeper than desired, it is recommended to start the restoration with LC BASE. It is extremely flowable, and will intimately adhere to the floor of the prep. b. Dispensing i. Insert a capsule of LC BASE into a compatible gun, remove the cap, and place dispensing tip at the bottom of the cavity prep. ii. remove the cap from the LC BASE syringe, affix a dispensing tip. Be sure to secure the dispensing tip onto the syringe so that it's tight. Place the needle tip in the bottom of the deep cavity prep. c. Without withdrawing the tip, slowly extrude a layer of up to 4mm of material, making sure that the tip stays fully immersed in resin to avoid air incorporation. Leave at least 2mm of space above and around the LC BASE, as room for the LC FILL. Slowly withdraw the tip to avoid incorporating air in the base. d. Light-cure LC BASE for the appropriate time, using the table above as a guide. e. No bonding agent is required between layers of composite resin. Proceed to fill the entire preparation with a veneering composite such as LC FILL.	
<b>Clinical Instructions for Use as an Indirect Restorative</b> The clinical procedure for indirect use of LC BASE and LC FILL is similar to the direct use of the product. The restoration is fabricated on a model outside of the mouth rather than in the mouth itself.	
Materials used to make an indirect model may include plaster, dental stone or Parkeℓℓ's Mach® II or Mach-Sio™ VPS Die Silicones. Consult www.parkell.com for useful information on making indirect VPS models using Parkeℓℓ's Mach Die Silicones. If using dental stone or a plaster model, use a separating medium such as Parkeℓℓ's BluSep™ to ensure release of the restoration from the working die(s).	
<b>Disinfection of Components</b>	

- All components should be cleaned of debris with a water-damp paper towel and disinfected with a paper towel saturated with an EPA-registered low-level (HIV/HBV claim) to intermediate-level (tuberculocidal claim) hospital disinfectant. Afterwards, the chemical residue should be wiped off with a water-damp paper towel and air-dried. Utilize the surface disinfecting protocol of the disinfectant manufacturer. Consult www.CDC.gov for the most current version of the "Guidelines for Infection Control in Dental Health-Care Settings".
- Lyschard LC BASE i de korrekte antal sekunder ifølge tabellen ovenfor som guide.
- d. Lyschard LC BASE i de korrekte antal sekunder ifølge tabellen ovenfor som guide.
- e. Det er ikke nødvendigt med bondingmiddel mellem kompositplasts lag. Førtset med at fylde hele præparationen med en komposit til facader, som f.eks. LC FILL.

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