



Case Solutions® Alcohol Wipes

CASE Solutions® Alcohol Wipes Specifications

Introduction:

Case Solutions® Alcohol wipes are single use surface wipes ideal for decontaminating and drying surfaces such as counters, sinks, faucets, light switches, and sterilization containers after cleaning. Case Solutions Alcohol wipes are composed of non-woven materials pre-moistened with a highly concentrated solution of ethanol and isopropyl alcohol in high purity water.

Directions for Use:

- Remove gross soil
- Clean contaminated surface with Penta Wipes surface wipes or other Case Solutions cleaners, if visibly dirty
- Rinse to remove gross soil and detergent residues
- Follow with Alcohol Wipes to disinfect and dry surface

Storage Conditions:


- Store at room temperature (10°C to 25°C) away from heat or open flame
- Flammable—store in cool or ventilated area
- Keep container closed when not in use
- Do not reuse empty container

Compatibility:

- Non corrosive, lower toxicity
- Compatible with all metals, silicone, glass, and plastic materials
- Tested successfully for compatibility over a two (2) year period

Caution:

- Wear personal protective equipment (PPE) when handling contaminated medical devices and chemicals or cleaning contaminated surfaces
- Do not push finger through slot in lid
- Avoid contact with eyes
- Do not reuse

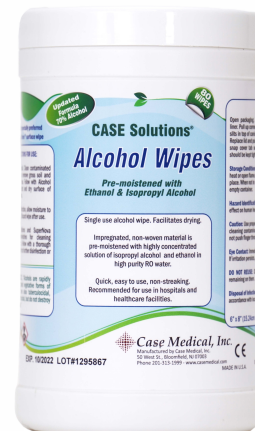

For more information, please see Case Solutions Alcohol Wipes Safety Data Sheet

CSWA002	CASE Solutions® Alcohol Wipes, 6 canisters (80 wipes each) per case
----------------	---

Case Medical, Inc.
50 West Street
Bloomfield, NJ 07003
www.casemed.com

U.S. and Canada 1-888-227-CASE
Worldwide +1 201-333-1999

CASE Solutions is a registered trademark of Case Medical.
Copyright © 2017-2020 by Case Medical, Inc.
All rights reserved. Rev C 10/20