



Case Solutions® Alcohol Wipes

CASE Solutions® Alcohol Wipes Specifications

Introduction:

Case Solutions® Alcohol wipes are single use surface wipes ideal for drying and decontaminating surfaces after cleaning. Case Solutions Alcohol wipes are composed of non-woven materials pre-moistened with a highly concentrated solution of ethanol and isopropyl alcohol in high purity water.

Directions for Use:

- Remove gross soil
- Clean contaminated surface with Penta Wipes surface wipes or other Case Solutions cleaners
- Rinse to remove gross soil and detergent residues
- Follow with Alcohol Wipes to disinfect and dry surface of non-critical items

Storage Conditions:

- Store at room temperature (10°C to 25°C) away from heat or open flame
- Flammable—store in cool or ventilated area
- Keep container closed when not in use
- Do not reuse empty container

Compatibility:

- Non corrosive, lower toxicity
- Compatible with all metals, silicone, glass, and all plastic materials
- Tested successfully for compatibility over a two (2) year period

Caution:

- Wear personal protective equipment (PPE) when handling contaminated medical devices and chemicals or cleaning contaminated surfaces
- Do not push finger through slot in lid
- Avoid contact with eyes
- Do not reuse

For more information, please see Case Solutions Alcohol Wipes Safety Data Sheet

CSWA002

CASE Solutions® Alcohol Wipes, 6 canisters (80 wipes each) per case

Case Medical, Inc.
50 West Street
Bloomfield, NJ 07003
www.casemed.com

U.S. and Canada 1-888-227-CASE
Worldwide +1 201-333-1999

CASE Solutions is a registered trademark of Case Medical.
Copyright © 2017-2020 by Case Medical, Inc.
All rights reserved. Rev C 10/20