




LIGHTFORCE™

ORTHODONTICS

The World's Only, Fully-Customized,
3D Printed Bracket System

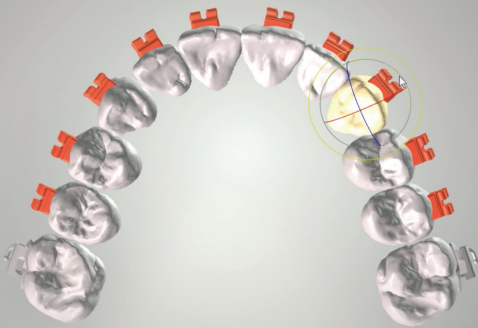

Connecting LightForce Orthodontics directly to your 3Shape® scanner.

If you are using a 3shape scanner you can simplify the submission process by having LightForce Orthodontics added to your menu.

Go to 3shape communicate, log in, click connections, add connection, and search for **ops@lightforceortho.com** making sure the "search for labs" box is checked.

Request connection. Lightforce Orthodontics will get an email and will confirm the connection.

*Once it is confirmed you can upload the case directly through your 3shape scanner.

**Remember, a treatment plan must still be created on the LightForce doctor portal to complete the case submission requirements. The internal team will link your scans to the completed treatment plan*