



WHILL Model Ci: Airline Handling Guide

For proper airport personnel handling of WHILL Model Ci personal mobility device.
Attach this document to device in a conspicuous area.

Owner's Contact Information:

Name: _____

Phone: _____

Device Serial #: _____



Battery

Type: Rechargeable Lithium-ion

Model: WHILL 210-06865

Specs: 25.2 V; 253.01 Wh

Other: Removable from device. Requires removal and storage in cabin for airline travel.

1

Freewheel Mode

Disengage brakes by pushing down the two red brake levers on the bottom of the device. Fold back support down.

2

Lifting Device

Only lift the device using the anti-tip bars on the back and handle on front of footplate

3

Engage Brakes

When the Model Ci reaches the cargo area, load device in upright position, then lock the brakes by lifting the two red levers up. This will prevent the device from shifting in-flight.

At destination, safely return the device to the owner at the gate by following steps 1 and 2 in reverse order.

WHILL rechargeable lithium-ion battery 210-06865 has been tested on UN Manual of Tests and Criteria Part 3 Subsection 38.3 and certified to meet the applicable ICAO, IATA and FAA requirements. However, WHILL Model Ci personal mobility device has not been specifically tested for air transport.