



WHILL Model Ci: User Handling Guide for Air Travel

For proper user handling and preparation of WHILL Model Ci personal mobility device during airline travel. Keep this document on hand for quick reference.

Flight Departure: Handing Off to Airport Personnel

- 

1

Power Down

Turn Model Ci off, then completely rotate both arms up.
- 

2

Document

Take pictures of the device in case damage occurs during transport. Wrap arms in bubble wrap to help prevent scratches (optional).
- 

3

Remove Battery

Remove the Li-ion battery from the compartment. Place in non-conductive bag and store in cabin's overhead bin. DO NOT forget the battery key if you often lock the battery in the compartment.

Flight Arrival: Receiving and Driving

- 

1

Inspect

Inspect your Model Ci upon receipt. Document any damage that occurred during transportation.
- 

2

Engage Brakes

Pull the two red brake levers up to re-engage the brakes (located below the seat).
- 

3

Drive

Sit in Model Ci and rotate arms down into driving position. Turn on device.

WHILL rechargeable lithium-ion battery 210-06865 has been tested on UN Manual of Tests and Criteria Part 3 Subsection 38.3 and certified to meet the applicable ICAO, IATA and FAA requirements. However, WHILL Model Ci personal mobility device has not been specifically tested for air transport.